

## Precautions:



- ⚠ **Caution:** Do not tug on the EyeMax™ to reposition it.  
A premature infant's skin is very delicate and can easily rupture.
- ⚠ **Caution:** Avoid obstructing the nose. Check regularly to ensure the EyeMax™ is not blocking the patient's airway.
- ⚠ **Caution:** Do not over tighten – causing excessive pressure on the eyes.
- ⚠ **Caution:** The EyeMax™ is not a bandage. Cover any existing skin abrasions with appropriate bandages before use.
- ⚠ **Caution:** Make sure the strap is covering the ear, not hooked over the ear, and that the ear is not folded over.
- ⚠ **Caution:** Replace the EyeMax™ with a new one if it becomes over-stretched.

Catalog # R300P01 **Regular:** occipital-frontal circumference **33-38** cm in packs of 20.

Catalog # R300P02 **Preemie:** occipital-frontal circumference **26-32** cm in packs of 20.

Catalog # R300P03 **Micro:** occipital-frontal circumference **20-25** cm in packs of 20.

Maxtec® Inc., 6526 South Cottonwood Street, Salt Lake City, Utah 84107

Tel 801.266.5300 • Fax 801.270.5590 • Toll Free 800.748.5355 • [www.maxtecinc.com](http://www.maxtecinc.com)

**EU Representative:** Airox S.A. • Parc d'activités Pau-Pyrénées • L'échangeur BP 833 • 64008 PAU Cedex, France




# Micro: OFC 20-25 cm (8-10 in.)



## Instructions for use:

1. Disconnect the Velcro.
2. Position the eye pads over the eyes.
3. Pass the lower strap under the small of the neck and attach velcro.
4. Secure the Velcro of both straps and position the EyeMax2™ as shown.
5. Make sure the strap completely covers the ears.
6. If the eye patch rides up away from the nose, tighten the lower strap.

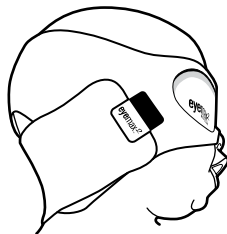

**Single Patient Use Only**

**Non-Sterile, Latex-Free**