




## FRONT

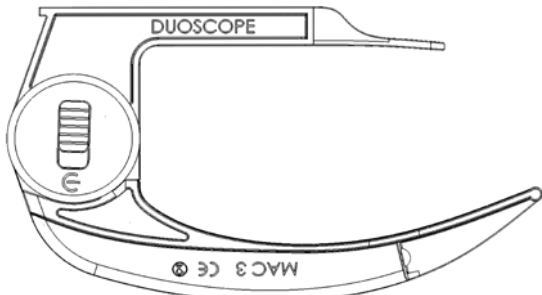

**Mediplas Pty Ltd** Head Office & International Department:  
12-16 Redland Drive, Mitcham VIC 3132 Australia  
Phone: +61 (0)3 9873 7233 Fax: +61 (0)3 9873 8877  
e-mail: sales@mediplas.com.au Web: www.mediplas.com.au

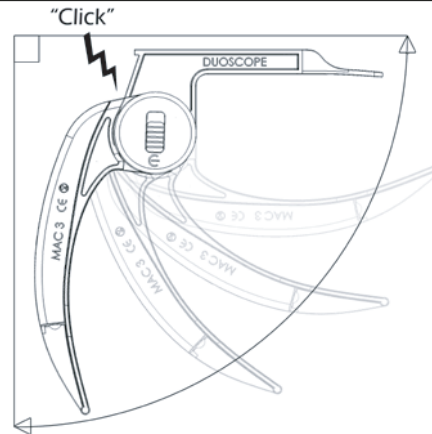


Extend the blades beyond 90 Degrees,  
until they lock into place.

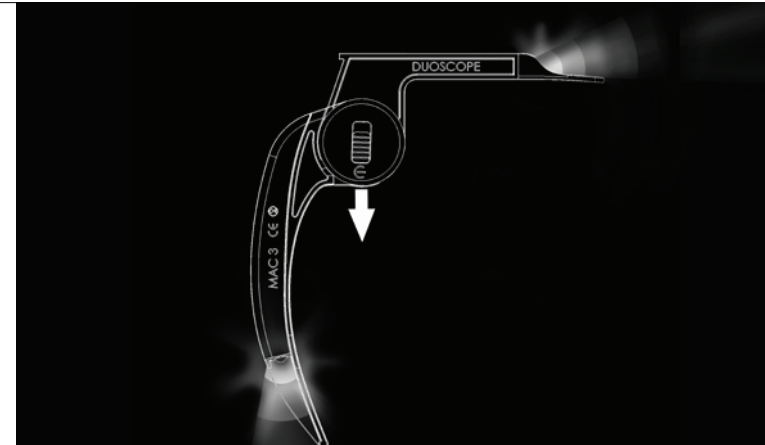

Activate LEDs by  
moving switch to ON (☺) POSITION

## BACK

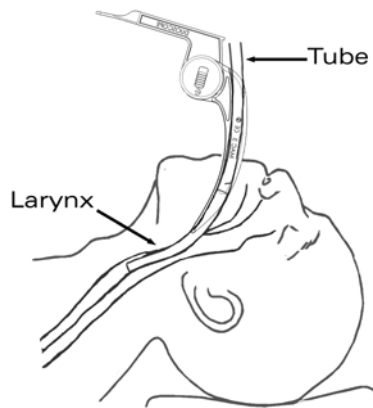


Intubate patient using the  
appropriate blade



Adult being intubated - Mac 3/ Mac 4 Blade

Infant being intubated - Miller I blade

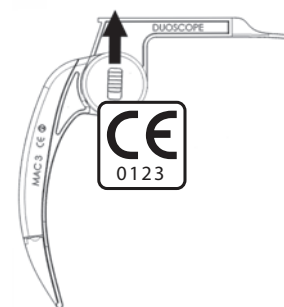

Switch off and discard Duoscope in accordance  
with local disposal laws. Batteries contain Lithium.