

# Disposable Oxygen Hoods/Tents

Part Number 2310000 to 2310134

## Assembly and General Instructions

- 1) Remove oxygen hood/tent from the packaging.
- 2) Unfold oxygen hood/tent and remove press-out discs from the 22mm tubing ports where present. In certain hoods/tents such as the Double-Tent Outer, the discs have been left in place. In other models they have already been removed.
- 3) Snap the back panel into the body section using all the snap connectors. Note: If the tent is a No-Bottom, NICU or Partial Bottom design, the snap connections on the bottom edge of the back panel are not used.
- 4) The three 22mm tubing ports in the back panel of the Outer Double-Tent are available for vacuum and aerosol/humidity tubing. Note: Tubing, connectors, and filters of the user's choice will need to be added.
- 5) Assembled hoods/tents should be used according to the hospital's protocols.

**Note:** Bedding should be placed between the patient and the hood/tent bottom where it exists.

**Note:** Oxygen Hoods should only be used in conjunction with oxygen monitoring.

**CAUTION:** Prior to placing the patient inside the hood/tent, the hood/tent should be inspected to ensure that the press-out discs have been removed from the hood/tent and the hood/tent area. Ensure that all the snap connections are in working order.

**CAUTION:** No smoking, no use of electrical appliances, no open flames should be allowed when oxygen is in use; only approved toys should be given to the patient.

**CAUTION:** It is important to regularly monitor the patient's physical position within the hood.

**CE0086**