VIAMED

COMPANY OPERATING PROCEDURE

VM2500-M Contents & Testing VM3/COP/54.00 4410500

Date:22/01/10	Revision Date:7-Dec-10	Issue: 1

VM2500-M 4410500 Contents & testing:

1 x VM2500-M

(blue casing) fitted with silicone cover Document the serial number and the software version and type, i.e. VM-2500-M, Version number, Europe, CD Type/Version

Check front panel is VM-2500-M, VIAMED

Check serial label is Viamed version

1 x IRMA CO2 analyser P/N: 4420505

Need to document the serial number and check that ref. is 200104

1 x IRMA airway adapter (adult/paediatric).

Check that Phase-In version and not Phase-In/ Smiths BCI. Our part number for this 4420512. If Phase-In/Smiths BCI replace with our version from stock. Single Item p/n: 4420512 (Multi Pack: 4420511)

1 x Reusable SpO2 sensor

Needs to be shipped, type as per the customer order, standard will be 0014650. Ship as a separate item on the order at zero value.

1 x USB data cable.

1 x Power supply

Type FW7660M/06

- 1 x Power supply plug adapter (EU)
- 1 x Power supply plug adapter (UK)
- 1 x Li-ion battery (CT-2500)
- 4 x AA batteries

1 x CD with user manual...

Document the CD version

1 x Quick start guide.

Carry out functional tests use test sensor and test IRMA adapter.

Check time and date, should need to change the time by 1 hr.

Delete (test) data.

Packaging sleeve needs large bar-coded label and colour dot affixing.

Must be Q.A. Into Intrastats. Probe will be merged automatically on the Q.A. Screen