VIAMED

COMPANY OPERATING PROCEDURE

VM2500-S Contents & Testing VM3/COP/54.01 4410520

Date:22/01/10	Revision Date:7-Dec-10	Issue: 1
Datc.22/01/10	TCVISION Date./-Dee-10	13300. 1

VM2500-S 4410520 Contents & testing:

1 x VM2500-S

(yellow casing) fitted with silicone cover Document the serial number and the software version and type, i.e. VM-2500-S, Version, Europe

Check front panel is VM-2500-S, VIAMED

Check serial label is Viamed version

1 x Nomo adapter P/N:4420525

1 x Disposable sampling line

Need to add from our stock 4420551

1 x Airway adapter (adult/paediatric)

Looks like a clear T piece 4420532

1 x Reusable SpO2 sensor

Type as per the customer order, standard will be 0014650. Ship as a separate item on the order at zero value.

1 x USB data cable.

1 x Power supply

Type FW7660M/06

- 1 x Power supply plug adapter (EU)
- 1 x Power supply plug adapter (UK)
- 1 x Li-ion battery (CT-2500)
- 4 x AA batteries

1 x CD with user manual...

Document the CD version

1 x Quick start guide.

Cary out functional tests use test sensor and test IRMA adapter.

Check time and date, should need to change the time by 1 hr.

Delete (test) data.

Packaging sleeve needs large bar-coded label and colour dot affixing.

Must be Q.A. Into Intrastats.