




VIAMED

ADD
TO MANUAL



TOM THUMB INFANT RESUSCITATOR

TOM THUMB INFANT RESUSCITATOR DATA SHEET

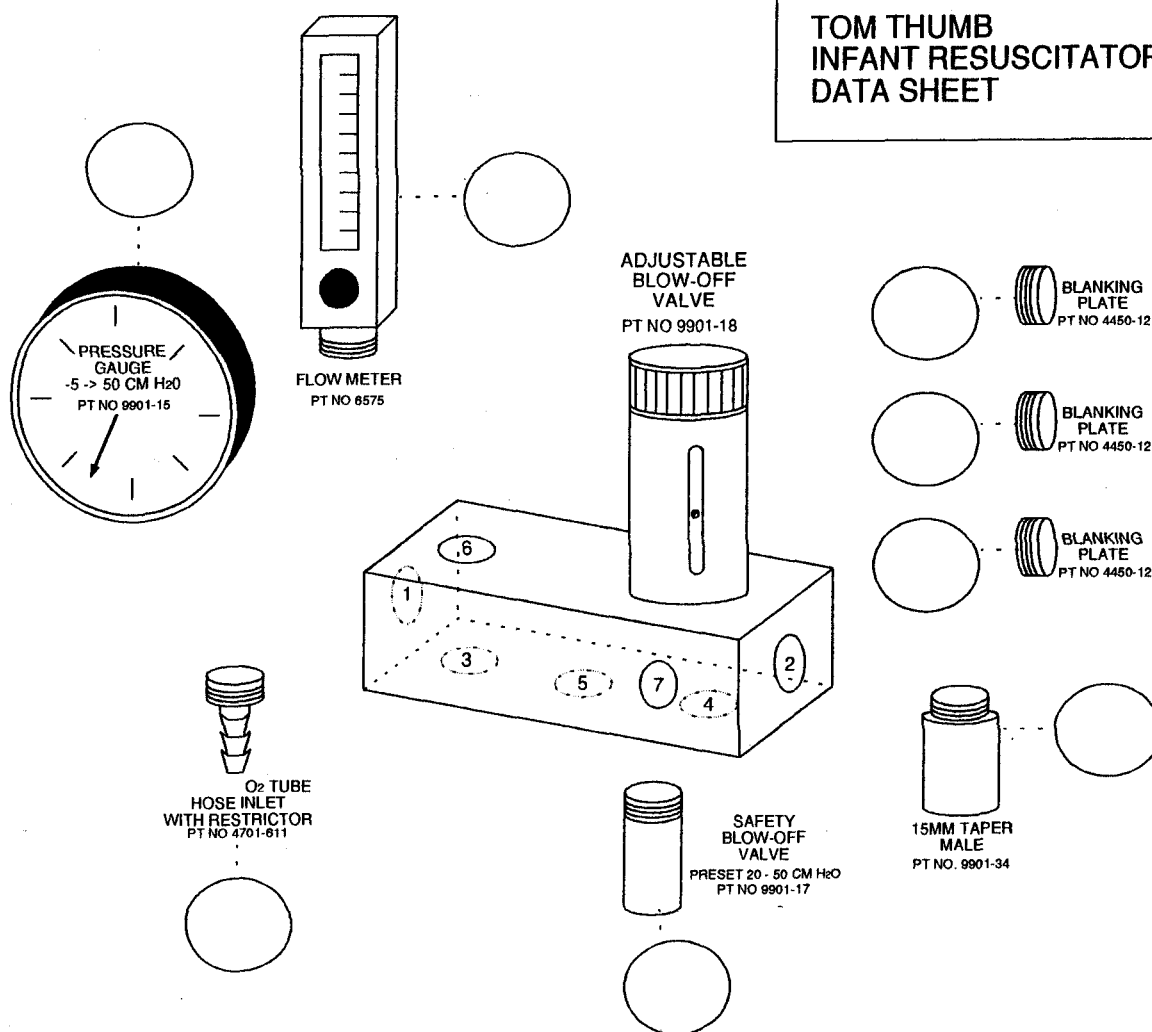

Please specify MAX Pressure for Safety blow-off valve PT. NO. 9901-17

50 [] 40 [] 30 [] 20 [] Other []

TO ORDER

Choose layout of components and number the circle with the hole number.

Limitation:

Flowmeter PT. NO. 6575 must be vertical in port '6'.

Adjustable Blow-off valve must be vertical in the position shown above.

Safety Blow-off valve PT. NO. 9901-17 must be in a vertical position on the bottom face of the block.

Contact

Title

Department

Hospital

Tel. No.

Additional Information:

Order Number:

Invoice Number:

Serial Number:

Q/A


Date:

Viamed Ltd. 15 Station Road, Cross Hills, Keighley, West Yorkshire BD20 7DT


Telephone: 0535 634542 / 636757 Fax: 0535 635582

Registration No. 1291765 in England

9901-04 4 7/3

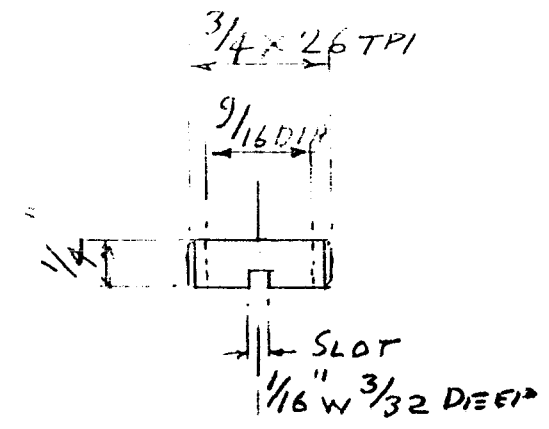


MATL BRASS

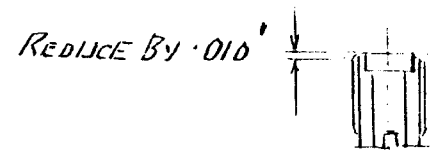


STD PARTN° 4740-41

990-05

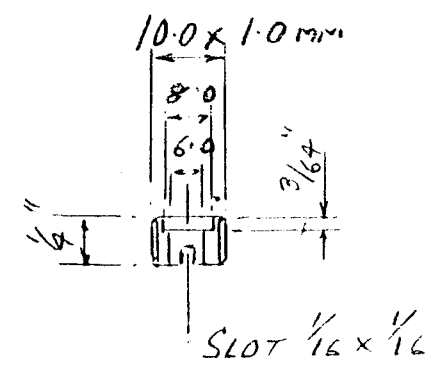

STD PARTN° 4740-41
CAN BE MODIFIED

9901-06

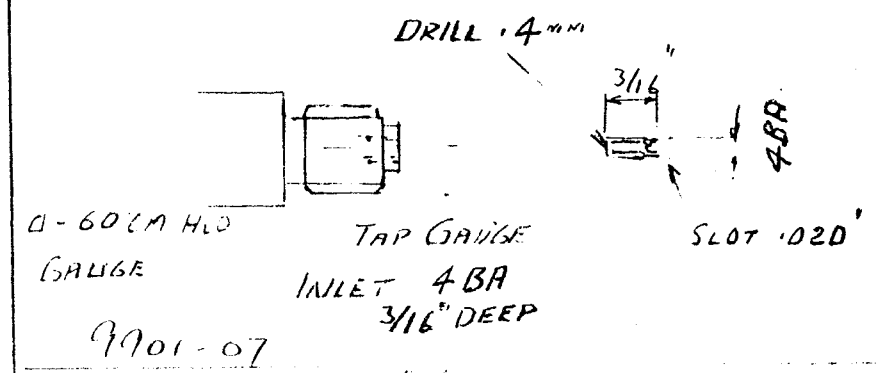


0-60 CM HLD
GAUGE


9901-07

ISSUE 4 3-3-97
ISSUE 3 11-11-93
ISSUE 2 1-8-97

9901-04 4 7/3

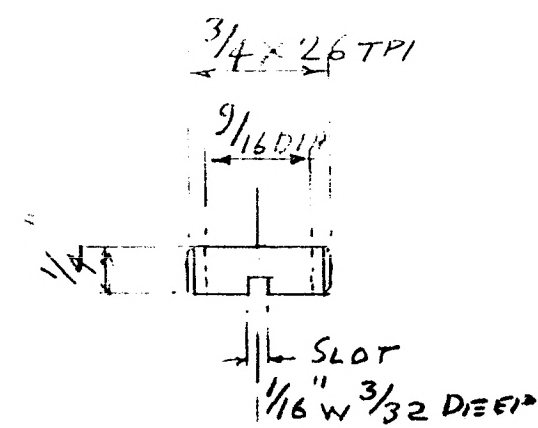


MATL BRASS

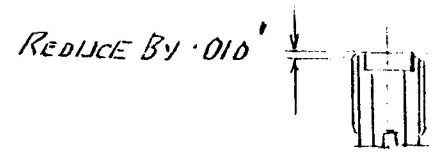


STD PART N° 4740-71

9901-05

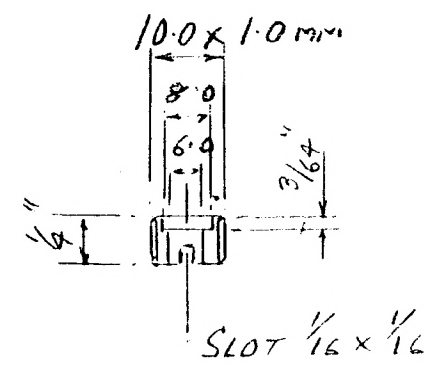







STD PART N° 4740-71
CAN BE MODIFIED

9901-06

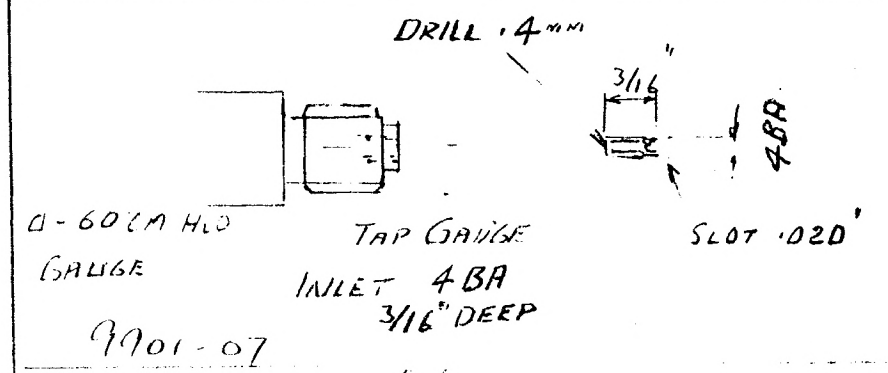








Q-60 CM HD
GAUGE
TAP GAUGE
INLET 4BA
3/16" DEEP
9901-07

ISSUE 4 3-3-97
ISSUE 3 11-11-03
ISSUE 2 1-8-97

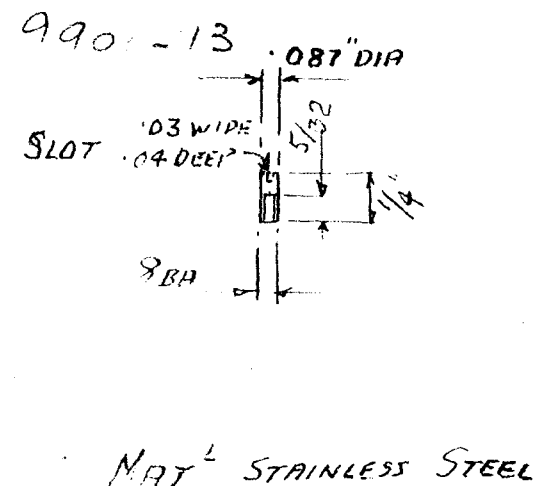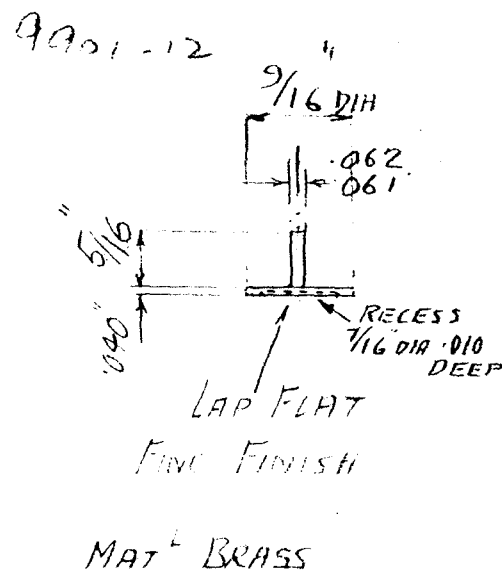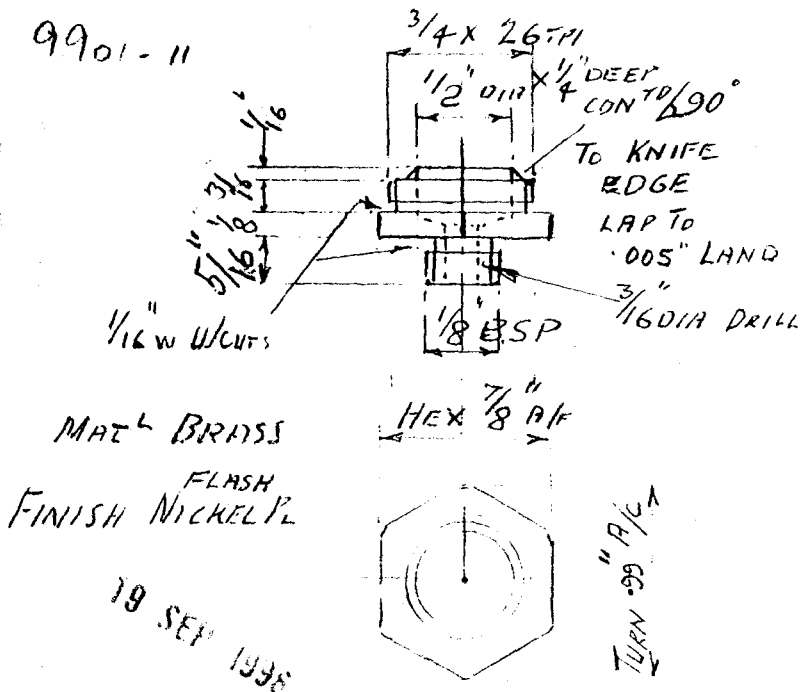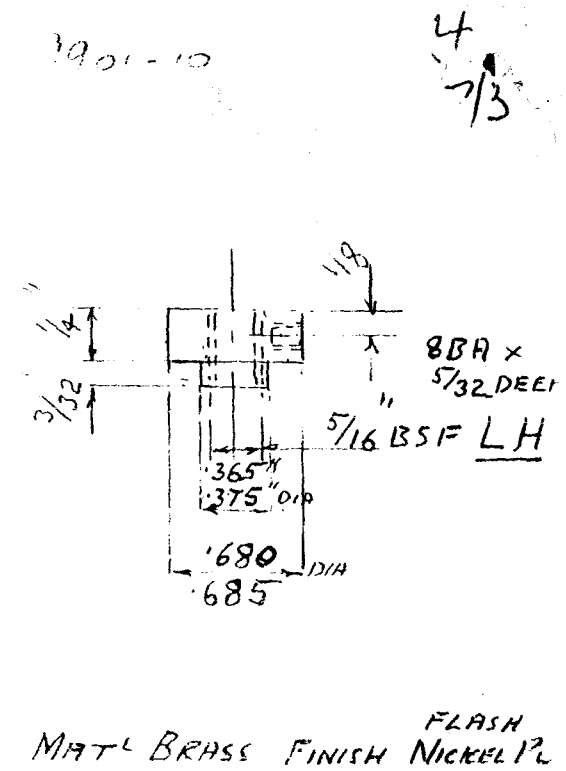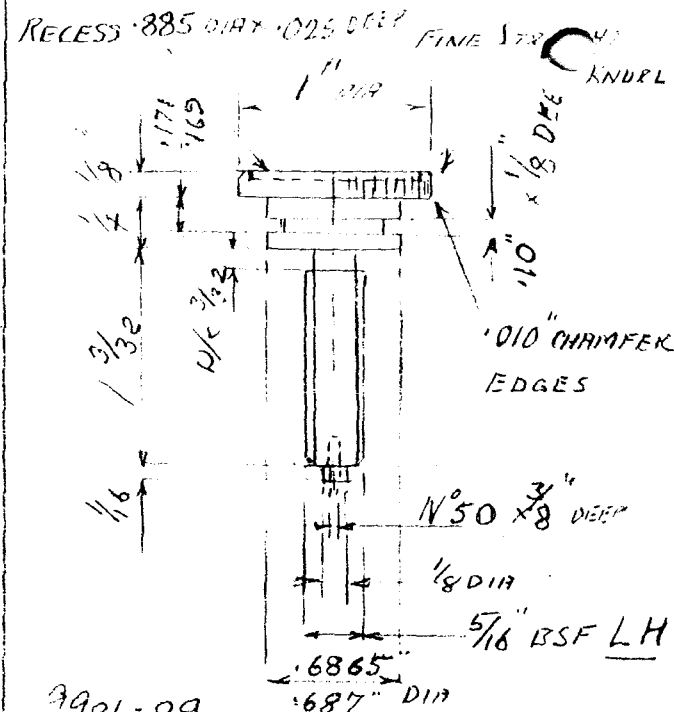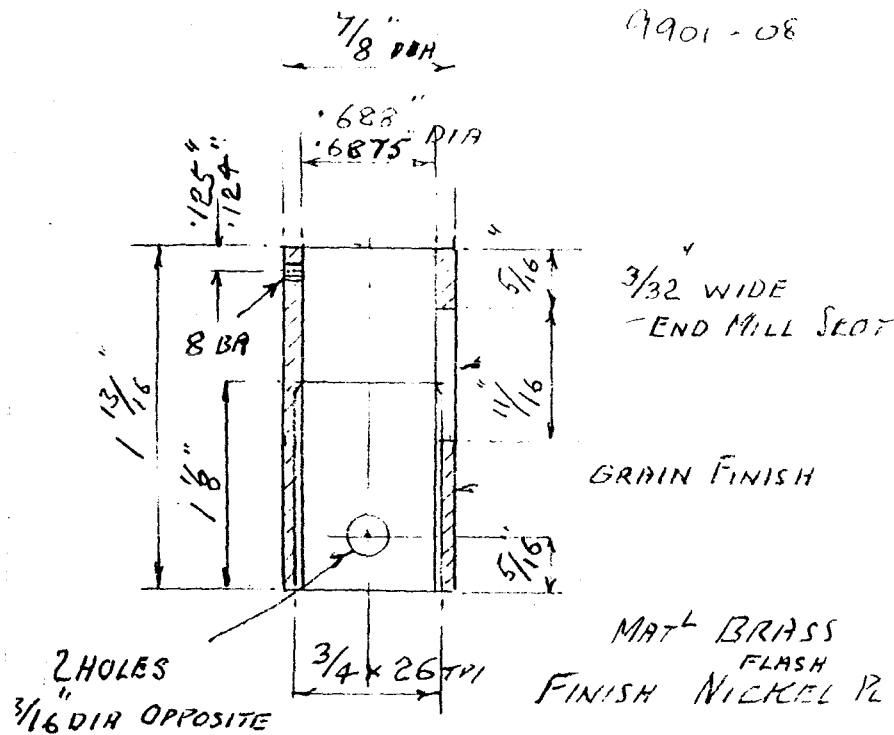


ISSUE 4 3-3-93
ISSUE 3 11-11-93
ISSUE 2 1-8-93

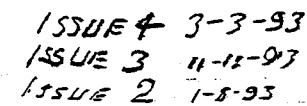

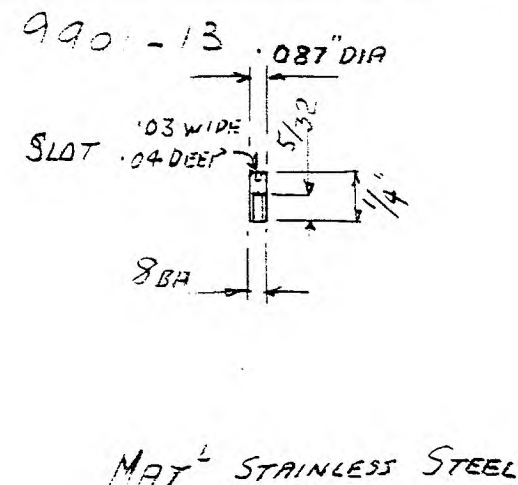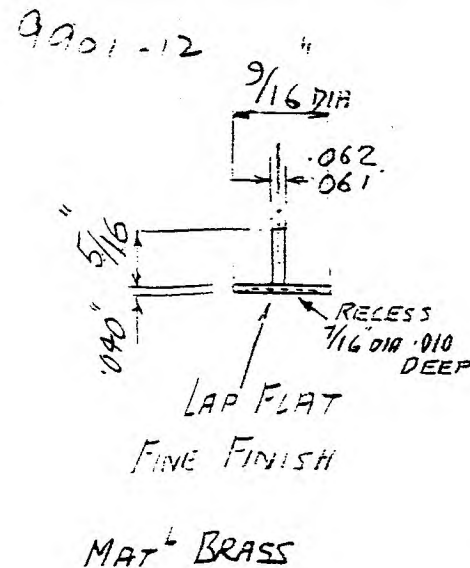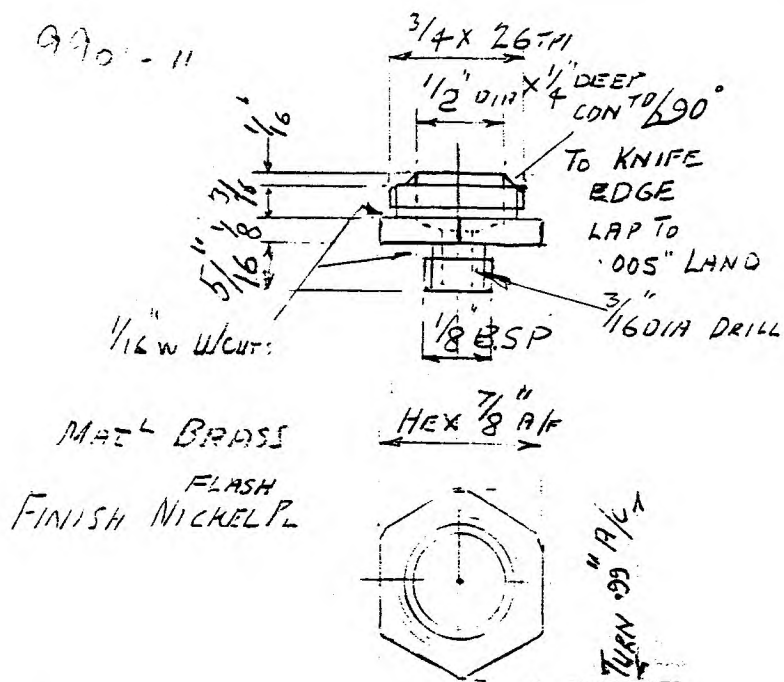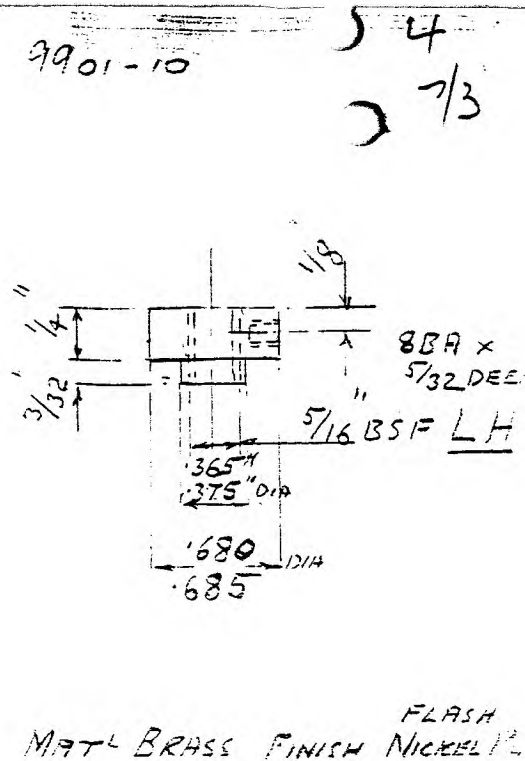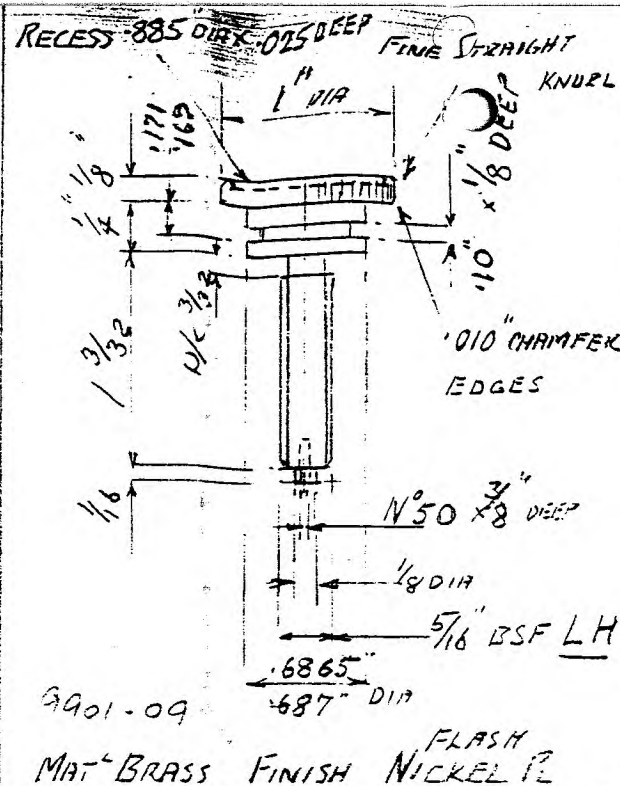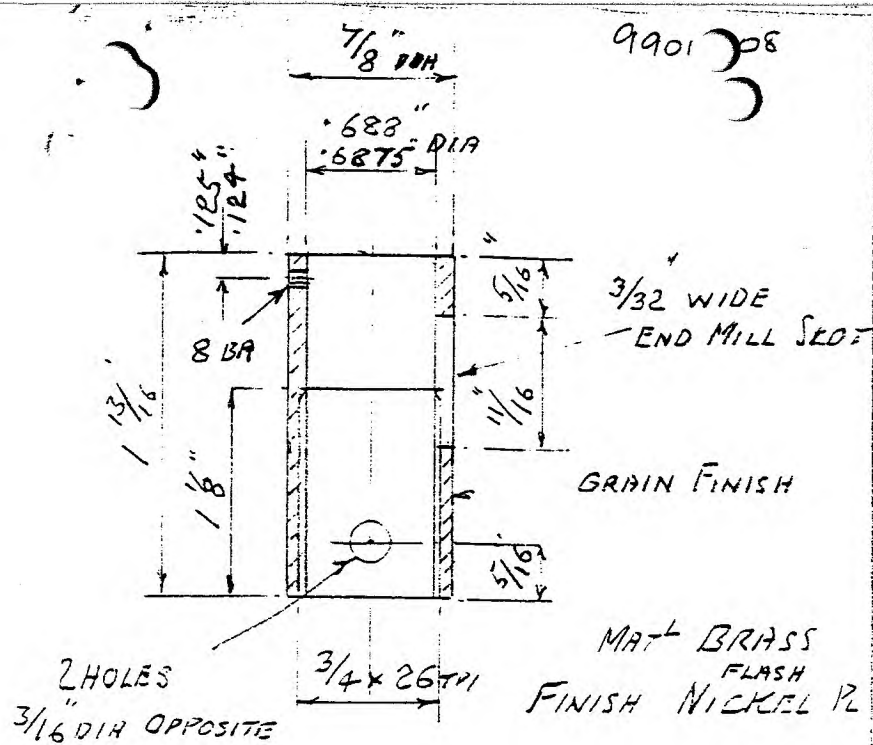


ISSUE 4 3-3-93
 ISSUE 3 11-12-93
 ISSUE 2 1-8-93

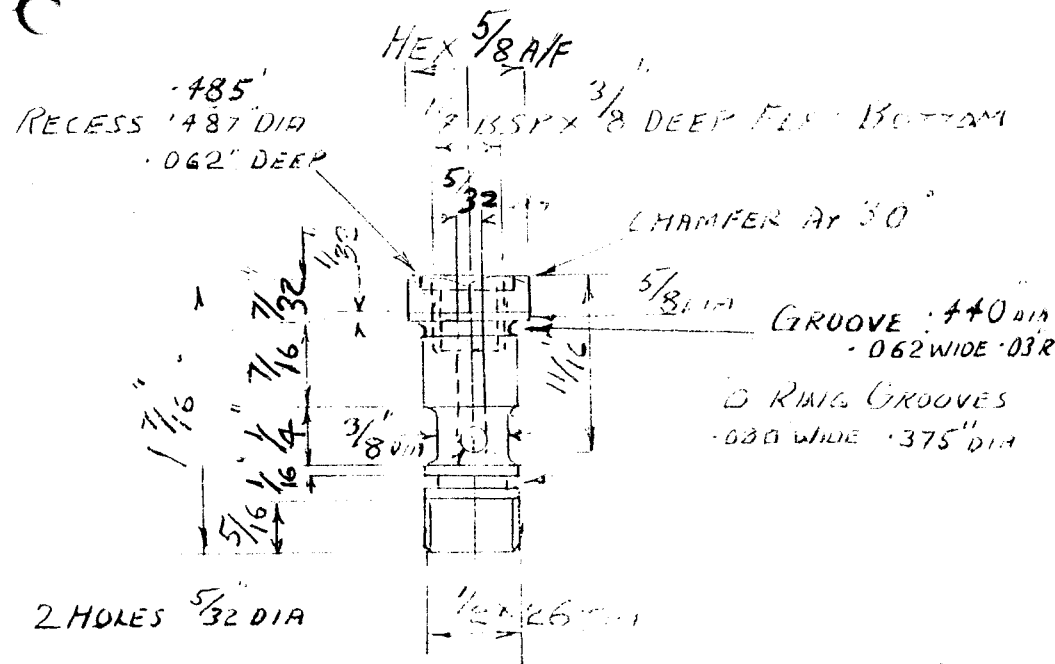

MAT^L BRASS FINISH NICKEL PL

9901-31


CHAMFER AT 30°

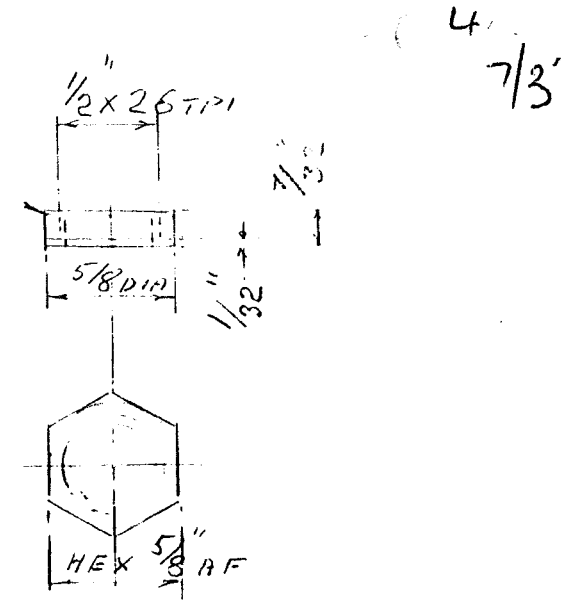

9901-24

MAT^L BRASS FINISH NICKEL PL

9901-14

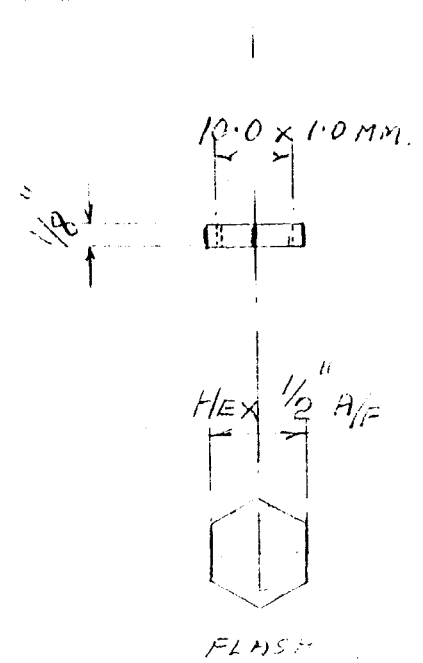

MAT^L BRASS FINISH NICKEL PL


50 CM H2O GAUGE

- O RINGS 4701-18 1 OFF
- 4701-54 2 OFF
- 4450-13 5 OFF 4.1 X 1.6
- 7/16\"/>

ISSUE 4 3-3-94
ISSUE 3 11-11-93
ISSUE 2 1-8-93

19 SEP 1998

VIAMED.
10 OFF.

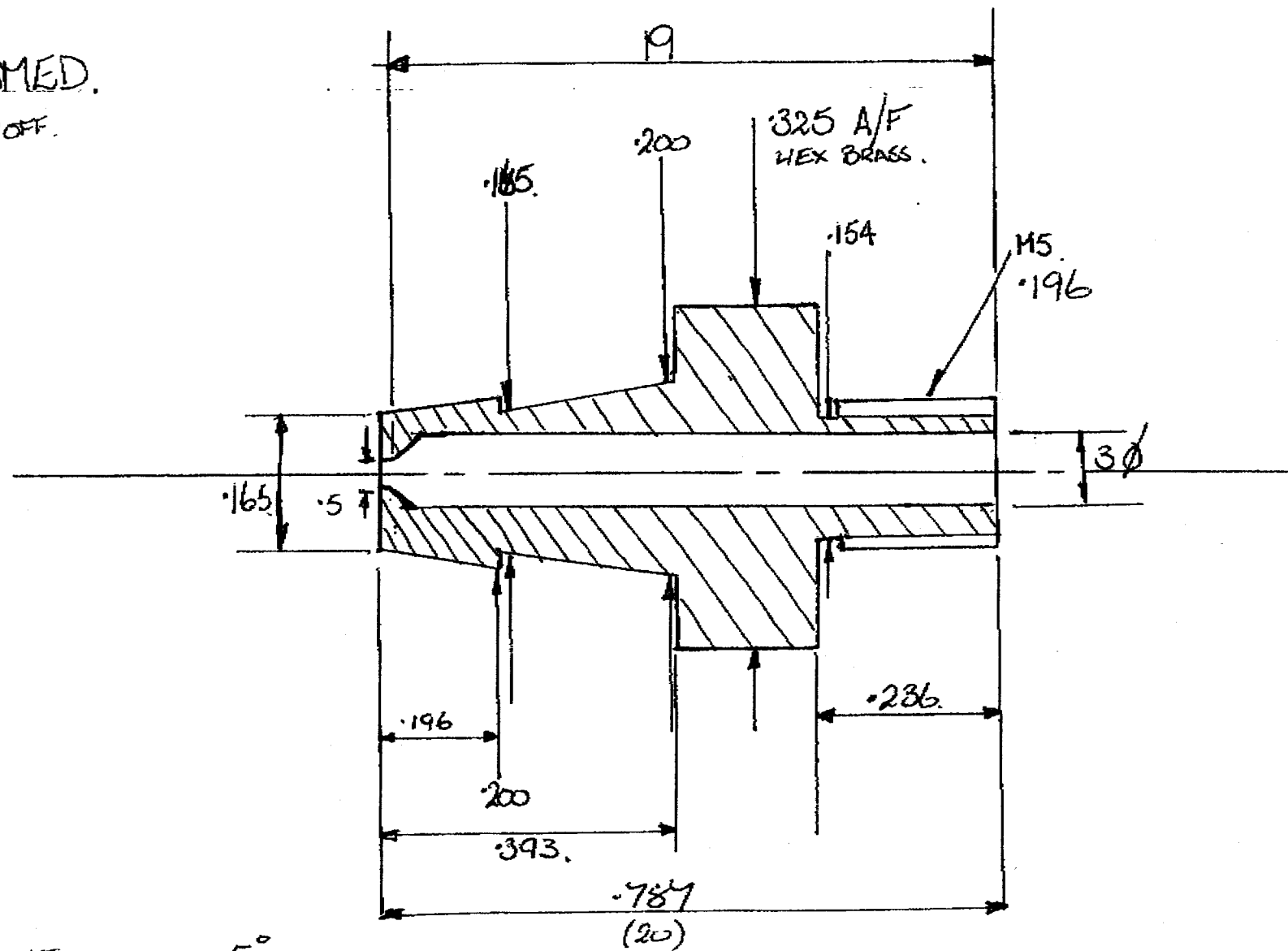


$$\tan B = \frac{0.0175}{0.196} = 0.08928 \quad 5^\circ$$



CLY
MATERIAL 2.
REPAIRS
IN LADDER

VIAMED.
10 OFF.

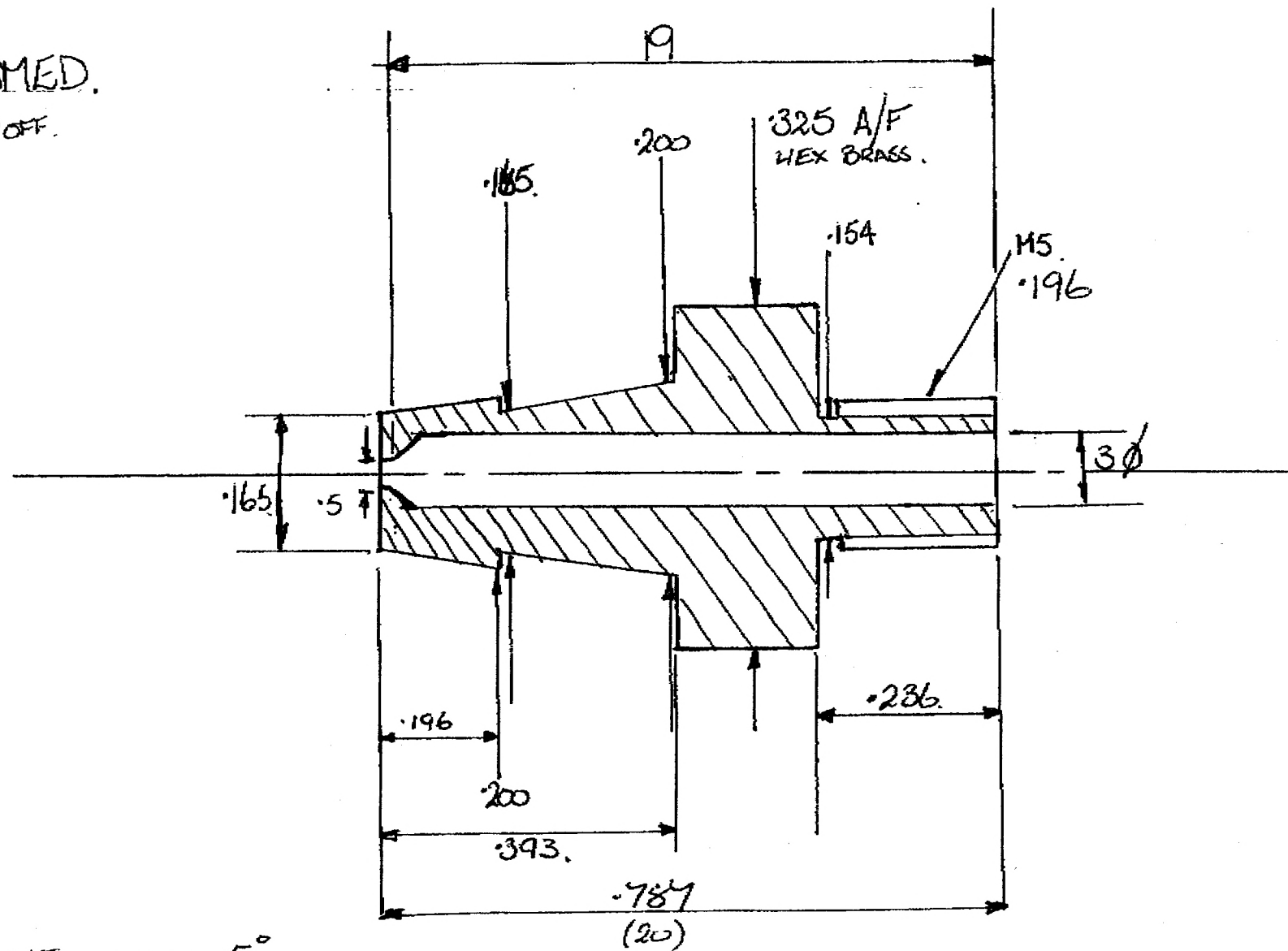


$$\tan B = \frac{.0175}{.196} = .08928 \quad 5^\circ$$



CC7
2002.1 copy Phase 2,
Especially
invaluable

RECEIVED FROM

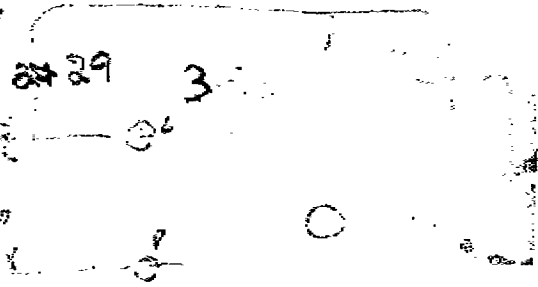

HUGGARD'S
GUN CLUB OFFSET AND SCOTT'S MOUNTERS

5 35

30 5

50

365

15

PAVED - CROCK - 100 - 100



23

M5




16

M3



39

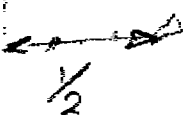

1/2

5

30 5


M5 35 100 100 100

M3 12 PAN LEAD OFF

M5 12 100 100

100 100 100

50
9/3/52

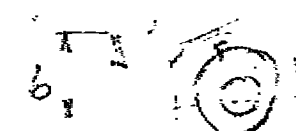

6


M3

30 5

M3

30 5

48


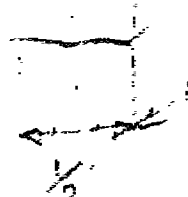


1/2


5/8 1/2

100 100 100
100 100 100


RECEIVED FROM

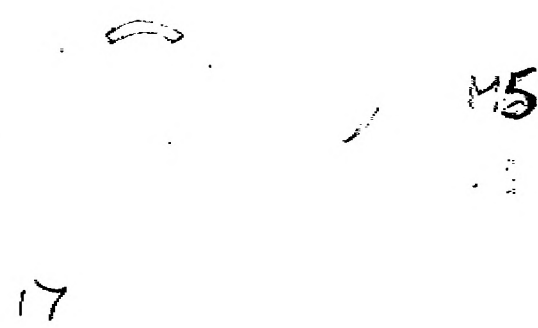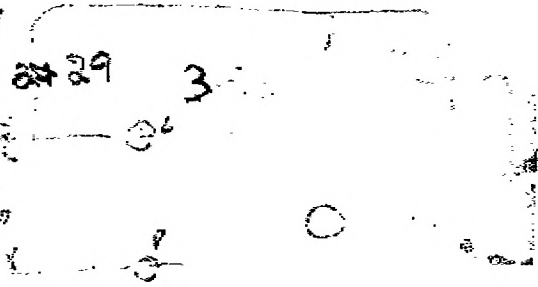

HUGHES 1007
ON CLIP OFFSET AND SCRIE HUNTERS


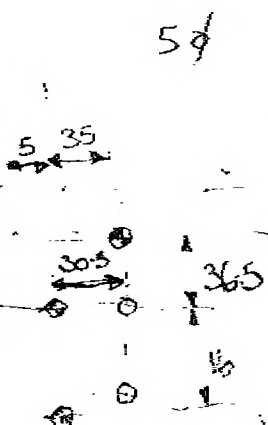


MAILED - CHECK FOR (C)

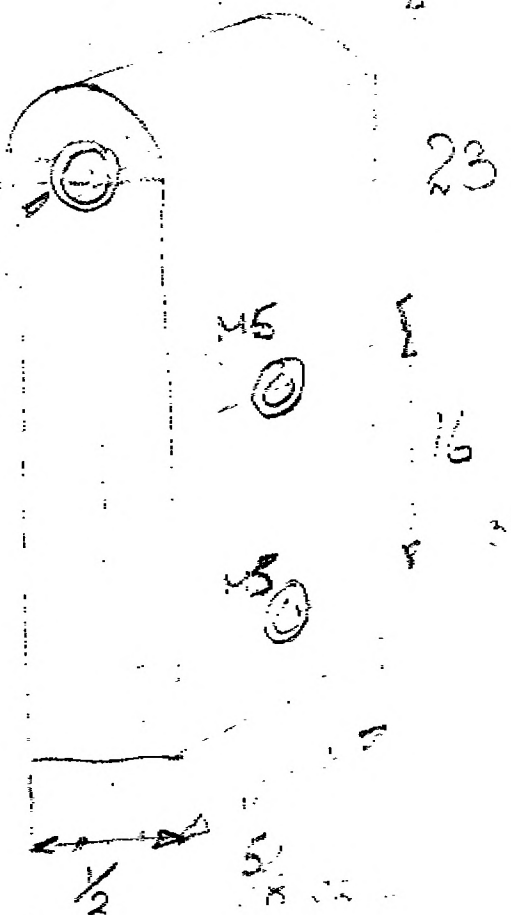


M5 35
M3 12 PAN LEAD OFF
M5 12
=

50
9/3/52

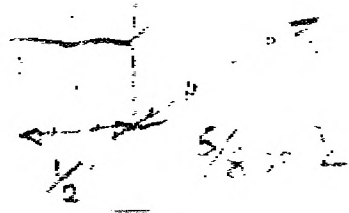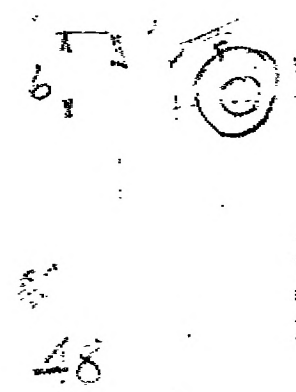


10/11/52
10/11/52


MAT^L BRASS
FINISH NICKEL PLATED

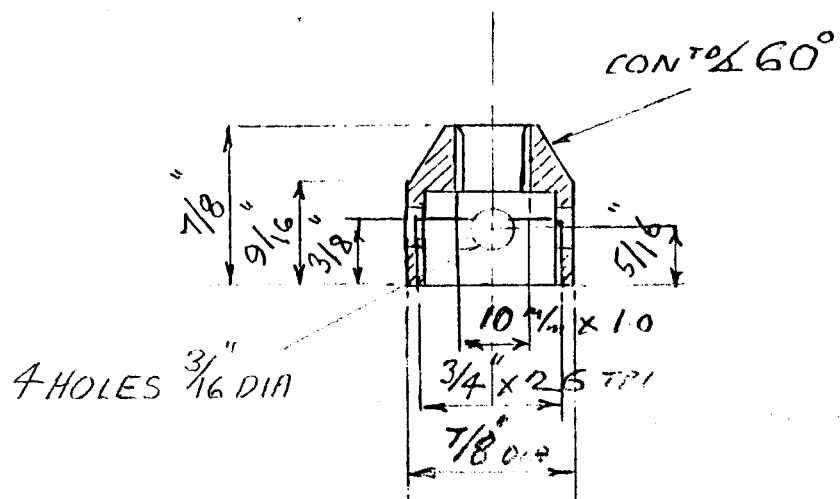

APPLY KILOPOISE 0001G TO SPINDLE

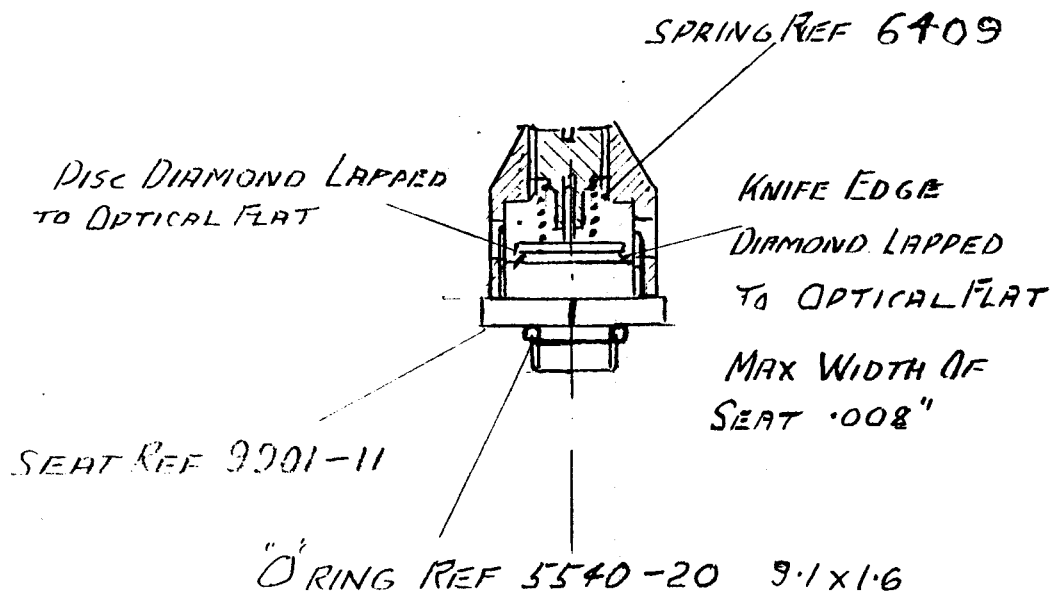


MAT^L BRASS
FINISH FLASH NICKEL

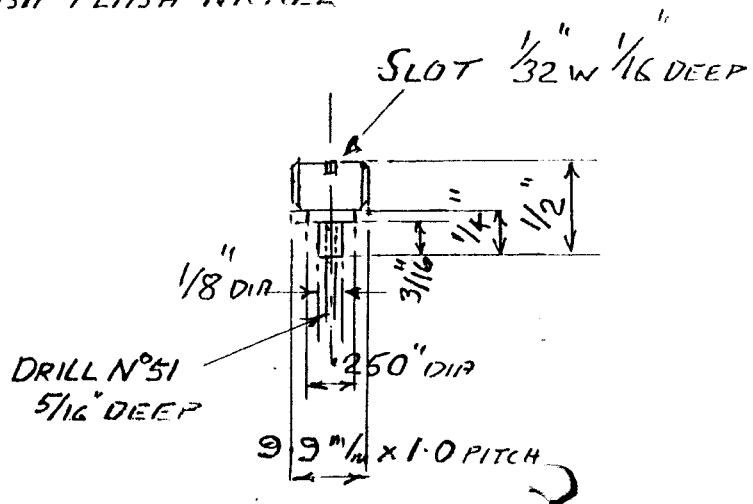


SET TO 30 CM H₂O AT 5 L/MIN FLOW


MAT^L BRASS
FINISH NICKEL PLATED

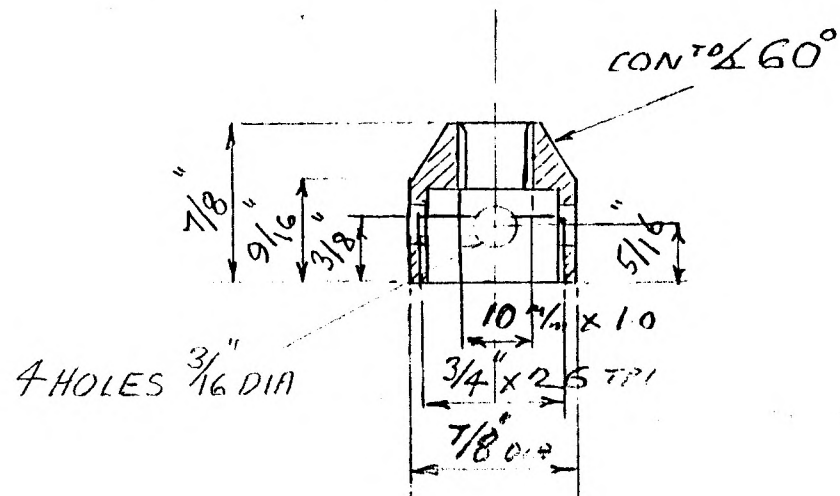

MAT^L BRASS
FINISH FLASH NICKEL

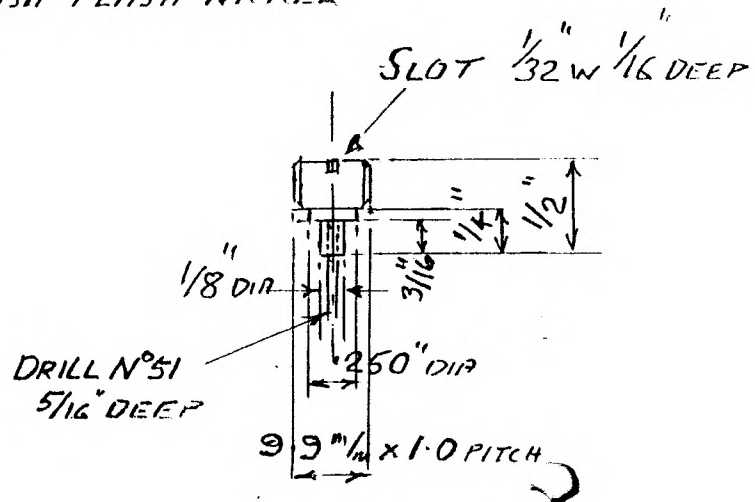


APPLY KILOPOISE 0001G TO SPINDLE

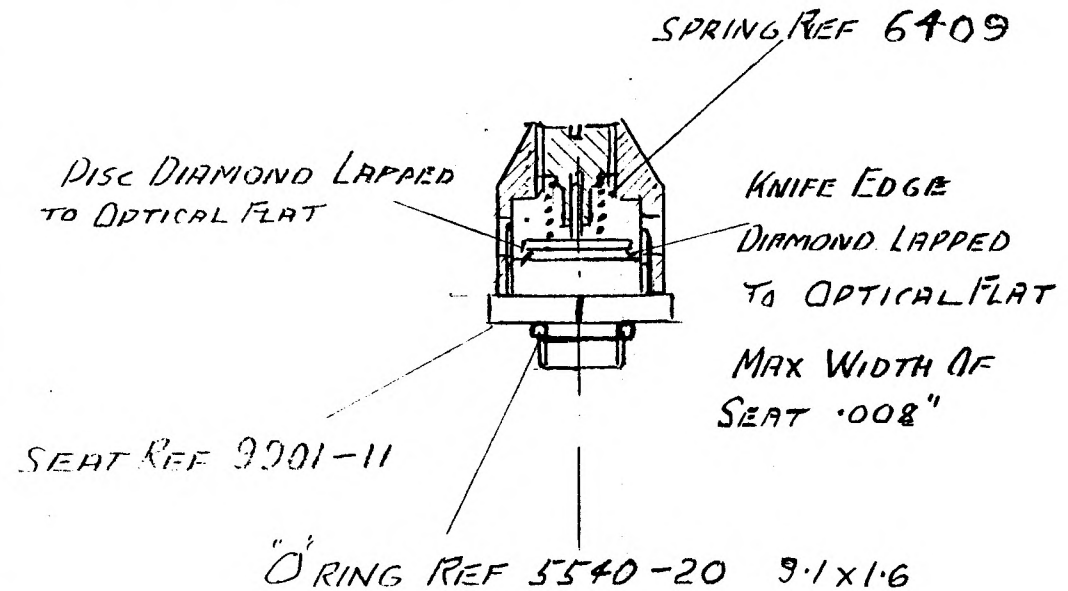


SET TO 30 CM H₂O AT 5 L/MIN FLOW

9901-03

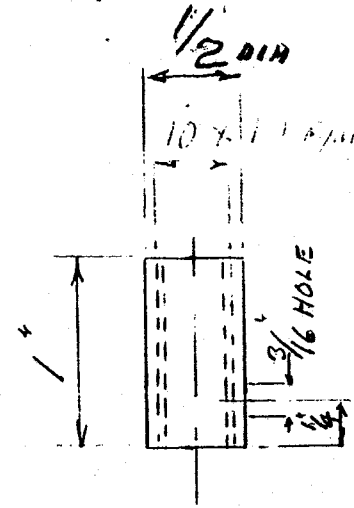


MAT^L BRASS FINISH NICKEL PL

9901-04

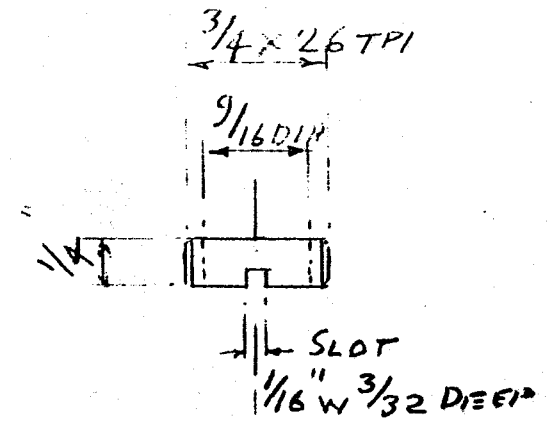


MAT^L BRASS

REDUCE BY .010"

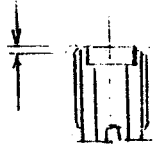


STD PARTN° 4740-41

9901-05

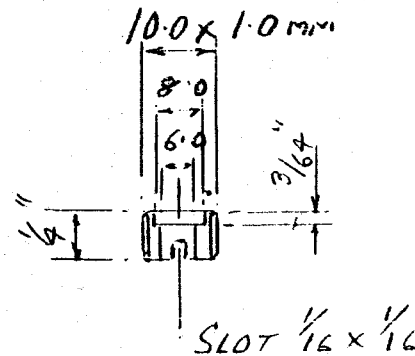


STD PARTN° 4740-41
CAN BE MODIFIED

9901-06

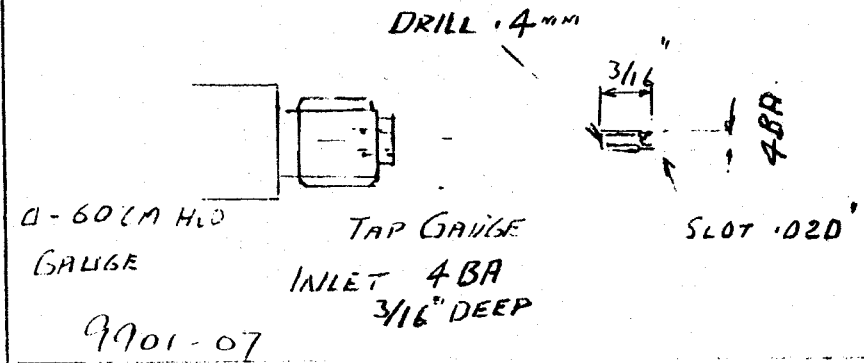


ISSUE 4 3-3-97
ISSUE 3 11-11-93
ISSUE 2 1-8-97

9901-03

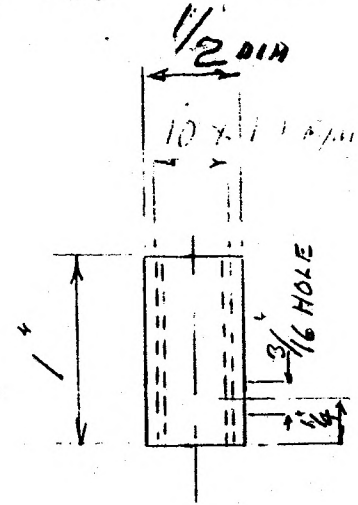


MAT^L BRASS FINISH NICKEL PL

9901-04

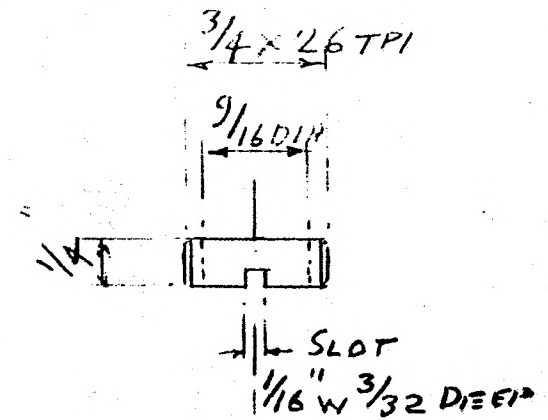


MAT^L BRASS


REDUCE BY .010

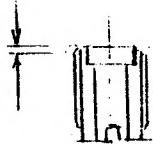

STD PARTN° 4740-41

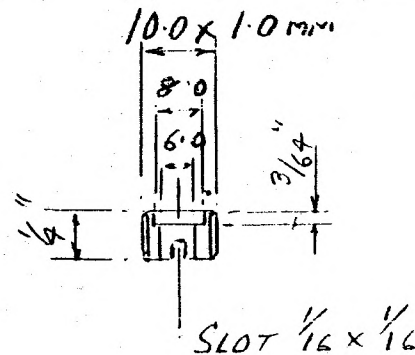

STD PARTN° 4740-41
CAN BE MODIFIED

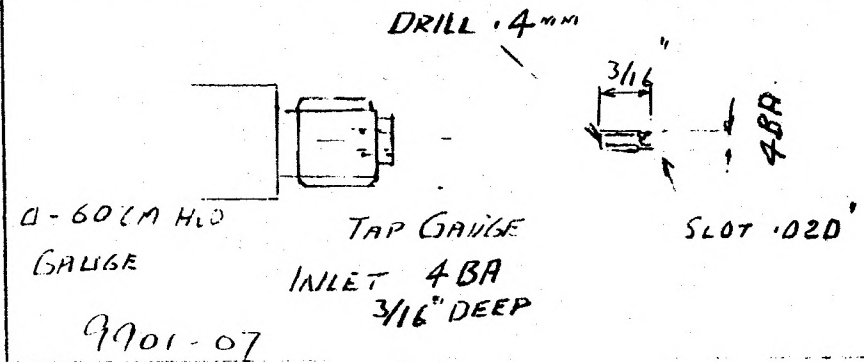

1-60 CM HLD
GAUGE

TAP GAUGE

INLET 4BA
 $\frac{3}{16}$ DEEP

9901-07

ISSUE 4 3-3-97
ISSUE 3 11-11-93
ISSUE 2 1-8-97



VIAMED





JOHN TROTMAN
G.S.M. GRAPHIC ARTS LTD

Fax 8841

24/5/94



↑
RESUSCITATION UNIT
(in white)

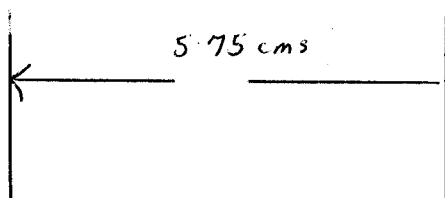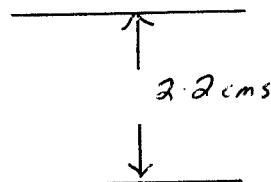

JOHN S LAMB



VIAMED





JOHN TROTMAN
G.S.M. GRAPHIC ARTS LTD

Fax 8841


24/5/94



↑
RESUSCITATION UNIT
(in white)

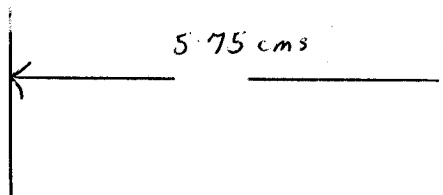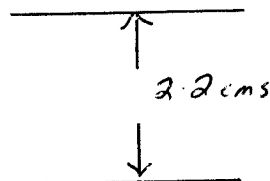


JOHN S LAMB

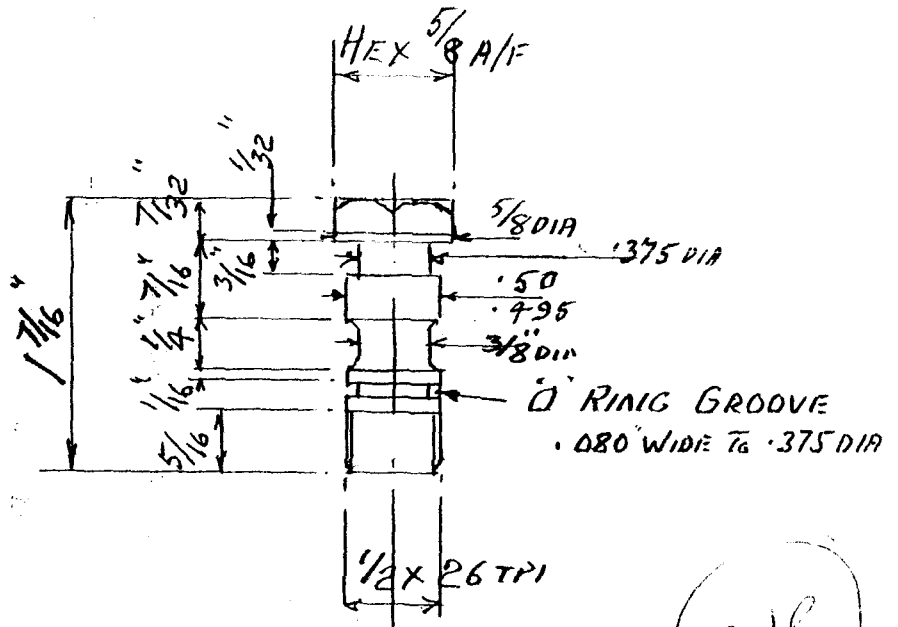

MAT^L BRASS FINISH FLASH NICKEL

N/R

DB SEP 1982

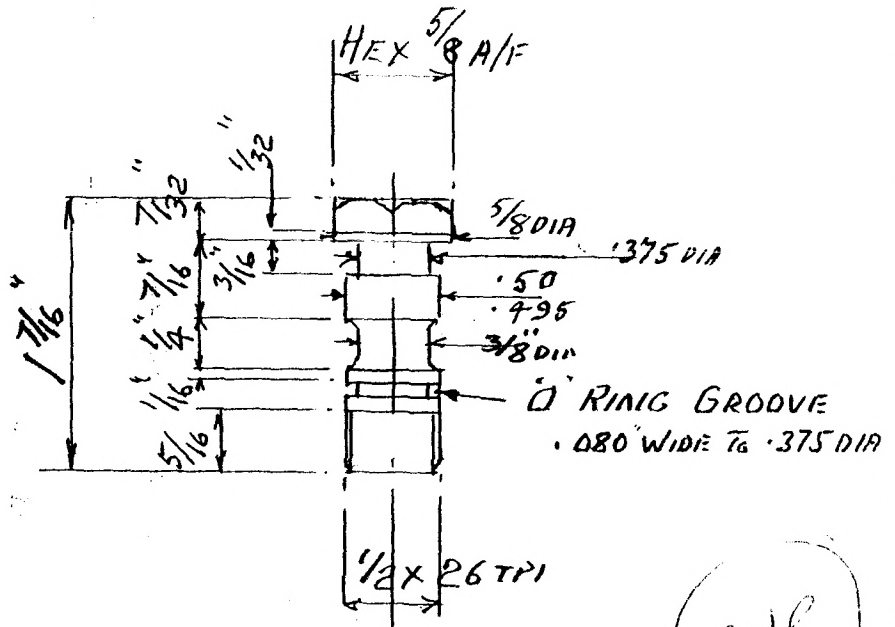

MAT^L BRASS FINISH FLASH NICKEL

N/R


DE SEP 1962



VIAMED



1. All Ports interconnected
2. Inlet & Outlet Ports 1,2,3,4, provide choice of fitting for Tubing connectors
3. Two Blanking plates required
4. Dimensions not critical

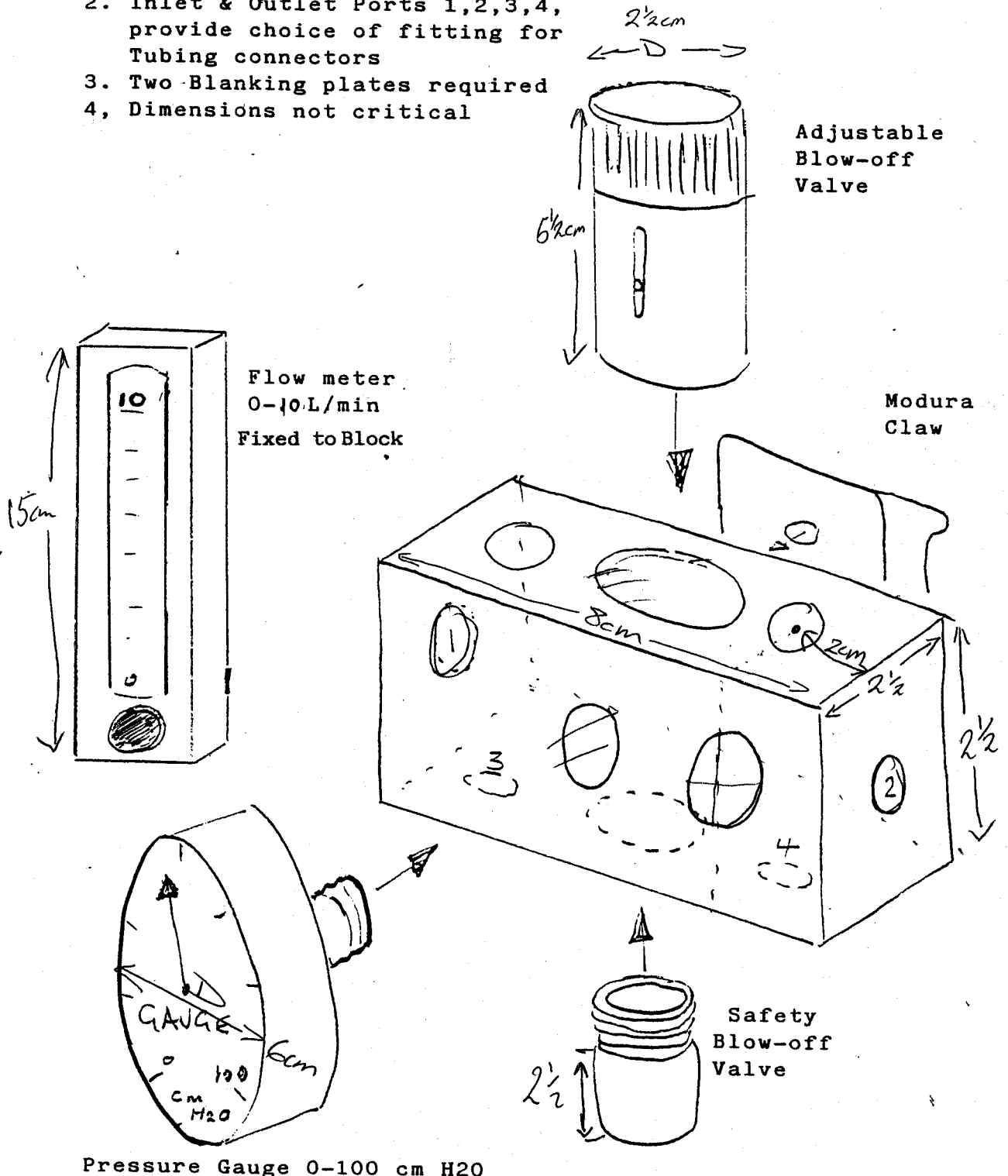

Pressure Gauge 0-100 cm H₂O

Newcastle upon Tyne Resuscitation Block

26/04/93

V 1.1

J.S.L.

Viamed Ltd. 15 Station Road, Cross Hills, Keighley, West Yorkshire BD20 7DT

Telephone: 0535 634542 / 636757 Fax: 0535 635582

Registration No. 1291765 in England


TT88




VIAMED




1. All Ports interconnected
2. Inlet & Outlet Ports 1,2,3,4, provide choice of fitting for Tubing connectors
3. Two Blanking plates required
4. Dimensions not critical

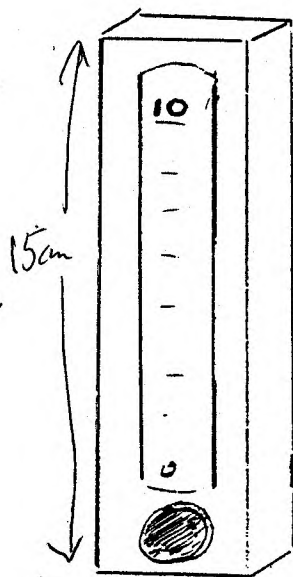


Flow meter
0-10 L/min
Fixed to Block

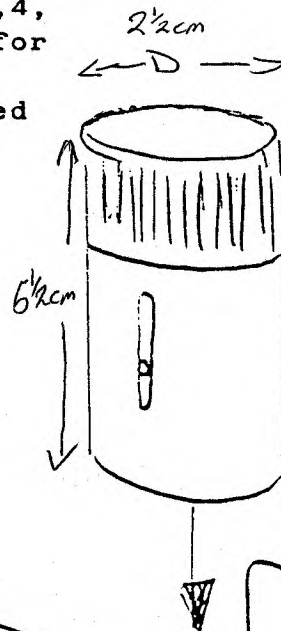


Adjustable
Blow-off
Valve

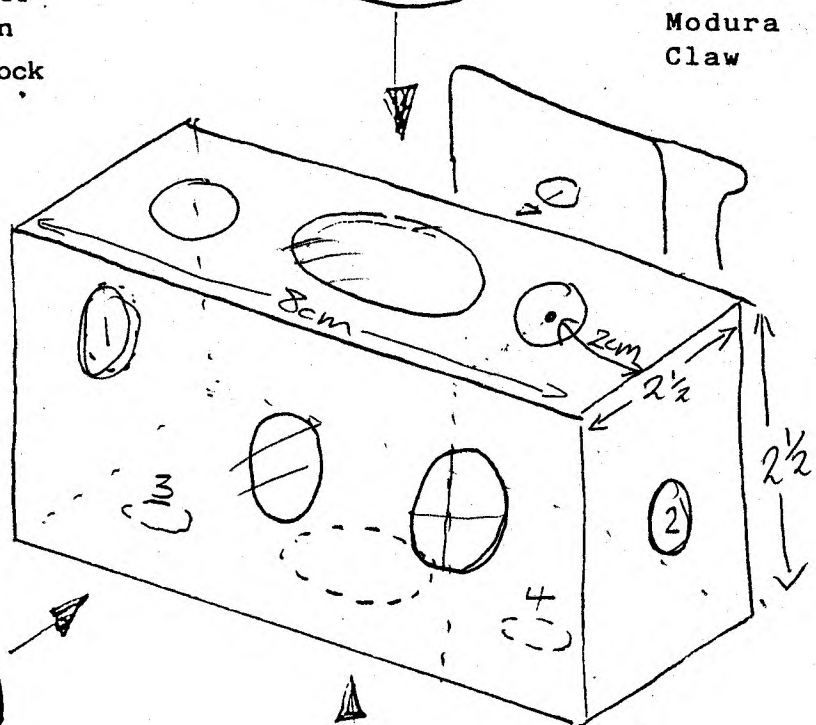

Modura
Claw

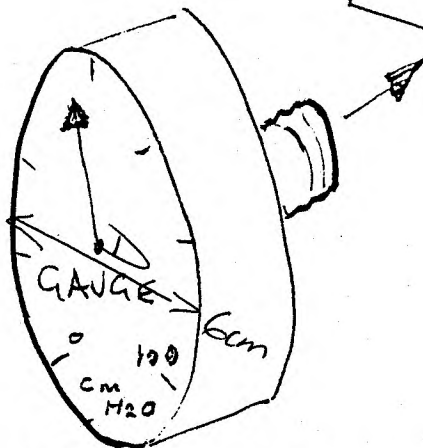

Pressure Gauge 0-100 cm H2O

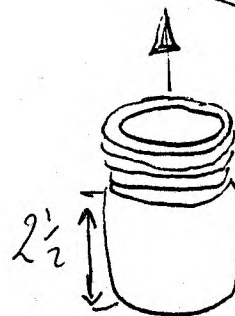

Safety
Blow-off
Valve

Newcastle upon Tyne Resuscitation Block

26/04/93

V 1.1



J.S.L.



Viamed Ltd. 15 Station Road, Cross Hills, Keighley, West Yorkshire BD20 7DT

Telephone: 0535 634542 / 636757 Fax: 0535 635582

Registration No. 1291765 in England

TT88

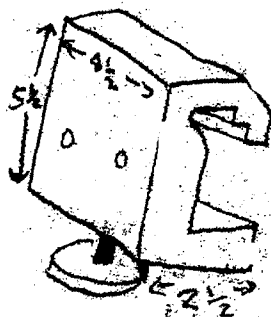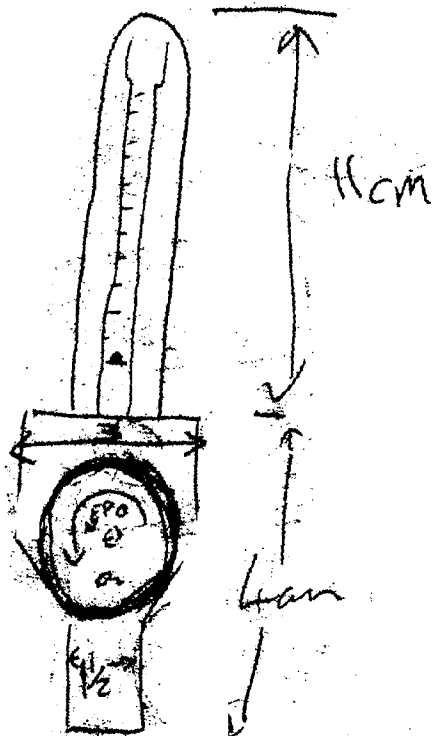

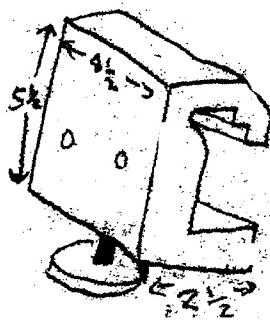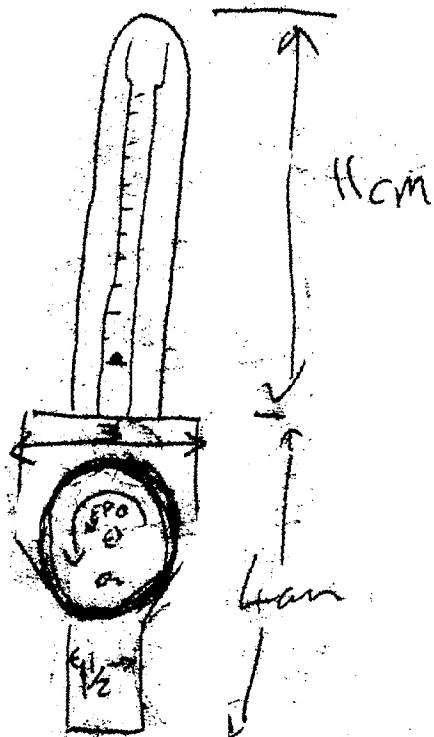




VIAMED



1. All Ports interconnected
2. Inlet & Outlet Ports 1,2,3,4, provide choice of fitting for Tubing connectors
3. Two Blanking plates required
4. Dimensions not critical

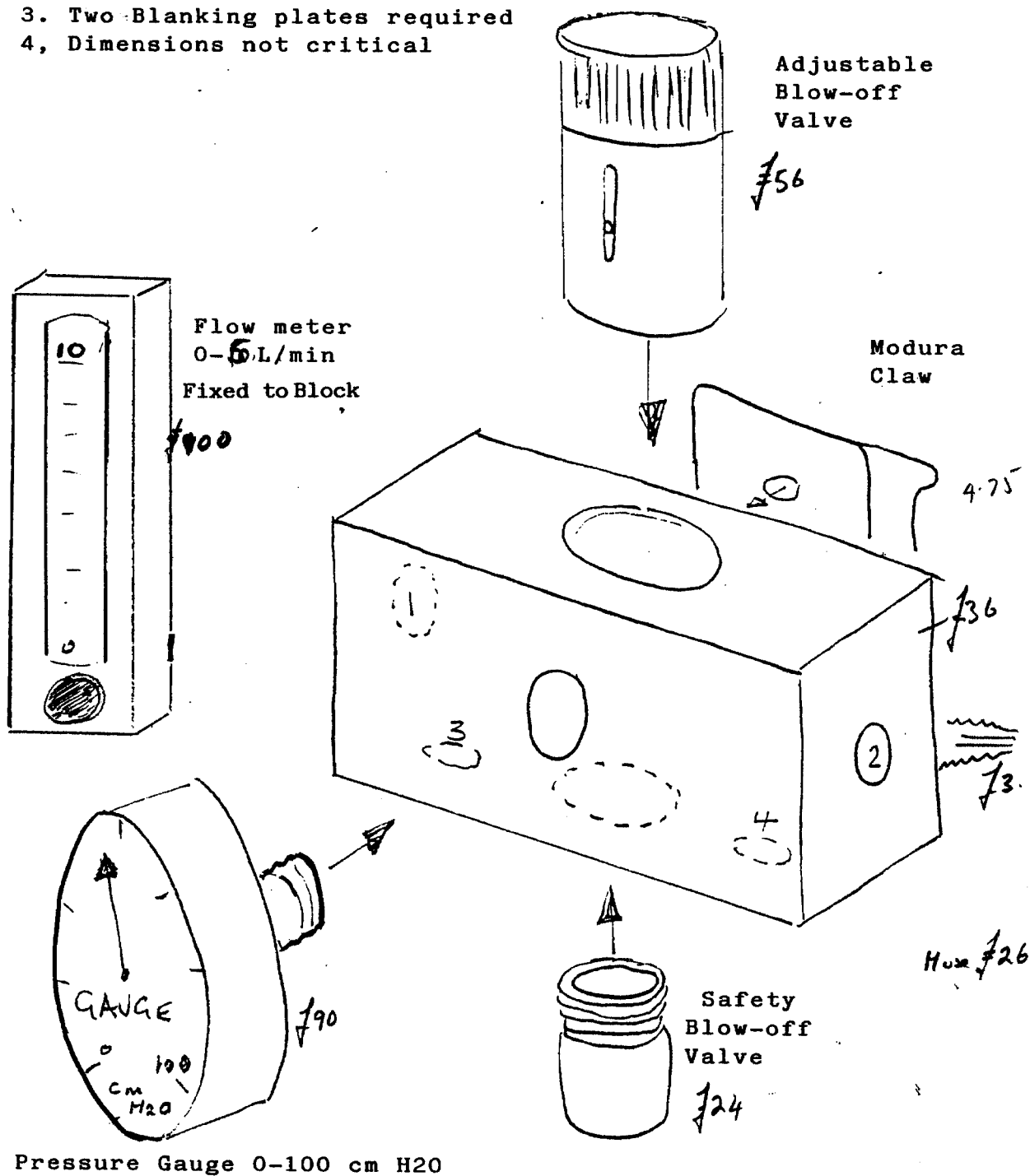

Pressure Gauge 0-100 cm H2O

Newcastle upon Tyne Resuscitation Block

26/04/93

V 1.1

J.S.L.

Viamed Ltd. 15 Station Road, Cross Hills, Keighley, West Yorkshire BD20 7DT

Telephone: 0535 634542 / 636757 Fax: 0535 635582

Registration No. 1291765 in England


TT 84



VIAMED



1. All Ports interconnected
2. Inlet & Outlet Ports 1,2,3,4, provide choice of fitting for Tubing connectors
3. Two Blanking plates required
4. Dimensions not critical

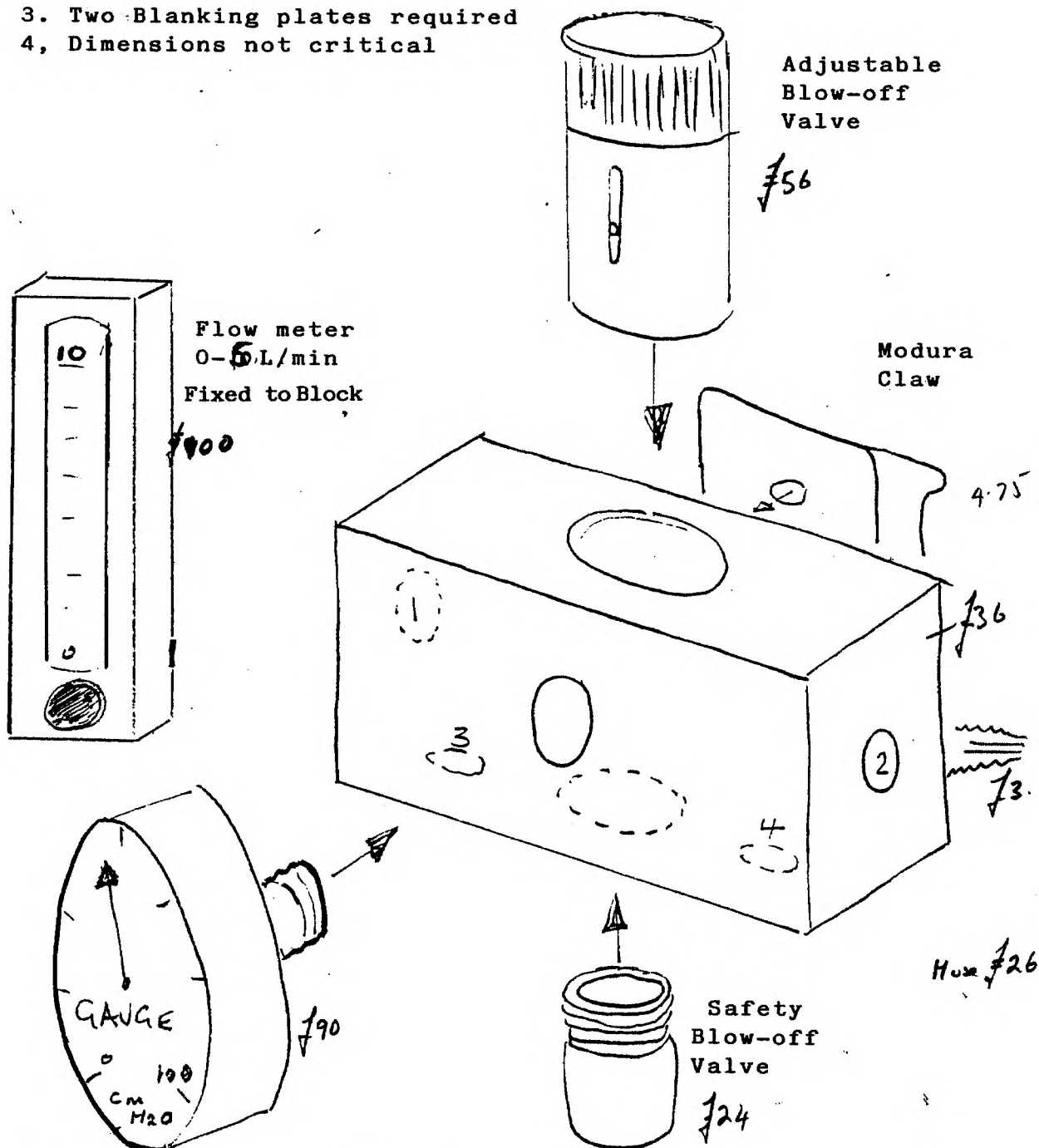

Pressure Gauge 0-100 cm H₂O

Newcastle upon Tyne Resuscitation Block

26/04/93

V 1.1

J.S.L.

Viamed Ltd. 15 Station Road, Cross Hills, Keighley, West Yorkshire BD20 7DT

Telephone: 0535 634542 / 636757 Fax: 0535 635582

Registration No. 1291765 in England

TT 84