EN**Instructions for Use - Reusable Temperature Probes**

Contains: 1 piece

Rectal/oesophageal application, types:**0212009, 0212010, 0212011, 0212012, 0212013, 0212014, 0212015, 0212016, 0212017
0212018, 0212019, 0212020****Skin application, types:****0212021, 0212022, 0212023, 0212024, 0212025, 0212026, 0212027, 0212028, 0212029
0212030, 0212031, 0212032****Intended use**

The Series 400 reusable temperature probes are intended for the continuous measurement and monitoring of patient temperature, when used with YSI, HP, Philips, Siemens temperature monitoring equipment, and other devices that are specially intended for use with Series 400 temperature probes (YSI resistance curve). A range of temperature probe types is available and they are approved for use by professional medical staff and for professional health care facility applications only. The materials that have direct patient contact during application comply with the requirements of biocompatibility according to ISO 10993-1.

Instructions for Use

1. Temperatures can be taken e.g. on the skin surface, in the oesophagus or rectally. Please use water-based lubricants only.
2. **Skin application:** Affix the probe to the skin using medical adhesive tape, or in special circumstances on the extremities such as fingers or toes. Ensure that the skin contact is not compromised by movement.
3. **Oesophageal application:** The probe should be placed in the lower third of the oesophagus.
4. **Rectal application:** The probe is in correct position when it has passed the sphincter.

Specifications

Reusable and autoclavable (hot steam sterilization)

Overall length: 280 cm (nominal)

Temperature range:

Storage/Transport: -30°C...+70°C

Operation: +0°C...+50°C

Humidity range:

Storage/Transport: 10... 95 % r. h. non-condensing

Operation: 10...100 % r. h.

Ambient pressure:

Storage/Transport: 800...1060 hPa

Operation: 800...1060 hPa

Accuracy: 25°C...45°C ± 0.1°C

Transient response time (2 K transient):

Adult types: < 50 s

Paediatric types: < 30 s

Neonatal types: < 15 s

Cleaning and disinfecting the probe (reprocessing)

1. Before re-use of the probe it must be cleaned, disinfected and/or sterilized.
2. Before cleaning or disinfecting the sensor, it has to be detached from the monitor.
3. Temperature probes can be cleaned with a cleaning agent, by wiping the probe with a soft cloth. Avoid applying any high mechanical force to the probe during the cleaning process.
4. To disinfect the probe, wipe the probe and the contact surfaces with disinfectant or immerse the probe in disinfectant solution. We recommend the use of Cidex® OPA or Incidin® Plus. Always refer to the IFU supplied by the disinfectant manufacturer. Avoid the use of disinfectants containing ketones.

Steam sterilisation

Steam autoclave at 132°C/4 min or 134°C/5 min using standard cycles. Follow the generally applicable instructions and regulations! Clean any residues from the temperature probe before sterilization and disinfect the probe according to applicable reprocessing rules.

Caution

Improper use of probes can lead to damage of the internal wiring and to loss of electrical insulation, as well as incorrect temperature readings.

Viamed Ltd., 15 Station Road, Cross Hills, Keighley,
West Yorkshire BD20 7DT, United Kingdom
Tel. +44 (0)1535 634542 / email: info@viamed.co.uk
<http://www.viamed.co.uk>

Warning

1. This temperature probe is intended for use only in conjunction with compatible monitors. Usage in conjunction with other devices can lead to malfunction.
2. Strong sources of electromagnetic fields, e.g. electro-surgical devices, can adversely affect the functioning of the equipment. If at all possible, the probe should be completely removed from the patient before a surgical device or other HF source is activated! Check the protective earthing if you intend to use HF devices and temperature measuring equipment in parallel. Minimize the potential danger by selecting a temperature measurement point far away from the HF current path.
3. We do not recommend the use of the product within range of magnetic resonance imaging (MRI) or computed tomography (CT) devices.
4. Stop using the probe immediately if it becomes damaged.
5. For further information and warnings please read the usage instructions accompanying the temperature measuring device.

Maintenance/Service

The temperature probes do not contain serviceable or replaceable components. Modification or replacement of components is not allowed. It is recommended to run an inspection of accuracy at intervals of two years.

Returning the product

Contact the customer service department at Viamed Ltd. to obtain an RMA number (Return Material Authorisation). Every product sent to us for checking must have been cleaned and disinfected beforehand.

Disposal

Before disposing of the product, it may be necessary to clean and disinfect it. Local regulations may apply for disposal of used medical devices.

Do not dispose in domestic waste

Product manufacturer

Product is supplied non-sterile

Consult instructions for use

IP 44

Degree of protection for this product is IP44

Ambient temperature range


Follow instructions for use

Ambient humidity range for storage and transportation

Product is free of latex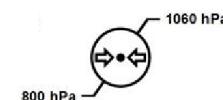

Ambient atmospheric pressure range for storage and transportation

bluepoint medical GmbH & Co. KG
An der Trave 15, 23923 Selmsdorf,
Germany

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