




Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk

Instructions for Use

- 1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
- 2. Remove the NeoMask from the packaging and disconnect the rear fastener.
- 3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
- 4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.
Do not over tighten. Ensure that the mask is not hooked over the ears and that the ears are not folded over.
- 5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.


 Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.

Large **1114015:** Occipital-Frontal Circumference **32 - 38cm.** Pack of 20

Medium **1114016:** Occipital-Frontal Circumference **26 - 32cm.** Pack of 20

Small **1114017:** Occipital-Frontal Circumference **20 - 26cm.** Pack of 20



 **Ontecklong Medical Supply Corporation Limited.**
Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe District
Nanjing, 210048 Jiangsu, P.R. China

Viamed Limited.
15 Station Road, Cross Hills,
Keighley, West Yorkshire
BD20 7DT, United Kingdom

LiKing GmbH
Fraunhofer Street 7
04178 Leipzig
Saxony, Germany