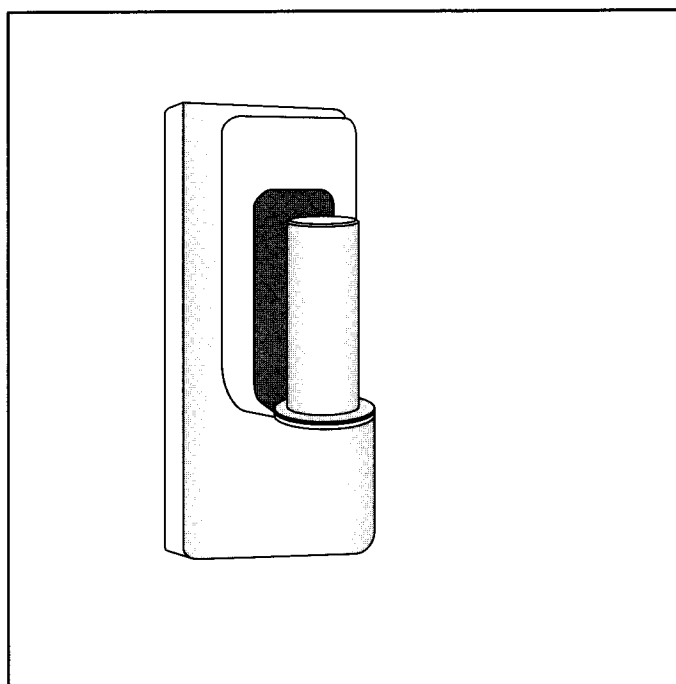
Installation instructions

Montageanweisung

ACROBAT 2000®

**Wall bearing
Wandlager**

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1	Static inspection	4
2	Selection of the fastening agent	5
3	Installation of the wall bearing	6
4	Installation of the extension arm stop	7
5	Technical data	8
	Drilling template	9

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Packing list, wall bearing**Packing part no.**

- 1 Wall bearing

Packliste Wandlager**Packstück Nr. 2131**

- 1 Wandlager

Instructions for safe installation

Dear user,
the wall bearing must only be installed by a technical expert (electrical engineer, technician) or a comparably qualified person.
The 230 V connection may only be made by an electrical engineer.

Dear installer,
please read these installation instructions very carefully and take note of the safety instructions and requirements given in these installation instructions.
If any peculiar problems occur that are not handled to an adequately detailed extent in this manual, please contact your supplier for your own safety.

Explanation of pictorial symbols:

Safety symbols draw your attention to operating errors that are critical with regard to safety.

WARNING!

Disregarding this instruction may present the danger of a serious or even fatal injury.

CAUTION!

Disregarding this instruction may result in medium to light injury and damage to property.

IMPORTANT!

Provides usage tips and useful information.

Transportation and storage:

The following storage conditions apply for up to 15 weeks:

- temperature: -25 to +70 °C
- relative humidity: 10 - 75 %
- air pressure: 500 - 1060 hPa

Storage only in closed or covered spaces; thereafter, the values of the operating conditions in the operating instructions apply.

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1 Static inspection

NOTE:

The static (structural) inspection must be carried out before the installation of the wall or ceiling anchorage!

- The strength of the construction must be designed, checked and certified by a structural engineer.
- The respective regional construction regulations that apply must be followed.
- If a wrong hole is drilled by mistake, e.g. drilling of a reinforcement rod, the structural engineer who is responsible must be contacted, since adequate static load distribution in the ceiling may have been endangered.

Declaration of acceptance:

It is hereby certified that the supporting wall/ceiling and the anchoring of the Acrobat Swing is safe and adequately strong.

Project: _____

Anchoring (please check the one that is applicable):

- with counter-plate ☐
- other ☐

Location: _____

Signature/Stamp (Structural Engineer / Construction Authority)

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2 Selection of the fastening agent

- The selection of the fastening agent according to table 1, „Load data“ and the safe execution of the fastening is the responsibility of the person in charge of the installation
- In case of light construction walls, we recommend fastening with a counter-plate (not shipped).

WARNING - Load data:

Please refer to the following table for the load data (rounded up) of the complete ceiling model.

The specified regional safety factors must be taken into account!

Load data: Wall model		
Maximum extension arm length [mm]	Maximum device load [N]	Bending moment M_B [Nm]
300	210	325
600	150	365
750	130	285
950	100	260
Maximum extension arm length [mm]	Vertical weight force F_G [N]	Dowel extraction force F_D [N]
300	300	1750
600	260	1950
750	245	1500
950	225	1400

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3 Installing the wall bearing

NOTE:

For the mains connection of the extension arm, a properly grounded socket in the vicinity of the wall bearing is required. The power supply cable of the extension arm is 2 meters long.

1. Remove the two **plastic covers (1)**.
2. Make the vertical markings using the drilling template on page 13.
3. Drill two holes according to the specifications of the fastening agent's manufacturer.
4. Install the fastening agent flush with the wall.

NOTE:

Do not continue the installation before the binding agent has hardened completely.

5. Insert the fastening agent into the two **holes (2)** and tighten the wall bearing in a vertical position.

NOTE:

You can use a stop to limit the lateral swiveling range of the extension arm. The installation of the stop is described in chapter 4.

6. Install the two **plastic covers (1)** and check them for firm seating.

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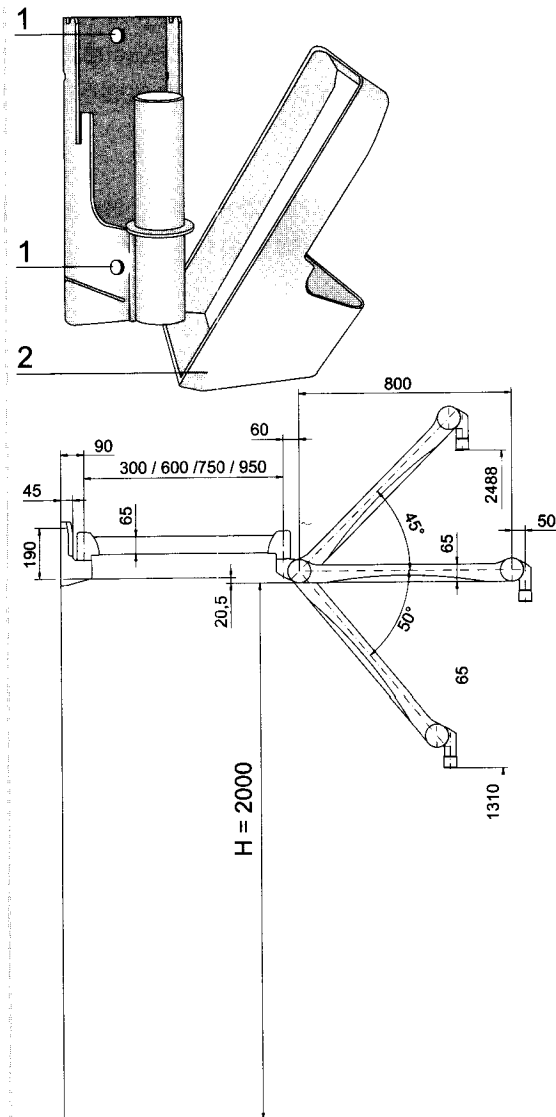
4 Installing the extension arm stop

NOTE:

The stop is supplied with a plastic cover with cut-out.

The upper plastic cover of the wall bearing is no longer required and may be discarded.

1. Remove and discard the upper plastic cover of the wall bearing.
2. Insert the **extension arm stop (2)** into the two **grooves (3)** of the wall bearing as shown in Figure 2.
3. Check the **extension arm stop (2)** for firm seating.
4. Install and engage the supplied **plastic cover with cut-out (1)**.
5. Check the **plastic cover with cut-out (1)** for firm seating.

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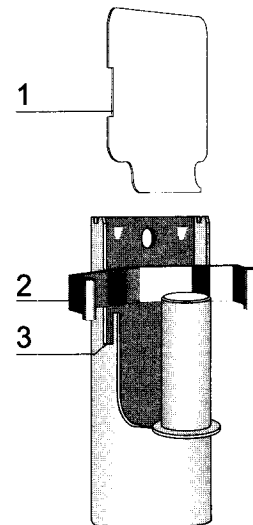
5 Technical Data

Electrical data	Ceiling model		Wall model	
Rated voltage	120 V	230 V	120 V	230 V
Nominal frequency	60 Hz	50 Hz	60 Hz	50 Hz
Rated current	15 A	16 A	15 A	16 A
Weight in kg	Ceiling model		Wall model	
Ceiling anchorage single (DB 2001)	10.6 kg		---	
Ceiling anchorage double (DB 2002)	13.9 kg		---	
Spacer tube	6.78 kg / m		---	
Wall bearing	---		1.8 kg	
Exten. arm 300 mm	---		1.3 kg	
Exten. arm 600 mm	2.6 kg		2.6 kg	
Exten. arm 750 mm	3.2 kg		3.2 kg	
Exten. arm 950 mm	4.2 kg		4.2 kg	
Spring arm	6.2 kg		6.2 kg	
Bow	5.0 kg		5.0 kg	
Classification according to EN 60601-1				
Device of protection class I				
The device is not suitable for usage in combustible mixtures of anesthetic agents with air or in combustible mixtures of anesthetic agents with oxygen or laughing gas.				
Suitable for continuous operation				
EU conformance				
The product conforms to the specifications in the guideline 93/42/EWG (medical products guideline) and the guideline 89/336/EWG (EMV guideline).				
Classification according to guideline 93/42/EWG, appendix IX				
Class I				
EU conformance				
The Acrobat 2000 equipment conforms to the specifications in the guideline 93/42/EWG (medical products guideline) and the guideline 89/336/EWG (EMV guideline).				
The Acrobat 2000 equipment is only approved for connecting medical examination lamps, operation lamps and flat-screens with CE markings. If these end devices are connected to the ACROBAT 2000 equipment, the person operating the equipment must submit a declaration as per article 12 of the guideline 93/42/EWG (medical products guideline). A new conformance evaluation is to be carried out for other end devices.				
Extension arms other than ONDAL types 77, 77-1, 79 and 2000 must not be installed.				
The maximum load must not be exceeded for ACROBAT 2000 equipment:				
Ext. arm length	Ceiling model		Wall model	
300 mm	---		21.0 kg	
600 mm	21.0 kg		15.0 kg	
750 mm	18.0 kg		13.0 kg	
950 mm	15.0 kg		10.0 kg	
Certifications				
VDE, UL, CE				

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