


VIAMED Ltd.

15 Station Road, Cross Hills, Keighley, West Yorkshire, BD20 7DT, UK.

 Website : www.viamed.co.uk. Email : info@viamed.co.uk.

Tel : +44 (0)1535 634542. Fax : +44 (0)1535 635542.

TOM THUMB CALIBRATION / TEST & Q.A. SHEET.

Description.	Tom Thumb.	Time & Date of Calibration / Test.	
Model.	TT 495.	Time & Date of QA.	
Serial No.		Serial no. : O ₂ flowmeter.	
		Serial no. : Air flowmeter.	

Do not start QA check within 1 hour of the calibration / test. Labelling is to be attached after calibration / test. Record the manometer reading in millibars below, ensuring the TT495 meets the limits specified.

Test Equipment	Test	Specification	Reading		P/F
			Cal	Q.A.	
Snoop liquid.	Check all ports / connections for leaks.	No bubbling.			
CE 078.	Adjustable Valve : @ 15 Lpm	Minimum : ≤ 7 cmH ₂ O.			
CE 078.	Adjustable Valve : @ 15 Lpm.	Maximum : ≥ 43 & ≤ 47 cmH ₂ O.			
CE 078.	Precision Valve : @ 15 Lpm.	Maximum : +3.0 cmH ₂ O over the adjustable valve setting.			
CE 078.	Pressure Gauge Test.	Pressure : @ 50 cmH ₂ O.			
CE079	O ₂ flowmeter accuracy	10 Lpm on O ₂ set on TT ≥ 9 & ≤ 11 Lpm			
CE079	Air flow meter accuracy	10 Lpm on air set on TT ≥ 9 & ≤ 11 Lpm			
Visual check.	Gauge cannot be removed without tools.				
Visual check.	Gauge appears straight.				
Visual check.	All adjustable settings are set to a minimum.				
Visual check.	Labels are attached.	CE label.			
		Viamed Flowmeter label.			
		Serial No. label.			
		Tom Thumb label			

Calibration :
Q.A check :
Signed :

Signed :