	Short User Instructions	Rev.3
Internal Document	MSP430 Update Assistant	Page 1 of 10

Short Instructions for Use:

MSP430 Update Assistant

Name: H. Fröhlich	Date: 27.04.2011
File: MSP430 Update Assistant_rev3	

bluepoint MEDICAL	Short User Instructions	Rev.3
Internal Document	MSP430 Update Assistant	Page 2 of 10

A. Purpose of the document

This document describes the use of the MSP430 Update Assistant.


With this software a new firmware can be uploaded to the microcontroller of the Handheld Multigas Monitors via USB.


In addition the following can be performed via PC interface:

- Device Format
- Set Date (and time)

B. Installing the software

1. Open the latest version of the Firmware updater “**Flash Update Assistant V1.4.EXE**”
2. Follow the instructions for installing the software
3. After installation an icon is created on the desktop

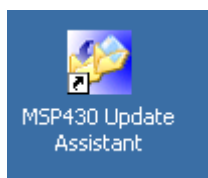

	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 3 of 10

C. Updating the Handheld Multigas Monitor

1. Turn on the Handheld Multigas Monitor

Warning:

Ensure that the battery of the device is fully charged or use the external power supply delivered with the device.


If the power supplied to the device is interrupted during the firmware update procedure, the device can no longer be updated with the MSP430 Update Assistant anymore.

2. Connect the device via USB to the PC.

The message “USB connected” is displayed.



3. Open the “MSP430 Update Assistant” Software

4. Select “**Detect Connection**” to find the connection to the Handheld Multigas Monitor

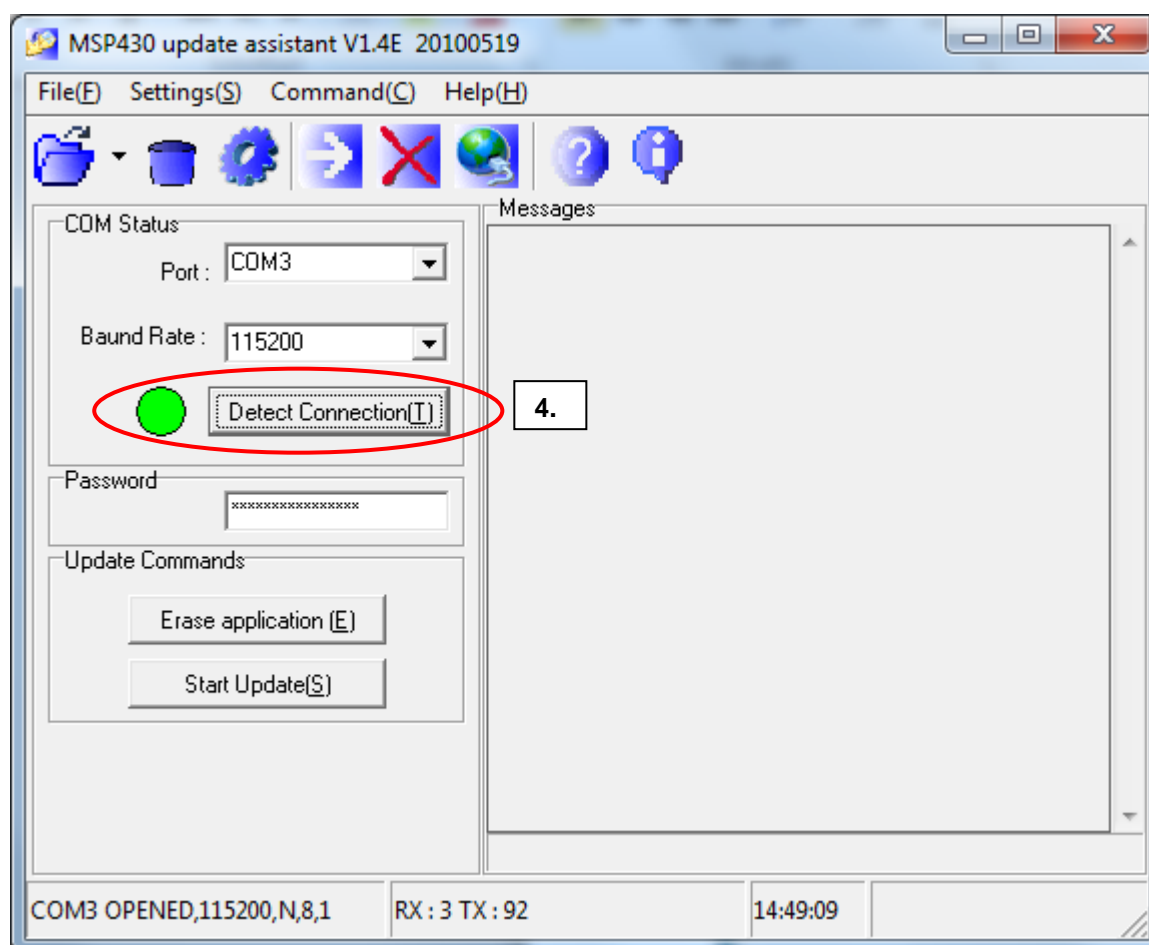



bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 4 of 10

5. Select the new firmware code file (.txt)

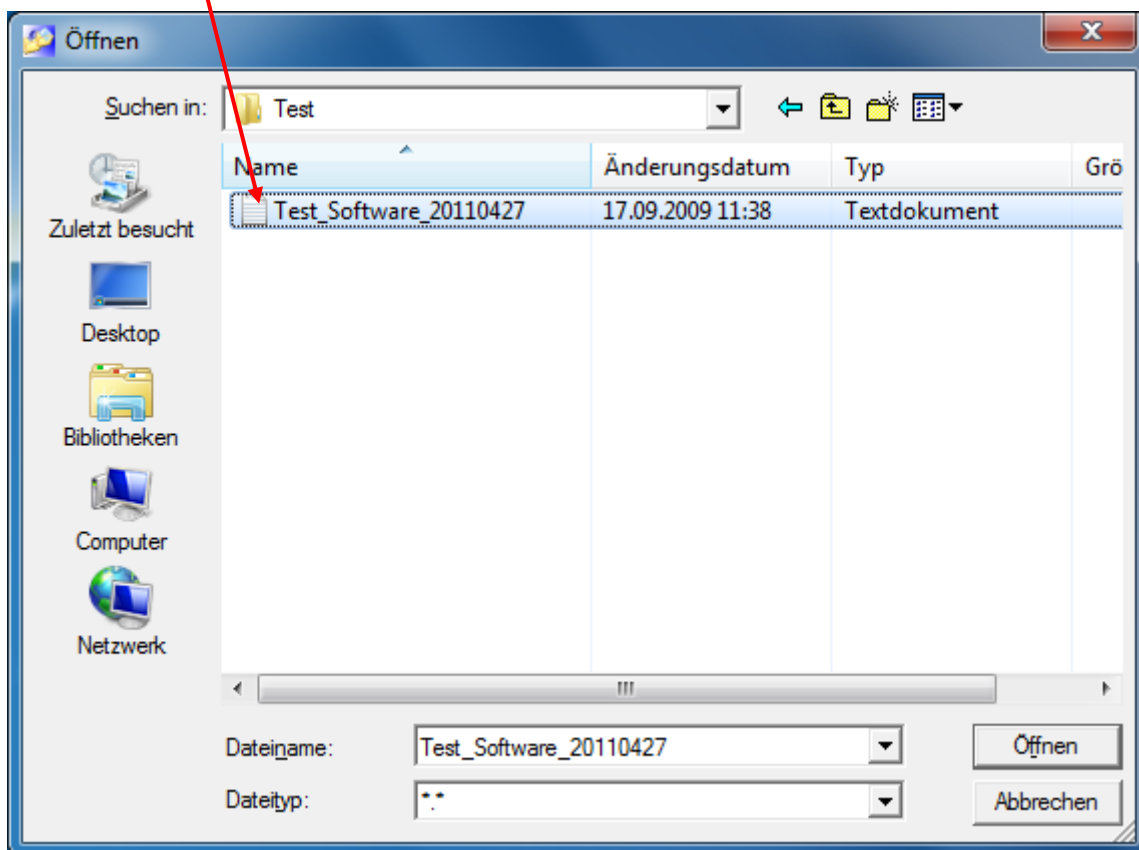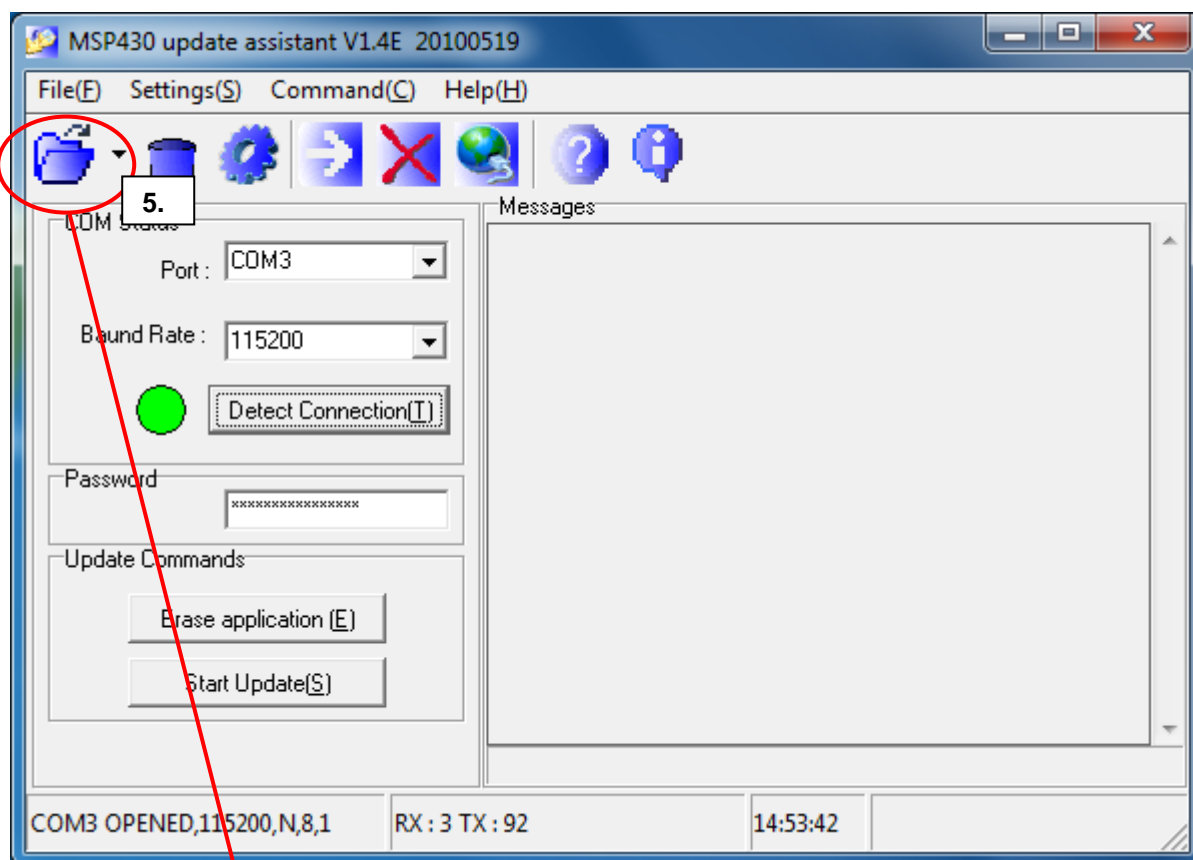



bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 5 of 10

6. Erase the current firmware by selecting “Erase application”

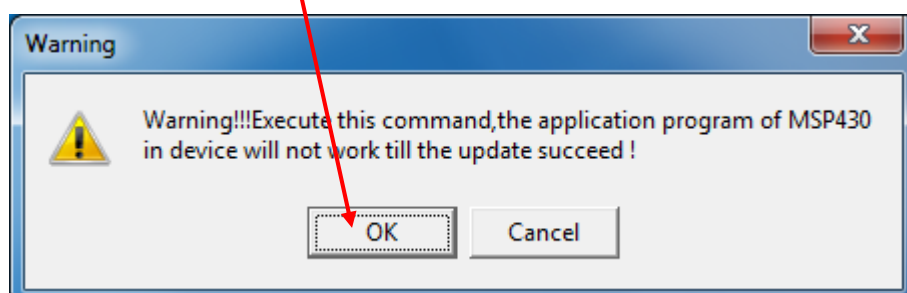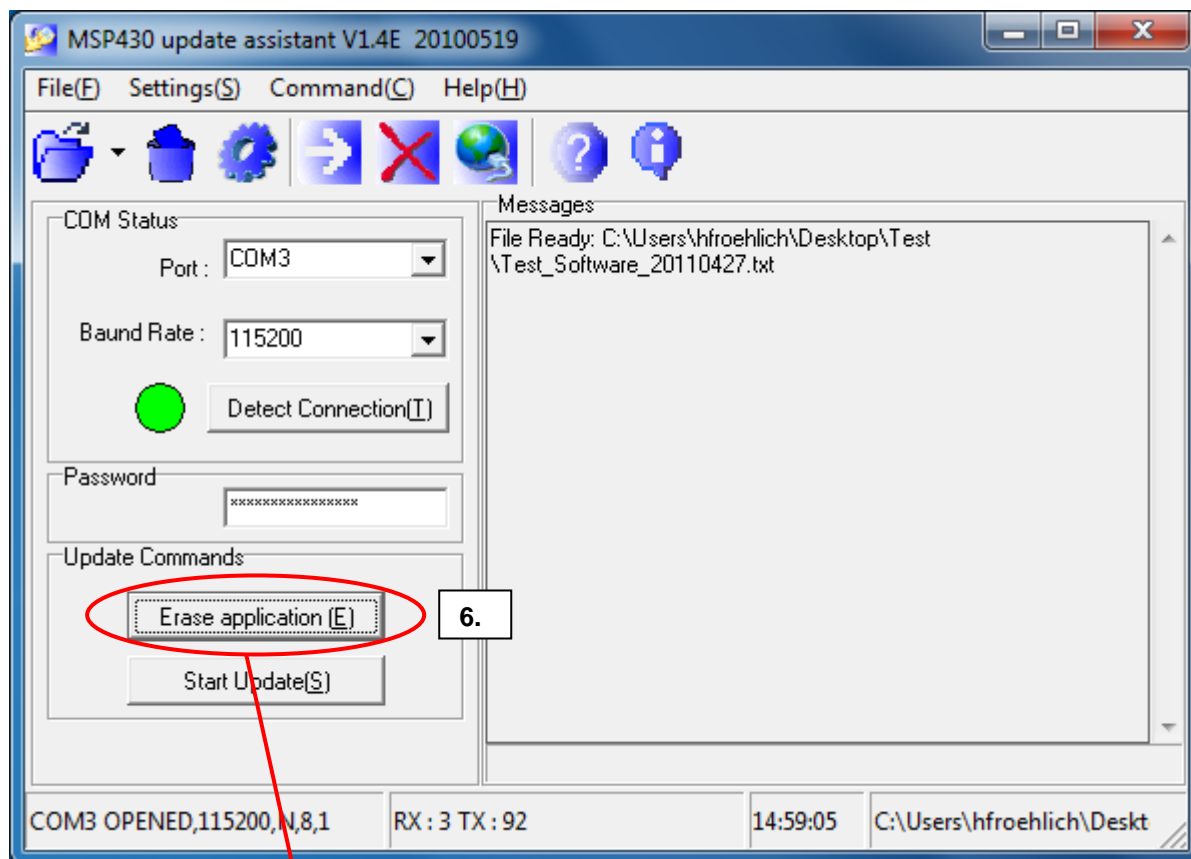


bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 6 of 10

7. Update the firmware by selecting “**Start Update**”

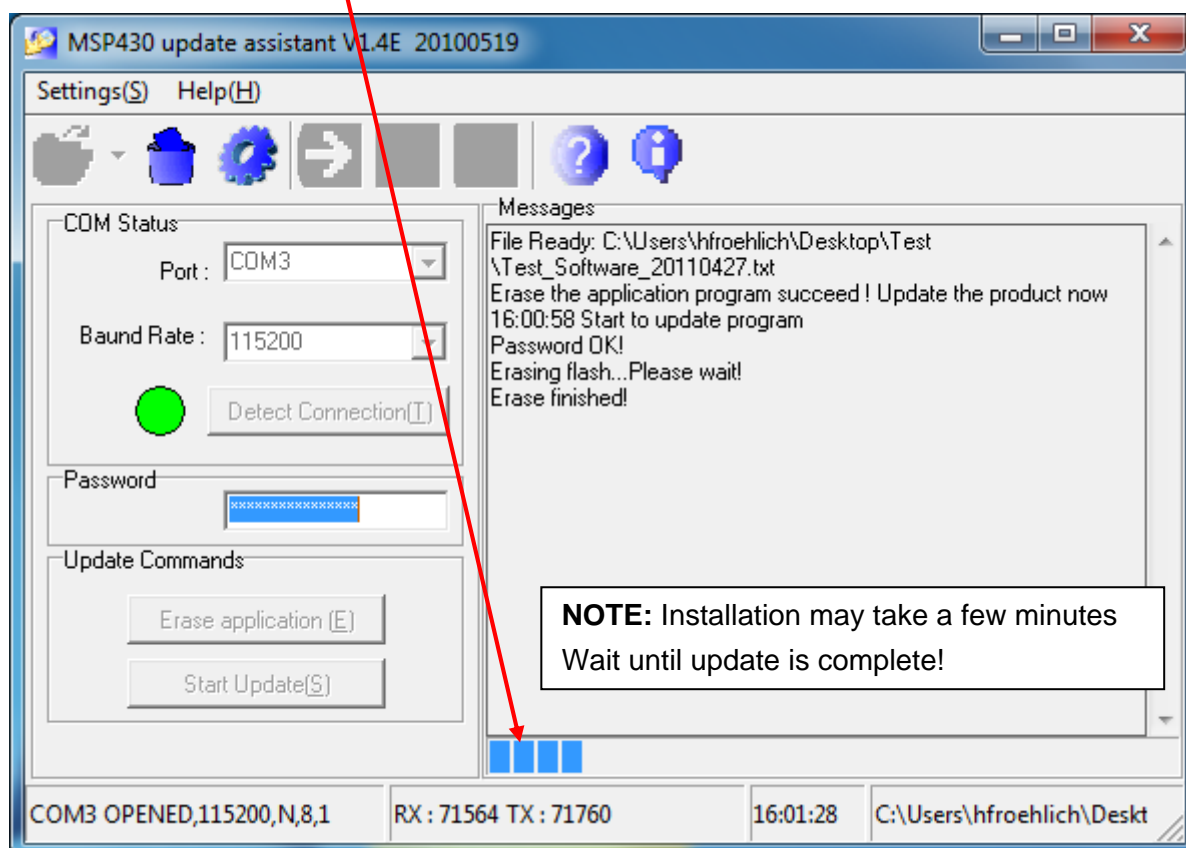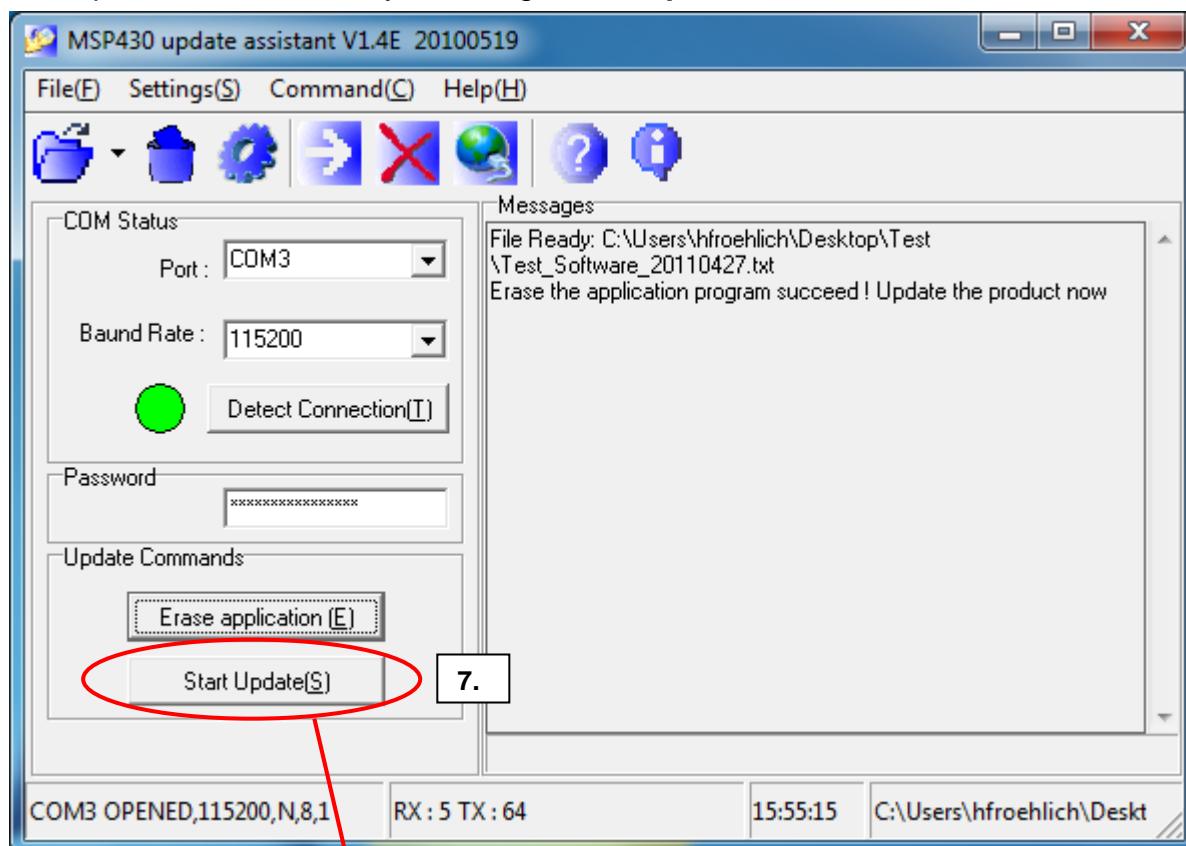

bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 7 of 10

8. Message is displayed when device Firmware has successfully been updated

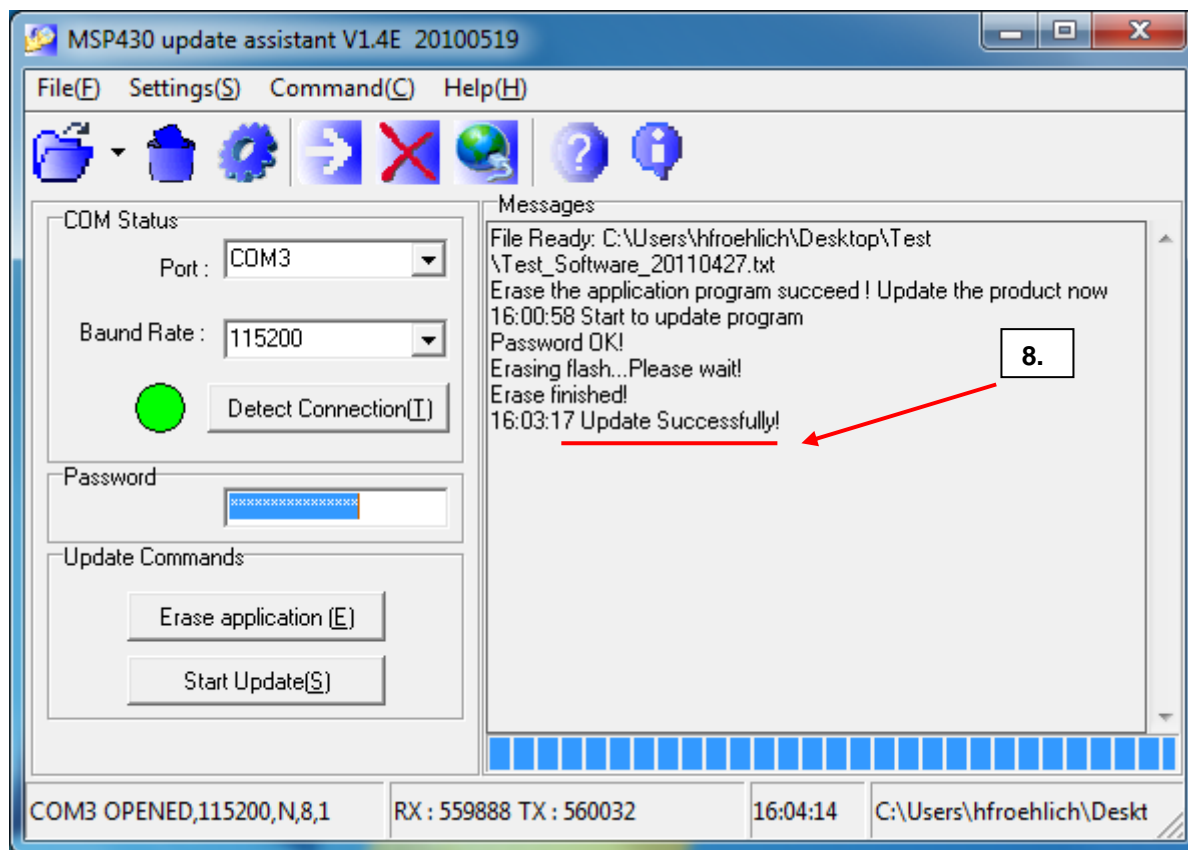

Warning:

To ensure correct function of the Firmware always format the device after each Firmware update.


See chapter “**D. Set Time and Device Format**” for detailed information on how to perform device format.

bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 8 of 10

D. Set Time and Device Format


After updating the device Firmware always set the device date and time and perform device format by following the instructions below:

9. Turn on the updated Handheld Multigas Monitor
10. Ensure that the device is connected to the PC via USB
The message "USB connected" is displayed.
11. Select "**Accessorial Function**" under "Settings" of the software version V1.4 or higher

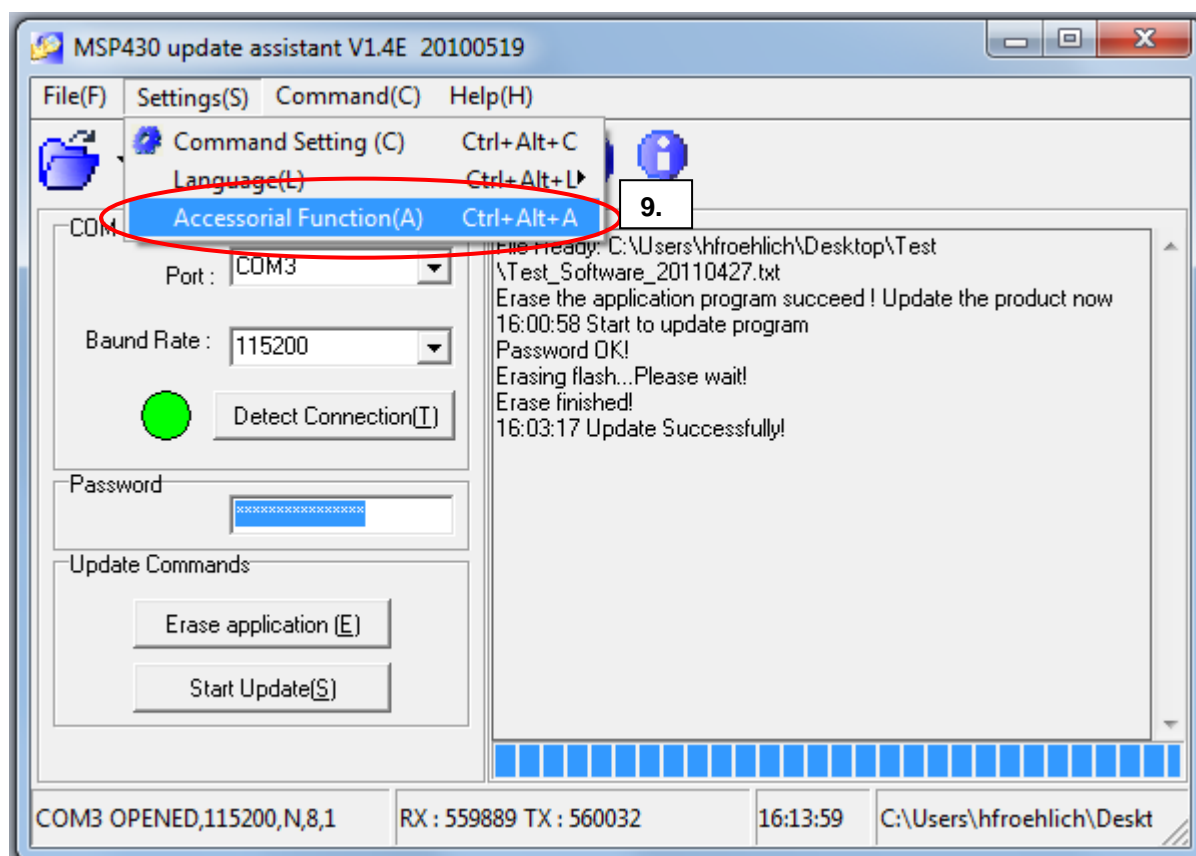

bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 9 of 10



12. Select “**Set Date**”

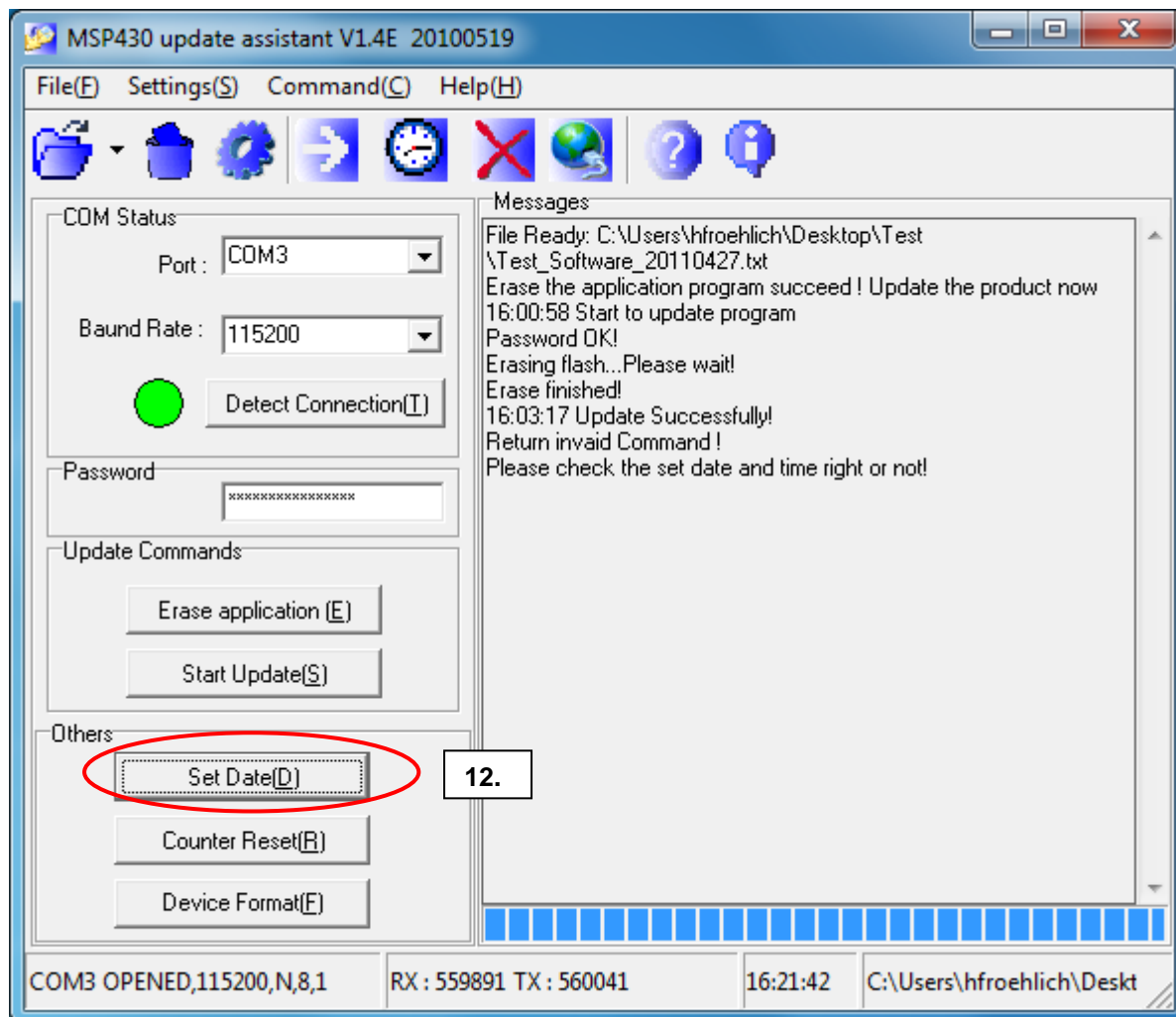

Note: Check at the device if the correct time is displayed. If not, retry

bluepoint MEDICAL	Short User Instructions	Rev. 3
Internal Document	MSP430 Update Assistant	Page 10 of 10

13. Now select “**Device Format**”

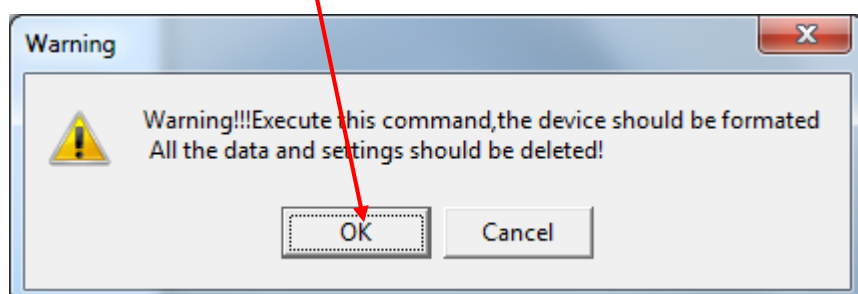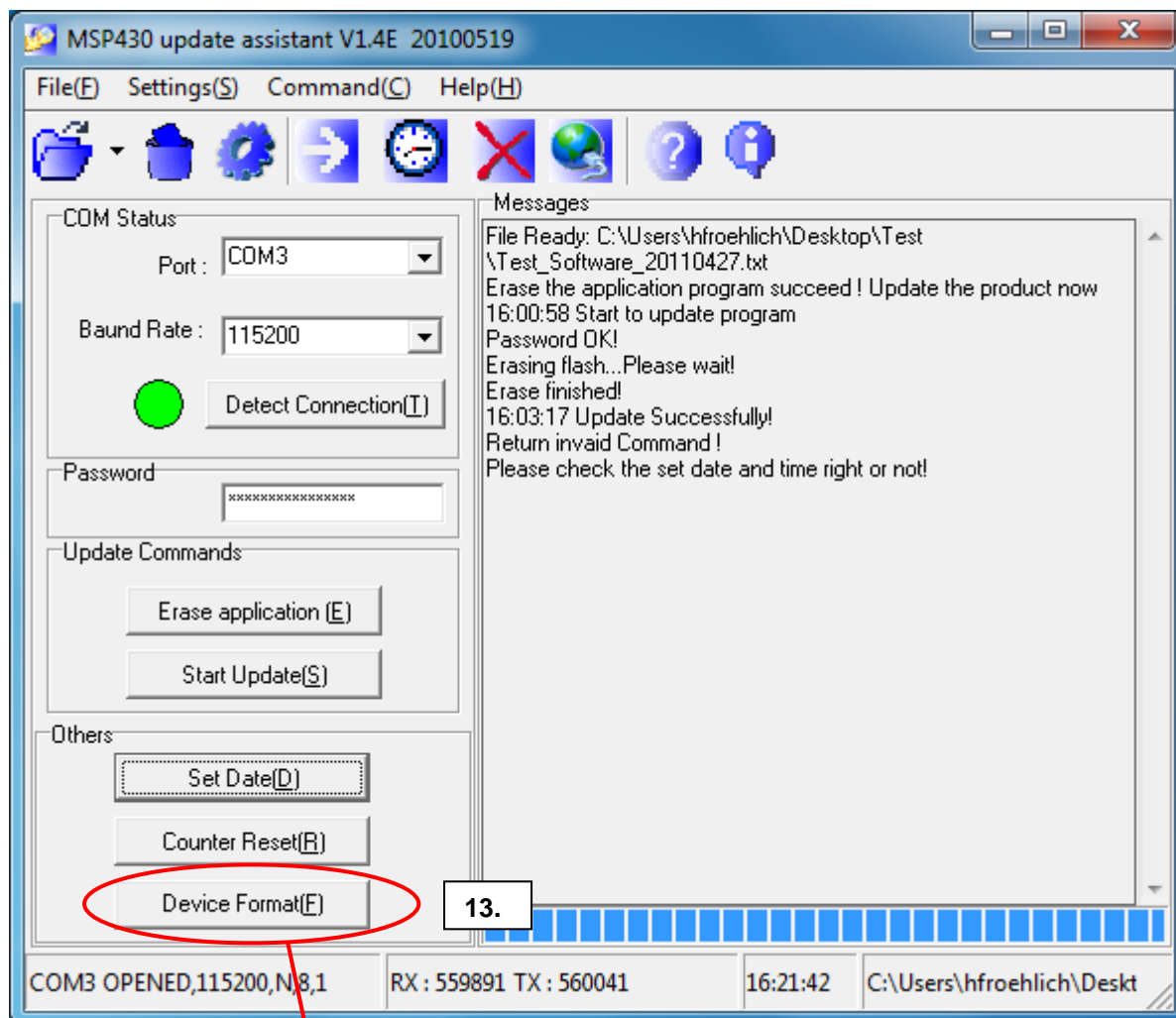

Note: If the device switches off after selecting “OK”, Format Device was successful.