

COMPANY OPERATING PROCEDURES**Maxtec Max-13(PN 0110413), Max 13F (PN 0110443) & Max 13-250 (PN 0110444)
O₂ Sensor Test Procedure**

VM3/COP/37.19

Date: 3-Feb-03

Revision date: 25-May-11

Issue: 3

Test & Equipment Required

Visual check only - no test equipment required.

Output specification as in document: (VM3COP37.00, column (i)).

Method.

1. Open the packaging and remove the O₂ sensor.
2. Check the O₂ sensor for damage and signs of leaking electrolyte.
3. Remove the O₂ sensor specification leaflet and discard.
4. Label with Viamed stickers if appropriate.
5. Reseal in serial numbered packet, integrity seal the edge of the packet with initialed and dated Viamed sticker and re-box.
6. Ensure the O₂ sensor is booked out in the stock books.