

COMPANY OPERATING PROCEDURES

Maxtec Max-250 (PN 0110428), Max-250E (PN 0110429) Max-22ESF (PN 0110430)
O₂ Sensor Test Procedure

VM3/COP/37.27

Date: 3-Feb-03

Revision date: 25-May-11

Issue: 3

Test & Equipment Required

Visual check only - no test equipment required.

Output specification as in document: (VM3COP37.00, column (i)).

Method.

1. Open the packaging and remove the O₂ sensor.
2. Check the O₂ sensor for damage and signs of leaking electrolyte.
3. Remove the O₂ sensor specification leaflet and discard.
4. Label with Viamed stickers if appropriate.
5. Reseal in serial numbered packet, integrity seal the edge of the packet with initialed and dated Viamed sticker and re-box.
6. Ensure the O₂ sensor is booked out in the stock books.