

COMPANY OPERATING PROCEDURES**Teledyne R10DS (PN 0110211) & R10DV (PN 0110212)
O₂ Sensor Test Procedure**

VM3/COP/37.32

Date: 3-Feb-03

Revision date: 25-May-11

Issue: 3

Test & Equipment Required

Visual check : Digital Voltmeter 200mV setting.

Output specification as in document: (VM3COP37.00, column (i)).

Method.

1. Open the packaging and remove the O₂ sensor.
2. Check the O₂ sensor for damage and signs of leaking electrolyte.
3. Check output 25mV +/- 2mV
- 4.
5. Label with Vandagraph stickers if appropriate.
6. Reseal in serial numbered packet, integrity seal the edge of the packet with initialed and dated Viamed sticker and re-box.