

COMPANY OPERATING PROCEDURES

Teledyne UFO-130-1 (0110080)

O₂ Sensor Test Procedure

VM3/COP/37.53

Date: 3-Feb-03

Revision date: 25-May-11

Issue: 3

Test & Equipment Required

Visual check only - no test equipment required.

Output specification as in document: (VM3COP37.00, column (i)).

Method.

1. Open the packaging and remove the O₂ sensor.
2. Check the O₂ sensor for damage and signs of leaking electrolyte.
3. Label with Viamed stickers if appropriate.
4. Reseal in serial numbered packet, integrity seal the edge of the packet with initialed and dated Viamed sticker and re-box.