



Instructions for use Disposable Oxygen Hoods/Tents

Part numbers 2310001 to 23100134

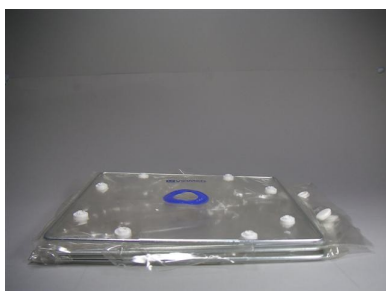

Assembly and General Instructions:

- 1) Remove oxygen hood/tent from the packaging.
- 2) Unfold oxygen hood/tent and remove press-out discs from the 22mm tubing ports where present. In certain hoods/tents such as the Double-Tent Outer, the discs have been left in place. In other models they have already been removed.
- 3) Snap the back panel into the body section using all the snap connectors. Note: If the tent is a No-Bottom, NICU or Partial Bottom design, the snap connections on the bottom edge of the back panel are not used.
- 4) The three 22mm tubing ports in the back panel of the Outer Double-Tent are available for vacuum and aerosol/humidity tubing. Note: Tubing, connectors, and filters of the user's choice will need to be added.
- 5) Assembled hoods/tents should be used according to the hospital's protocols.
- 6) Bedding should be placed between the patient and the hood/tent bottom where it exists.

 Warning.	<ul style="list-style-type: none"> • Prior to placing the patient inside the hoods/tent, the hood/tent should be inspected to ensure that the press-out discs have been removed from the hood/tent and the hood/tent area. Ensure that all the snap connections are in working order. • No smoking, no use of electrical appliances, no open flames should be allowed when oxygen is in use; only approved toys should be given to the patient • When using all types of Viamed oxygen hoods, use at least 6 litres / minute of supplied gas. • Carry out checks of the infants' blood oxygen level to be certain of adequate CO₂ washout. • For babies of less than 1.5Kg, pre-warm the oxygen hood to approximately 35°C before use. <p>Placing a cold oxygen hood over a small baby may decrease the body temperature as radiant-loss from the baby can contribute to the warming of the oxygen hood.</p>
---	--