



VIAMED Ltd.

15 Station Road, Crosshills, Keighley, West Yorks, BD20 7DT.

Website : www.viamed.co.uk.

Email : info@viamed.co.uk.

APGAR Timer.

Instructions for use.

Reset button



Start / Stop
button

1. Remove the battery box cover, pull out the battery insulating strip and refit the battery box cover.
2. Ensure the screen display reads 0:00:00.00.
3. To start or stop the timer respectively, press & release the red Start / Stop button.
4. To zero the timer, press & release the black Reset button.

If the display shows LOBAT, replace the battery with an equivalent 1.5V alkaline AA battery.

