



VIAMED Ltd.

15 Station Road, Cross Hills, Keighley, West Yorkshire, BD20 7DT, UK.

Website: www.viamed.co.uk. Email: info@viamed.co.uk.

Tel: +44 (0)1535 634542. Fax: +44 (0)1535 635582.

TOM THUMB CALIBRATION / SERVICE SHEET.

Description.	Tom Thumb.	Serial No.	
Model.	TT 490-15.	Time & Date of Service / Calibration.	

Record the manometer reading in millibars below, ensuring the TT490-15 meets the limits specified.

Test Equipment	Test	Specification	Reading	P/F
			Calibration	
Snoop liquid.	Check all ports / connections for leaks.	No bubbling.		p
Manometer	Adjustable Valve: @ 10 Lpm	Minimum: ≤ 8 cmH ₂ O.		p
Manometer	Adjustable Valve: @ 10 Lpm.	Tolerance: ≥ 42 & ≤ 47 cmH ₂ O.		p
Manometer	Precision Valve: @ 10 Lpm.	Maximum: 50 cmH ₂ O Minimum 2.0 cmH ₂ O Higher than the adjustable valve setting.		p
Manometer	Pressure Gauge Test: @ 30 cmH ₂ O.	Tolerance: ≥29 & ≤31 cm H ₂ O		p
Visual check.	Gauge cannot be removed without tools.			p
Visual check.	Gauge appears straight.			p
Visual check.	All adjustable settings are set to a minimum.			p
Visual check.	Labels are attached.	CE Label.		p
		Viamed Flowmeter Label.		
		Serial No. Label.		
		Tom Thumb Label		

Service.		Parts Replaced.	
Changed "O" Rings	√	Diamond Copper Washer	√
Re-greased Adjustable Valve Internal Parts	√	Adjustable Valve	
Re-calibrated Tom Thumb	√	Adjustable Valve Internal Parts	
		Safety Valve	
		Safety Valve Internal Parts	
		Pressure gauge	
		Pressure Gauge Face	
		"O" Rings	√
		Flow Gauge	
		Flow Gauge Cover	
		Outlet	
		Inlet	
		Blanking Bolts	
		Rail Clamp	
		Main Body Block	



Tom Thumb Resuscitation Unit Servicing Manual (TT490).
Version 1.1. 16/07/04
Part no: 0390023.

Calibration:

Signed: