

Resuscitation Rotating Acrylic Bed

Additional Information

Brief description of device:

The device is in four parts

- 1 Bed circular foundation plate:
 - 2 This white acrylic circular sheet is screwed to the door of the cabinet
 - 3 The rotating bed frame is bolted on top of the plate with a large bolt that allows the bed frame to rotate
- 4 Rotating bed frame
 - 5 The rotating frame is circular but because it requires sides and must also fit into the cabinet when the cabinet is closed there are four gaps. When the bed is locked these gaps fit around the equipment in the cabinet allowing the door to close.
 - 6 The centre of the rotating bed is a bolt which is a source of dirt collection. When the bed is cleaned this bolt should also be cleaned and checked that the bed rotates freely.
- 7 Mattress
 - 8 The mattress is round but apart from its shape should be cleaned and handled as in the general body of the manual on bed care and cleaning
- 9 Lock
 - 10 The lock is used to locate the bed in four fixed positions
 - 11 The bed should never be left between these positions either in use or in storage.
 - 12 Care should be taken to prevent the operators fingers becoming trapped in the mechanism
 - 13 Care should be taken to ensure that the patients limbs do not protrude out of the bed perimeter when the bed is being rotated