Flowsensor H

Instructions for Use – Mode d'emploi – Gebrauchsanweisung

Flow sensor

The Flowsensor H is a single patient use flowsensor used to measure the flow in the patient circuit for use with Hamilton ventilators.

Caution: Reusing the sensor may effect patient's safety due to the possible contamination of the sensor.

Ventilator connection

To connect the flowsensor to the ventilator attach the sensor tubes underneath the front panel of the ventilator. The blue tube must be attached to the "PROXIMAL" inlet of the ventilator.

Patient connection

Insert the flowsensor between the Y piece of the patient circuit and the patient connection. The side of the flowsensor equipped with the blue tube should face the patient. It is recommended that short flexible tubing is used between the sensor and the patient connection. Position the sensor with the tubing upright to prevent kinking and condensation.

Tubing clip

The tubing clip is designed to secure the flowsensor tubes to the ventilation tubes. Place the wide opening of the clip around the inspiratory ventilation tube. Place the flowsensor tube in the narrow opening of the clip, leaving a loop of tubing between the clip and the flowsensor to accommodate the full range of patient movement.

Calibration

Caution: The flowsensor must be calibrated prior to patient use.

To calibrate, follow the instructions in the appropriate ventilator manual. After successful completion of the procedure the flowsensor is ready for patient use.

Consult the ventilator's manual for detailed information!

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