

FIRE
POLICE

Department Manager

«DEPARTMENT»

«COMPANY»

«ADDRESS1»

«ADDRESS2»

«ADDRESS3»

«CITY» «STATE»

«ZIP»

Dear Sir or Madam,

**AlcoTrue M Breathalyser provides rapid and accurate
determination of breath alcohol concentration**

The AlcoTrue M is a professional handheld medical breathalyser for accurate determination of breath alcohol concentration which ~~rivals its competitors~~ and closely matches evidential devices.

The AlcoTrue M is designed for diagnostic use in ~~scenarios where alcohol may affect the behaviour of a member of the public~~ ^{and it} displays measurement of breath alcohol ~~of a patient~~ when there are concerns regarding blood alcohol level or alcohol consumption.

The main benefits include:

- **Fast warm-up time** < 6 seconds; allowing for quick and easy diagnosis.
- **Rapid, precise and reliable clinically accurate measurements** even at sub-zero ambient conditions; can be used on-site or in the field.
- **Easy to read display.**
- This high spec product provides up to five measurement modes, ideal for any situation;
 - **Active mode** – To determine the precise alcohol level in the end-expiratory breath of the person to be tested after achieving a minimal breath volume from 1.2 L. It is ensured that only deep lung breath (alveolar breath) is analysed.
 - **Fast mode** – Intended for persons suffering from COPD, Asthma or reduced lung capacity, whom might not be able to provide the standard minimum sample volume.
 - **Passive mode*** - To obtain a precise measurement, the sample uptake must be triggered when the person is breathing out. The measurement mode is to determine the quantitative value of alcohol concentration.
 - **Scan mode*** – Qualitative testing of the ambient air alcohol content, qualitative determination of alcohol content in exhaled air of a person who is not able to provide a breath sample (e.g. unconscious person), qualitative test of "non-alcoholic" drinks for alcohol content.
 - **Mouth alcohol mode*** - Consists of two tests with a determined waiting period between first and second measurement.

* Measurement modes only available if activated.

- Also available with a portable printing system.

Please find enclosed further information regarding the AlcoTrue M. Should you wish to discuss this in further detail please do not hesitate to contact a member of our sales team on 01535 634542 or by emailing alcotrue@viamed.co.uk.

There is also a smaller alternative breathalyser called the AlcoTrue C ~~available~~.
Go to www.viamed.co.uk/alcotrue-c for more information.

Yours faithfully

Catrin

Catrin Hollings
Marketing

Should you no longer wish to receive
marketing material via post,
sign-up to our email mailing list at
www.viamed.co.uk/sign-up

Larger?

