SAFEHEAT PAD

Wärme Pad

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Henan Hexin Hengda industrial Co., Ltd,
East Lake Hengda Industrial Park Shihe
District, 464000 Xinyang City, Henan, China

LOTUS GLOBAL CO., LTD.
1 Four Seasons Terrace West Drayton,
Middlesex London, UB7 9GG United Kingdom

CE 0197

SAFEHEAT PAD

Warming Pad

Please read all instructions and information before use. Then you can use the products safely from a low temperature burns or personal injury.

Characteristics:

Product name: HSO SAFEHEAT PAD

Article Number: HSOPAD01

Expiration Date and LOT Number: Separate Labelling on the Film

Average Temperature: 42-45°C

Highest Temperature: 58°C

Heat Duration: 12 hours

Raw Materials: Iron Powder, Water, Activated Carbon, Vermiculite, Super absorbent polymer, Salt

Easy to use, no odor, no microwave radiation, no stimulus to skin, natural ingredients safe and environment friendly

How to use:

1. Open the pouch and take out SAFEHEAT PAD.
2. Swing SAFEHEAT PAD for about 10 seconds.
3. Place the SAFEHEAT PAD on your body. Do not apply this warmer pad directly on the skin, put the heat pack in a blanket, pocket, case or underwear.
4. Wait for about 5-10 minutes, the SAFEHEAT PAD will heat up.
5. If SAFEHEAT PAD become hardened or not occur heat, throw it away into wastebasket.

Intended use:

- When you feel cold, for warm touching
- Multy function, relax the muscles and stimulate the blood circulation
- Medical use, Home, Camping or Outdoor use.

Do not use

- Do not use SAFEHEAT PAD on your skin directly.
- Do not use SAFEHEAT PAD with bed clothes or a heater while sleeping.
- Do not use SAFEHEAT PAD on damaged or injured skin.
- Do not use SAFEHEAT PAD if you have hypersensitive skin.
- Do not use SAFEHEAT PAD for heart and blood vessel related disease.
- Do not use SAFEHEAT PAD under 12 years old. (Children and Infant)

Precautions for low-temperature burn injury

- **A low temperature burn injury causes the symptoms of erythema and blister by connecting heating elements higher than body temperature for a long time.**
- Please note that a low temperature burn injury may occur without subjective symptoms.
- If symptoms of low temperature burn injury are shown, Stop using the heat pack and then consult a doctor.

Warning

- If you feel too warm or irritation in use, remove the heat pack immediately.
- Do not apply heat pack to same area of skin for a long time.
- Do not apply pressure on the skin in use.
- This Product is disposable. Do not attempt to use microwave for reheating.
- This product used for external use only. Do not eat or swallow contents.
- Do not punch or tear nonwoven.
- The elderly and those with temperature sense and blood disabilities, such as diabetes need to be careful, as they may feel heat different than ordinary people.
- Those who have difficulties in removing heat pack, such as elderly or handicapped persons, need to be especially careful in using them.
- Those who use the heat pack for the first time need to be careful of a low temperature injury.
- Do not attach heat pack on fur-coat and luxury clothing.
- Remove heat pack from your clothing before washing.
- Use the heat pack within the period of expiration date.

Nicht zur Wiederverwendung.
For one use, or for use on a single patient during a single procedure.

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Henan Hexin Hengda industrial Co., Ltd.
East Lake Hengda Industrial Park Shihe
District, 464000 Xinyang City, Henan, China

LOTUS GLOBAL CO., LTD.
1 Four Seasons Terrace West Drayton,
Middlesex London, UB7 9GG United Kingdom

CE 0197