

NeoMask

Phototherapy Eye Mask



Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk

Instructions for Use

1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
2. Remove the NeoMask from the packaging and disconnect the rear fastener.
3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.
Do not over tighten. Ensure that the mask is not hooked over the ears and that the ears are not folded over.
5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.
6. If necessary - for optimal fit, the upper tab can be adjusted. The two parts of this tab can be pulled apart, then laid flat against the forehead: the end with the fastener wrapped over on top of the other.

 Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.



Large **1114015:** Occipital-Frontal Circumference **32 - 38cm.** Pack of 20

Medium **1114016:** Occipital-Frontal Circumference **26 - 32cm.** Pack of 20

Small **1114017:** Occipital-Frontal Circumference **20 - 26cm.** Pack of 20



Ontecklong Medical Supply Corporation Limited.
Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe District
Nanjing, 210048 Jiangsu, P.R. China

