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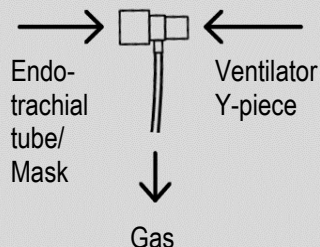
	CE mark		Consult accompanying documents
	Use by		Do not reuse
	Quantity		Storage temperature
	Catalogue number		Consult instructions for use
	Latex free		Keep away from sunlight
	Manufacturer		Date of manufacture
	Lot number		Humidity limitation
	Gas inlet		Do not connect syringe
	PHT free		Type number
	Atmospheric pressure limitation		Do not use if package is open or damaged

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## Disposable- Gas Sampling Lines H

**⚠ Warning:** Do not cut or remove any part of the sampling line. Cutting the sampling line could lead to erroneous readings.

**⚠ Warning:** Loose or damaged connections may compromise ventilation or cause an inaccurate measurement of respiratory gases. Securely connect all components and check connections for leaks according to standard clinical procedures.

**⚠ Warning:** Due to risk of cross contamination, do not re-use.

**⚠ Caution:** Do not attempt to clean, disinfect, sterilize or flush any part of the sampling line as this can cause damage to the monitor.

**⚠ Caution:** Dispose of sampling lines in accordance with standard operating procedures or local regulations for the disposal of contaminated medical waste.

**Note:** Replace the sampling line in accordance with hospital protocols or when a blockage is indicated by the device. Excessive patient secretions or a build-up of liquids in the airway tubing may occlude the sampling line, therefore requiring more frequent replacement.

### Only for sampling lines with an airway adapter:

**⚠ Warning:** When using with a closed suction system, do not place the airway adapter (tee) between the suction catheter and endotracheal tube. This is to ensure that the airway adapter does not interfere with the functioning of the suction catheter.

**Note:** During nebulization or suction, in order to avoid moisture build up and sampling line occlusion, remove the sampling line connector from the monitor.

**Note:** When connecting a sampling line to the monitor, insert the sampling line connector clockwise into the monitor CO<sub>2</sub> port and turn the connector until it can no longer be turned, so as to ensure that it is connected securely to the monitor. This will ensure that there is no leakage of gases during measurement at the connection point and that measurement accuracy is not compromised.

**Note:** Following connection of the CO<sub>2</sub> sampling line, check that CO<sub>2</sub> values appear on the monitor display.

**Note:** During setup, ensure that airway adapter can be easily attached and detached from the tubing before proceeding.

### Contraindications

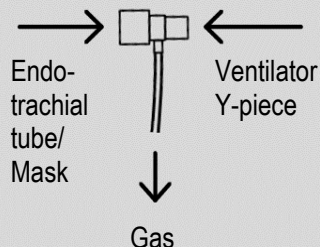
There are no known contraindications for patient monitoring with the sampling lines provided that the data obtained by the gas monitoring is evaluated with consideration given to the patients clinical condition.

**Federal law (U.S.) restricts this device to sale by or on the order of a physician.**

## Instructions for use

For reasons of hygiene, replacing sampling lines within a maximum period of 72 hours is recommended.

### Sampling line with airway adapter for intubated patients

CE	CE mark	Consult accompanying documents
Use by	Use by	Do not reuse
Quantity	Quantity	Storage temperature
Catalogue number	Catalogue number	Consult instructions for use
Latex free	Latex free	Keep away from sunlight
Manufacturer	Manufacturer	Date of manufacture
Lot number	Lot number	Humidity limitation
Gas inlet	Gas inlet	Do not connect syringe
PHT free	PHT free	Type number
Atmospheric pressure limitation	Atmospheric pressure limitation	Do not use if package is open or damaged