

Department Manager

«DEPARTMENT»

«COMPANY»

«ADDRESS1»

«ADDRESS2»

«ADDRESS3»

«CITY» «STATE»

«ZIP»



Dear Sir or Madam,

AlcoTrue M Breathalyser provides rapid and accurate determination of breath alcohol concentration

The AlcoTrue M is a professional handheld medical breathalyser for quick and accurate determination of breath alcohol concentration in the exhaled breath of patients with clinical suspicion of alcohol intoxication, which rivals its competitors and closely matches evidential devices.

The AlcoTrue M is designed for diagnostic use e.g. hospitals, rehabilitation clinics and other medical scenarios. They are used when there are concerns regarding blood alcohol level or alcohol consumption, for example in: emergency medicine, psychiatry, addiction clinics or at a methadone maintenance clinic to avoid side effects.

The main benefits include:

- Fast warm-up time < 6 seconds; allowing for quick and easy diagnosis.
- Rapid, precise and reliable clinically accurate measurements even at sub-zero ambient conditions; can be used on-site or in the field.
- Easy to read display.
- This high spec product provides up to five measurement modes, ideal for any situation;
 - **Active mode** – To determine the precise alcohol level in the end-expiratory breath of the person to be tested after achieving a minimal breath volume from 1.2 L. It is ensured that only deep lung breath (alveolar breath) is analysed.
 - **Fast mode** – Intended for persons suffering from COPD, Asthma or reduced lung capacity, whom might not be able to provide the standard minimum sample volume.
 - **Passive mode*** - To obtain a precise measurement, the sample uptake must be triggered when the person is breathing out. The measurement mode is to determine the quantitative value of alcohol concentration.
 - **Scan mode*** – Qualitative testing of the ambient air alcohol content, qualitative determination of alcohol content in exhaled air of a person who is not able to provide a breath sample (e.g. unconscious person), qualitative test of “non-alcoholic” drinks for alcohol content.
 - **Mouth alcohol mode*** - Consists of two tests with a determined waiting period between first and second measurement.

* Measurement modes only available if activated.

- Also available with a portable printing system.



Please find enclosed further information regarding the AlcoTrue M. Should you wish to discuss this in further detail please do not hesitate to contact a member of our sales team on 01535 634542 or by emailing alcotrue@viamed.co.uk.

Yours faithfully

Catrin

Catrin Hollings
Marketing

Should you no longer wish to receive marketing material via post, sign-up to our email mailing list at www.viamed.co.uk/sign-up