




## Instructions for Use

1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
2. Remove the NeoMask from the packaging and disconnect the rear fastener.
3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.  
**Do not over tighten.** Ensure that the mask is not hooked over the ears and that the ears are not folded over.
5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.

 Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.

**Large**

**1114015:** Occipital-Frontal Circumference **32 - 38cm.** Pack of 20

**Medium**

**1114016:** Occipital-Frontal Circumference **26 - 32cm.** Pack of 20

**Small**

**1114017:** Occipital-Frontal Circumference **20 - 26cm.** Pack of 20



**Onstecklong Medical Supply Corporation Limited.**  
Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe District  
Nanjing, 210048 Jiangsu, P.R. China

**LiKing GmbH**  
Fraunhofer Street 7, 04178 Leipzig, Saxony, Germany  
Tel: 0049 341 580 963 70



**20**



## Instructions for Use


1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
2. Remove the NeoMask from the packaging and disconnect the rear fastener.
3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.  
**Do not over tighten.** Ensure that the mask is not hooked over the ears and that the ears are not folded over.
5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.

Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.

**Large** 1114015: Occipital-Frontal Circumference **32 - 38cm**. Pack of 20

**Medium** 1114016: Occipital-Frontal Circumference **26 - 32cm**. Pack of 20

**Small** 1114017: Occipital-Frontal Circumference **20 - 26cm**. Pack of 20

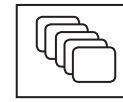

**20**



**Onstecklong Medical Supply Corporation Limited.**  
 Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe District  
 Nanjing, 210048 Jiangsu, P.R. China



**LiKing GmbH**  
 Fraunhofer Street 7, 04178 Leipzig, Saxony, Germany  
 Tel: 0049 341 580 963 70