

Department Manager  
Wiltshire Air Ambulance  
Outmarsh  
Semington  
BA14 6JX

Ref: 20210727/371

Dear Sir or Madam,

**AlcoTrue M Breathalyser provides rapid and accurate determination of breath alcohol concentration**

The AlcoTrue M is a handheld professional medical breathalyser intended for quick and accurate determination of breath alcohol concentration in the exhaled breath of patients with clinical suspicion of alcohol intoxication.

The AlcoTrue M displays measurement of breath alcohol of a patient when there are concerns regarding blood alcohol level or alcohol consumption, for example in: emergency medicine, psychiatry, addiction clinics or at a methadone maintenance clinic to avoid side effects.

The main benefits include:

- **Fast warm-up time** < 6 seconds; allowing for quick and easy diagnosis.
- **Rapid, precise and reliable clinically accurate measurements**
- **Easy to read display.**
- This high spec product provides up to five measurement modes, ideal for any situation;
  - **Active mode** – To determine the precise alcohol level in the end-expiratory breath of the person to be tested after achieving a minimal breath volume from 1.2 L. It is ensured that only deep lung breath (alveolar breath) is analysed.
  - **Fast mode** – Intended for persons suffering from COPD, Asthma or reduced lung capacity, whom might not be able to provide the standard minimum sample volume.
  - **Passive mode\*** - To obtain a precise measurement, the sample uptake must be triggered when the person is breathing out. The measurement mode is to determine the quantitative value of alcohol concentration.
  - **Scan mode\*** – Qualitative testing of the ambient air alcohol content, qualitative determination of alcohol content in exhaled air of a person who is not able to provide a breath sample (e.g. unconscious person), qualitative test of “non-alcoholic” drinks for alcohol content.
  - **Mouth alcohol mode\*** - Consists of two tests with a determined waiting period between first and second measurement.

\* Measurement modes only available if activated.

- Also available with a portable printing system.

Please find enclosed further information regarding the AlcoTrue M. Should you wish to discuss this in further detail please do not hesitate to contact a member of our sales team on 01535 634542 or by emailing [alcotrue@viamed.co.uk](mailto:alcotrue@viamed.co.uk).

Yours faithfully



Catrin Hollings  
Marketing

