

# NeoMask


Phototherapy Eye Mask



**Viamed Limited** · 15 Station Road · Cross Hills  
Keighley · West Yorkshire · BD20 7DT · United Kingdom  
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582  
Email: [info@viamed.co.uk](mailto:info@viamed.co.uk) Website: [www.viamed.co.uk](http://www.viamed.co.uk)

## Instructions for Use

1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
2. Remove the NeoMask from the packaging and disconnect the rear fastener.
3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.  
**Do not over tighten.** Ensure that the mask is not hooked over the ears and that the ears are not folded over.
5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.

 Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.

**Large Type III**

**1114015:** Occipital-Frontal Circumference **32 - 38cm.** Pack of 20

**Medium Type III**

**1114016:** Occipital-Frontal Circumference **26 - 32cm.** Pack of 20

**Small Type III**

**1114017:** Occipital-Frontal Circumference **20 - 26cm.** Pack of 20



**20**



**Onstecklong Medical Supply Corporation Limited.**  
Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe District  
Nanjing, 210048 Jiangsu, P.R. China



**Viamed Limited.**  
15 Station Road, Cross Hills,  
Keighley, West Yorkshire  
BD20 7DT, United Kingdom



**LiKing GmbH**  
Fraunhofer Street 7  
04178 Leipzig  
Saxony, Germany