

Use & Care for Blue Diamond® Bean Bags

Intended Use:

To aid in patient positioning during surgical procedures.

Prior to use:

The bladder should be visually inspected for any tears, punctures, or suspect areas.

UNIT SHOULD NOT BE USED IF POTENTIALLY COMPROMISED!!

The beads should be smoothed out to an even thickness.

General Application Usage:

Patient should be placed on the Bean Bag when it soft. The Bean Bag should be molded around the patient's body or extremity. The Bean Bag should be held firmly against the patient while it is being molded. (This may require a second assistant).

Applying Suction:

Suction should be supplied from the headwall through a flexible tube. Continue to apply pressure to the BeanBag while the air is being evacuated from the bladder.

'Click' lock the relief valve once the BeanBag is completely rigid.

NOTE: a thicker sidewall will supply a more rigid, superior support.

OPTIONAL: Leave suction line attached to BeanBag for added security.

Verify:

Confirm that BeanBag is not leaking and that desired position was achieved.

Cleansing:

The bladder is manufactured with a highly resistant urethane film. It can be cleaned with your standard OR liquid cleansers that are used on the OR table pad surface.

DAVID SCOTT COMPANY * 508-875-3333 * www.davidscottco.com