

VM3COP61.21 TOF 3D Simple Mode

- 1) Insert the batteries and power up the unit.
- 2) Press the centre key or set the time and date if necessary.



- 3) Using the left or right arrow keys, find the SET function and press the centre key.



- 4) Using the right and left arrow keys, select the parameters settings marked 'P' and press the centre key to enter.



- 5) Using the right and left arrow keys, select the setting for stimulation pulses.



- 6) Use the up and down arrow keys to change the value to --



- 7) Repeat the process for the TET and DBS settings.
- 8) The parameters setting screen should look like this.



- 9) Press the centre key to return to the operational mode.
- 10) Use the right and left arrow keys to cycle through the simplified menu.
- 11) The simplified setup is now complete.

