

# VM-2500-MG Multigas/SpO<sub>2</sub> Monitor

## Technical Data

<b>Display</b>		
Parameters displayed	Numerical Graphical Indicators	
	End-tidal (et) CO <sub>2</sub> , N <sub>2</sub> O and agent concentrations, inspired (Fi) CO <sub>2</sub> , N <sub>2</sub> O, and agent concentrations, oxygen saturation (SpO <sub>2</sub> ), Respiration Rate (RR), Pulse Rate (PR) Capnogram and trends of numerical data (15 min/1 hour/6 hours) Signal strength and signal quality, pulse amplitude, battery status, alarm mute, pulse tone mute, storage status, real-time mode, neonatal mode, time	
Characteristics	Active OLED colour display, 262,000 colours, 240 x 320 pixel (42 mm x 56 mm)	
<b>Capnography and Anaesthetic agent measurement specifications</b>		
Measurement Range	etCO <sub>2</sub> and FiCO <sub>2</sub> FiN <sub>2</sub> O Hal, Iso, Enf Sev Des Respiration Rate	0 - 15% 0 - 100% 0 - 8% 0 - 10% 0 - 22% 0 - 150 1/min
Accuracy	etCO <sub>2</sub> and FiCO <sub>2</sub> N <sub>2</sub> O Hal/Iso/Enf/Sev/Des Respiration Rate	± (0.2 Vol % + 2% of reading) ± (0.3 Vol % + 4% of reading) including interfering gases ± (2 vol % + 2% of reading) ± (2 vol % + 5% of reading) including interfering gases ± (0.15 Vol % + 5% of reading) ± (0.2 Vol % + 10% of reading) including interfering gases ± 1 digit
Warm-up time		< 20 seconds to full specification
<b>Pulse Oximetry Specifications</b>		
Measurement Range	SpO <sub>2</sub>	0 - 100%
	Pulse Rate	20 - 300 1/min
Accuracy	SpO <sub>2</sub>	± 2% (70 to 100%)
	Pulse Rate	± 1 digit (up to 100 1/min) or ± 1% (> 100 1/min)
<b>Trend Information</b>		
Long term trend		up to 150 hours
Short term trend		15 min/1 hour/6 hour
<b>Alarms</b>		
Limits		Adjustable limits for all numerical parameters except for MAC
Alarms		Audible and visual alarms (complies with EN 60601-1-8)
<b>Storing Data</b>		
Communication Interface	USB 2.0	
Data Memory Capacity	up to 150 hours	
Real-time mode		Visualisation and storage of numerical parameters on a computer every 4 seconds
Computer software		Capnograph MG PC Software for data download and real-time mode
<b>Power Supply</b>		
Battery		Working time with full functionality approximately 4.5 hours, 4 alkaline batteries (AA/LR6/AM3/MN1500/Mignon), 1.5V
Li-Poly battery		Working time with full functionality approximately 7 hours, Li-Poly battery, Model No. CT-2500, 3.7V, 2500 mAh, charging time approx 5 hours.
AC power supply		Model No. FW 7660M/06, medical power supply, input: 90 - 260V AC/50 - 60 Hz/ 250 mA, output: 6V DC/1.4A.
<b>Environmental conditions</b>		
Operation	10 - 40°C, 15 - 95% RH (non-condensing), 60 - 120 kPa (excluding Li-Poly battery)	
Storage	- 20 - 70°C, 10 - 95% RH (non-condensing), 60 - 120 kPa (excluding Li-Poly battery)	
<b>Classification</b>		
Class	IIb (according to MDD 93 / 42 / EEC)	
Safety	Class of protection II/Type BF - Type and degree of protection against electrical shock	
Construction	IPX1 (with silicone protection cover)	
Standards	EN 60601-1:1988; IEC 60601-1-2:2007; IEC 60601-1-4:2000; IEC 60601-8:2006; ISO 21647:2004; ISO 9919:2005; ISO 5356-1:2004; DIN EN 1789:2007; EN864:1996.	
<b>Dimensions and Weight</b>		
	150 (L) x 75 (W) x 35 (D) mm, < 400g (complete device with batteries)	
<b>Part Numbers</b>		
VM-2500-MG	Veterinary Capnograph with Multigas Language Package - Central European, including: English, Dutch, German, Spanish, French, Italian, Swedish and Russian	4410560



Distributed by:



**Viamed Limited** · 15 Station Road · Cross Hills  
Keighley · West Yorkshire · BD20 7DT · United Kingdom  
Tel: +44 (0)1535 634 542   Fax: +44 (0)1535 635 582  
Email: [info@viamed.co.uk](mailto:info@viamed.co.uk) Website: [www.viamed.co.uk](http://www.viamed.co.uk)



Part Number: 0092196  
Date: 01/13



BS EN ISO 9001:2008  
ISO 13485:2003

FS 26344 MD 78787

veterinary




# VM-2500-MG

## Multigas/SpO<sub>2</sub> Monitor

High performance and versatile anaesthetic agent monitoring

- Innovative micro-optic technology
- Direct mainstream measurement without time delay
- Warm-up time: < 1 minute to full specification
- Maintenance and calibration-free technology
- Colour OLED display

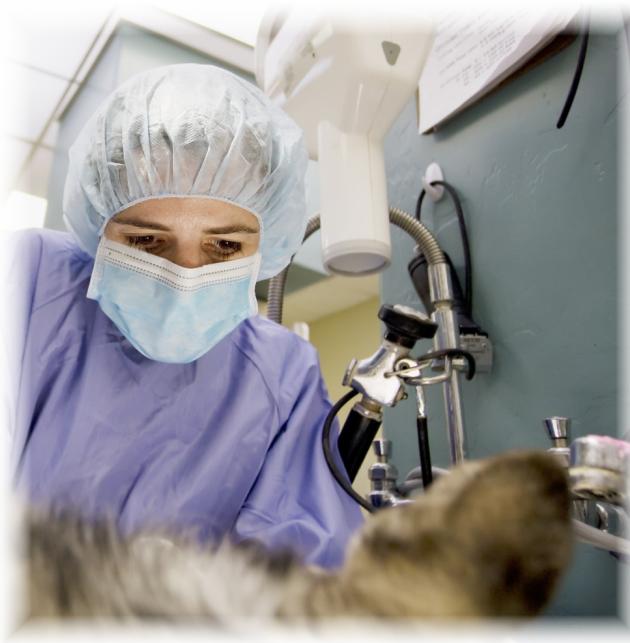

VM-2500-MG

The ultra-compact, light and easy to handle design of the Capnograph Multigas Monitor is the perfect flexible and mobile monitor for identifying and quantifying the five most important anaesthetic agents as well as other gases and parameters.

- Halothane, Isoflurane, Enflurane, Sevoflurane and Desflurane
- N<sub>2</sub>O
- etCO<sub>2</sub>, FiCO<sub>2</sub>
- Oxygen saturation
- Respiration and Pulse Rate

By direct mainstream measurement, there is no monitoring time delay.



VIAMED

**The ultra-compact, ergonomic design of the Capnograph MG combines outstanding performance and reliability in pulse oximetry and mainstream monitoring of anaesthetic agents and CO<sub>2</sub>**

## Standard Package

- Capnograph Multigas Monitor
- IRMA™ AX+ analyser
- IRMA™ Airway adapter (Small and Large)
- Reusable SpO<sub>2</sub> sensor\*
- USB data cable
- Power supply (FW7660M/06)
- Power supply plugs (UK and EU)
- 4 x AA batteries
- 1 x Li-Poly battery (CT-2500)
- Silicone protective cover
- PC software
- Instruction manual on CD

N.B. Please select either Lingual Sensor or Multi-site Y Sensor, and indicate at the time of ordering




## Reliable automatic agent identification and quantification

The **IRMA AX+ Analyser** is equipped with state-of-the-art NDIR technology with up to 9-channel gas type analysis in the 4 - 10 µm range and offers reliable agent identification and quantification, even in gas mixtures. The **IRMA AX+ Analyser** is lightweight, small and shock-resistant. It weighs less than 25g.


## Key features of the Capnograph MG

- Innovative micro-optic technology
- Direct mainstream measurement without time delay
- Compact, robust and ultra-light multigas analyser
- Warm-up time: < 1 minute to full specification
- Maintenance and calibration-free technology
- Self-explanatory, ergonomic operating functions facilitate intuitive operation
- The Colour OLED information display, as well as the simple information structure - support quick decisions and a rapid user reaction in critical situations
- Leading-edge power management with standard alkaline batteries or Li-Poly batteries or medical power supply (or combined)

## Screen Display Options

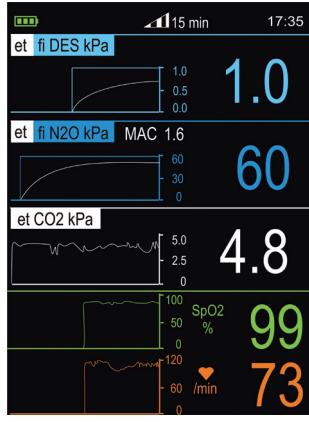

Multi Trend - 15 minutes

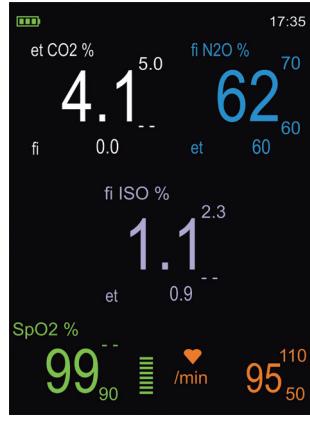

Numerical  
1 Anaesthetic Agent



Standard  
1 Anaesthetic Agent



Standard  
2 Anaesthetic Agents



## Powerful Capnograph MG PC Software

With the convenient Capnograph MG PC Software all measured values, selected alarm limits and alarm messages can be transferred to the computer via the USB interface. The data can be viewed on the computer and patient information added. The PC software enables basic statistical evaluation of the stored data.

The software can also be used to display and store the measurement values and alarm messages on the computer in parallel to ongoing measurements. This function is activated by setting the real-time mode of the Capnograph MG. When this mode is activated, the device transfers the actual measurement values for the anaesthetic agents, N<sub>2</sub>O, etCO<sub>2</sub>, FiCO<sub>2</sub>, SpO<sub>2</sub>, Respiration and Pulse Rate to the computer via the USB interface.

## Capnograph MG Multigas Analyser

The **IRMA AX+ Multigas Analyser** which weighs just 25g, sets new standards in mainstream anaesthetic agent monitoring and provides reliable and comprehensive monitoring of CO<sub>2</sub>, N<sub>2</sub>O, Halothane, Isoflurane, Enflurane, Sevoflurane and Desflurane through direct measurement without time delay.

- 9-channel NDIR gas type analysis (4 - 10 µm)
- Calibration and maintenance-free
- Compact and shock-resistant design
- Plug-and-measure technology

The **IRMA Airway Adapter** with low dead space volumes and supplied in two versions for small and large animal applications.

- Large version: dead space volume 6ml. Part Number: 4420511
- Small version: dead space volume 1ml. Part Number: 4420521
- Innovation non-condensing light transmission XTP™ window




## Applications

Capnograph MG is well suited for monitoring CO<sub>2</sub>, N<sub>2</sub>O and oxygen saturations as well as the anaesthetic agents Halothane, Isoflurane, Enflurane, Sevoflurane and Desflurane during anaesthesia and mechanical ventilation.

Capnograph MG can be used on small to large animals in veterinary surgeries, mobile operating rooms and recovery rooms.

# Carrying Case for VM-2500MG

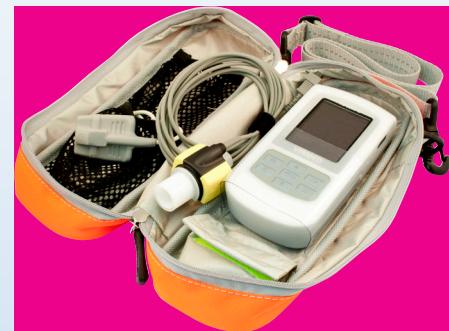

Part Number: 4420621

- Fluid repellent
- Non rot
- U.V. stable
- Washable at 60°C
- Self healing and waterproof zips
- micrAgard silver-lining inners
- Abrasion and tear resistance
- High visibility and reflective properties

Part Number: 4420620

## Mounting Options



'V' Mount Pole Clamp  
Multi-Position



'V' Mount Rail Clamp  
Small: 14 - 25mm span



'V' Mount Pole Clamp  
Large: 16 - 40mm span



Universal Mounting  
Clamp




'V' Wall Mount



Universal  
Mounting Plate

## SpO<sub>2</sub> Sensors and Accessories

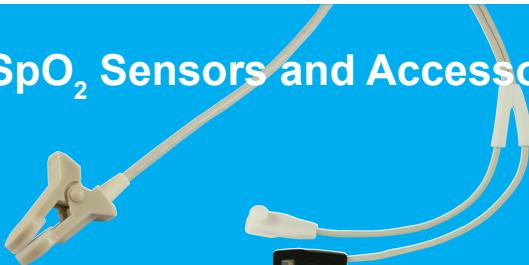

Lingual Sensor

Part Number: 0015000

Multi-site Y Sensor

Part Number: 00150002



Extension Cables

1.2m Part Number: 0014895

2.4m Part Number: 0014896

### For use with Multi-site Y Sensor



Small  
Lingual Clip

Part Number: 0025010



Medium  
Lingual Clip

Part Number: 0025011



Large  
Lingual Clip

Part Number: 0025012