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! Read through the instruction manual thoroughly before using the unit !

Supplied *Memory card* or on www.nufer-medical.ch/downloads/manual/ceratherm_600-3/

Print version: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66

	Operation: Switches unit on and off (observe chap. 8) Display: The unit's operating mode (observe chap. 8)
	Operation: Alarm confirmation (observe chap. 8) Display: Alarm (observe chap. 8 and safety guidelines in chap. 2)
	Operation: Preset heat level 1 (20% output) Display: Heat level 1 active
	Operation: Preset heat level 2 (50% output) Display: Heat level 2 active
	Operation: Preset heat level 3 (75% output) Display: Heat level 3 active
	Operation: Preset heat level 4 (98% output) Display: Heat level 4 active
	Display: Heating active (observe chap. 8.3)
	Operation: Lighting 20% on/off (observe chap. 9) Display: None
	Operation: Lighting 100% on/off (observe chap. 9) Display: None
	Operation: Master switch (supply voltage on/off, see chap. 7) Display: Mains supply on
	Potential equalisation connection. If the unit is used in a room that corresponds to Category 3 or 4 under Swiss Low-Voltage Installation Standard NIN / 7.10, this connection must be connected to the room's potential equalisation connection.

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Vers. imprimée: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66
Vers. stampata: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66