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**! Read through the instruction manual thoroughly before using the unit !**

Supplied *Memory card* or on [www.nufer-medical.ch/downloads/manual/ceratherm\\_600-3/](http://www.nufer-medical.ch/downloads/manual/ceratherm_600-3/).

Print version: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66

	<b>Operation:</b> Switches unit on and off (observe chap. 8) <b>Display:</b> The unit's operating mode (observe chap. 8)
	<b>Operation:</b> Alarm confirmation (observe chap. 8) <b>Display:</b> Alarm (observe chap. 8 and safety guidelines in chap. 2)
	<b>Operation:</b> Preset heat level 1 (20% output) <b>Display:</b> Heat level 1 active
	<b>Operation:</b> Preset heat level 2 (50% output) <b>Display:</b> Heat level 2 active
	<b>Operation:</b> Preset heat level 3 (75% output) <b>Display:</b> Heat level 3 active
	<b>Operation:</b> Preset heat level 4 (98% output) <b>Display:</b> Heat level 4 active
	<b>Display:</b> Heating active (observe chap. 8.3)
	<b>Operation:</b> Lighting 20% on/off (observe chap. 9) <b>Display:</b> None
	<b>Operation:</b> Lighting 100% on/off (observe chap. 9) <b>Display:</b> None
	<b>Operation:</b> Master switch (supply voltage on/off, see chap. 7) <b>Display:</b> Mains supply on
	Potential equalisation connection. If the unit is used in a room that corresponds to <b>Category 3 or 4</b> under Swiss Low-Voltage Installation Standard NIN / 7.10, this connection must be connected to the room's potential equalisation connection.

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Wersja: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66
Druckversion: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66
Version imprimée: ewimed Switzerland AG, Freiburgstrasse 453, CH-3018 Bern, ☎ +41 (0)31 958 66 66