


## **Ceratherm 600-3 Radiant warmer with double-joint arm, spring balanced arm and ceiling anchorage, with height adjustment, Art.-Nr. 521-60024-3**


### **Mounting instructions**

1. Check the central axis length.


1.1 The length of the central axis will be measured as follows:

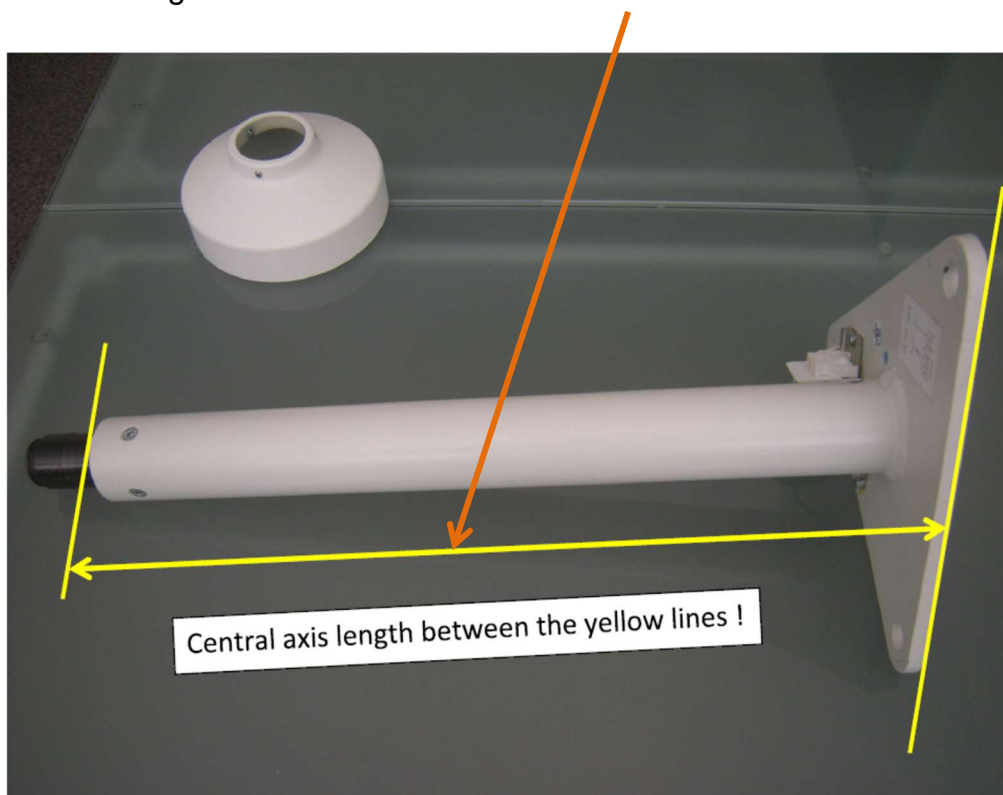

2. Assemble or disassemble the double-jointed arm on/from the central axis (depend on situation: The system will be delivered completely assembled). Assemble or disassemble at this point.

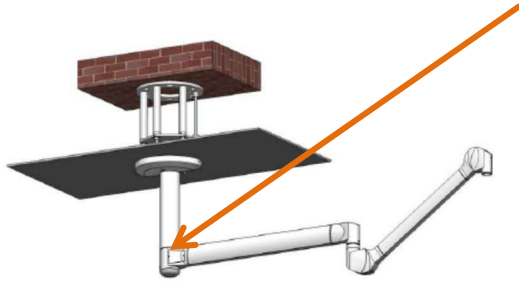


- 2.1 Remove the screw and lift off the cover with the slip rings.

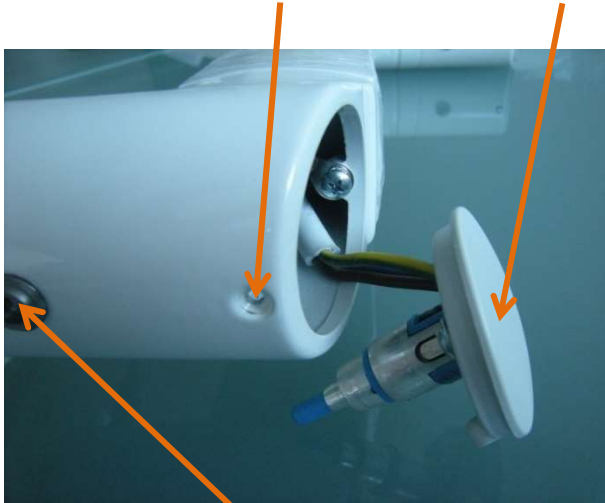

- 2.2 Remove the brake screw.

- 2.3 Remove the circlip (seger ring) with a suitable seger pliers. Remove washers and separate the double-joint arm from the central axis.

Important: Avoid inclining -> Sensitive roller bearings!

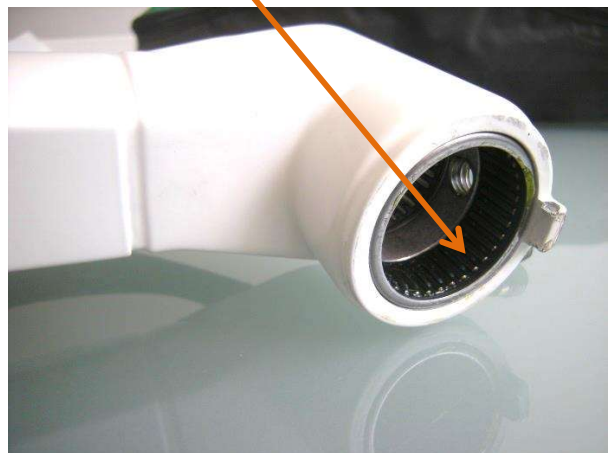

3. The assembly is carried out in reverse order.

Important: Pay attention to the arrangement of the washers. The plastic washer must be placed between the steel washers. The circlip (seger ring) must be completely located into the groove!



#### 4. Ceiling mounting

4.1 Mount the anchor plate to the load-bearing ceiling.



4.2 Mount the central axis to the threaded bolts of the anchor plate and secure it vertically align.

4.3 Connecting electrical system incl. additional equipotential bonding connection to separate terminal.



4.4 Inserting the suspended ceiling.





4.5 Slide the canopy onto the central axis and fix it with the lateral screws.



5. Mount the double-jointed arm to the central axis according to point 3.



6. Fit the brake screw and the cover.



7. Mount the infant warmer to the mounting plate using the screws provided.  
The Spring force is already preadjusted.

