

# Posey Products Revision Questions (Edition 1.0)

Please review the Introduction (DOCID 116642) and Technical (DOCID 168661) training notes in the document index and answer the following questions.

Email \*

emmaclark10@icloud.com

**Why are Posey pulse oximetry sensor wraps not used with disposable sensors? \***

- ☐ They are not adjustable.
- ☒ Disposable sensors have self-adhesive tape.
- ☐ They cause skin damage.
- ☐ They are only for paediatric use.

**Where can Posey sensor wraps be applied? \***

- ☐ Neonatal foot, infant finger, and adult wrist
- ☒ Neonatal big toe, infant foot, and paediatric or adult finger
- ☐ Adult wrist, neonatal hand, and paediatric ankle
- ☐ Paediatric toe, adult arm, and neonatal hand

**What method is used to secure Posey I.D. bracelets? \***

- ☐ Adhesive tape
- ☐ Metal clasp
- ☒ Hook and loop fasteners
- ☐ Elastic band

**Why can Posey SecureSleeves remain in place during X-ray procedures? \***

- ☐ They contain metal supports.
- ☒ They are X-ray translucent.
- ☐ They are made of plastic.
- ☐ They are disposable.

**What is the primary purpose of the Posey SecureSleeve? \***

- ☐ To secure tracheostomy tubes
- ☐ To protect against falls
- ☒ To reduce limb contractures and prevent movement during IV procedures
- ☐ To support pulse oximetry sensors

**Can Posey I.V. armboards be reused? \***

- ☒ Yes, but only for the same patient after cleaning.
- ☐ No, they are strictly single-use.
- ☐ Yes, they are designed for multiple patients.
- ☐ No, they are disposable after first use.

**Which of the following additional feature does the Posey Secure Trach Tie have? Select all correct answers. \***

- ☒ Adjustable length for comfort
- ☐ A second tie for added security
- ☐ Waterproof material
- ☒ Latex-free design

**Do any Posey products supplied by Viamed contain latex? \***

- ☐ Yes, all Posey products contain latex.
- ☒ No, all Posey products are latex-free.
- ☐ Only certain splints contain latex.
- ☐ Only Secure Trach Ties contain latex.

**What sizes are available for Posey Foam Trach Ties? \***

- ☐ Infant and paediatric only
- ☒ Small, medium, large, and extra-large
- ☐ Neonatal, paediatric, and adult
- ☐ Small and medium only

**Who is the legal manufacturer or the Posey products? \***

- ☒ Posey Ltd.
- ☐ Teledyne Analytical Industries
- ☐ Tidi Products LLC
- ☐ Viamed Ltd.

This content is neither created nor endorsed by Google.

Google Forms