

# Radiant Warmer Revision Questions (Edition 1.0)

Please review the Introduction (DOCID

182866 ) and Technical (DOCID 124724) training notes in the document index and answer the following questions.

Email \*

viamed.kate.griffiths@gmail.com

What is the default power output at **Level 3** on a Ceratherm radiant warmer? \*

- ☐ 20%
- ☐ 50%
- ☒ 75%
- ☐ 98%

What type of heating element does the Ceratherm 600-3 use? \*

- ☐ Halogen lamp
- ☐ Quartz tube
- ☒ Ceramic element
- ☐ Electric coil

What is the part number for the Ceratherm radiant warmer with mobile floor stand? \*

- ☐ 0310302
- ☐ 0310303
- ☒ 0310304
- ☐ 0310314

How long after the last key-press will the safety alarm be triggered? \*

- ☐ 5 minutes
- ☐ 10 minutes
- ☒ 15 minutes
- ☐ 20 minutes

What is the wavelength of the infrared radiation emitted by the Ceratherm 600-3? \*

- ☐ 1  $\mu\text{m}$
- ☐ 2  $\mu\text{m}$
- ☒ 3  $\mu\text{m}$
- ☐ 5  $\mu\text{m}$

What happens if the safety alarm is not cancelled within 8 seconds? \*

- ☐ The warmer turns off
- ☐ The power increases to 98% (Level 4)
- ☐ The alarm resets itself
- ☒ The power drops to 20% (Level 1)

In which of the following settings is the Ceratherm 600-3 typically used? \*

- ☐ Dental surgeries
- ☒ Delivery suites
- ☐ GP offices
- ☐ Opticians

What is the minimum clearance required from the underside of the warmer head unit to the patient surface? \*

- ☐ 60 cm
- ☐ 70 cm
- ☒ 80 cm
- ☐ 90 cm

What is the standard brightness of the low LED lamp setting on the Ceratherm 600-3? \*

- ☐ 10%
- ☒ 20%
- ☐ 50%
- ☐ 75%

What type of temperature control is used in the Ceratherm radiant warmers? \*

- ☐ Fully automatic with servo feedback
- ☒ Manual control with selectable power levels
- ☐ Smartphone app-controlled
- ☐ Thermostat-regulated only

This content is neither created nor endorsed by Google.

Google Forms