

neonatal iv arm boards



IV Arm Boards

- Helps prevent arms from bending during IV procedures
- Flexible padded board is covered with soft vinyl for patient comfort
- Disposable boards require straps provided to hold in place

Application Instructions:

1. Position the patient's arm so it lies flat on the board, palm facing up.
2. Use straps provided to secure


1114050 **Small:** Infant to 1 year (78mm (L) x 27mm (W))

1114051 **Medium:** 1 year to 2 years (100mm (L) x 38mm (W))

1114052 **Large:** 2 years to 5 years (120mm (L) x 42mm (W))

Cleaning Instructions:

Wipe clean with mild soap or disinfectant.

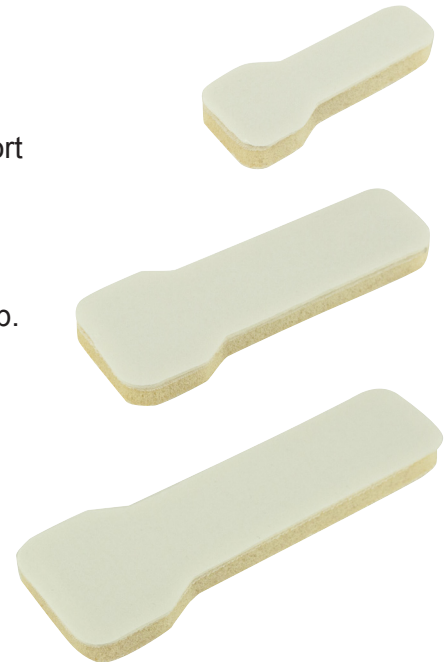

Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk