

Royal Preston Hospital - Servicing visit summary, March 2014

Order ref: 226211475

Completion date: 5th March 2014

Location: Sharoe Green Unit, Delivery Suite (12 systems) and Maternity B (1 system)

Foreword: The resuscitation cabinets at this location were installed in 2004 and are of a design that has since been superseded. Due to discontinuation by the manufacturer that Viamed commissioned to fabricate these, it is no longer possible to obtain parts for this older design. Viamed's proposed solution where a cabinet body is no longer serviceable is a replacement cabinet body of the latest design.

The engineer noted the following issues that we need to make the Trust aware of.

Room 5

Resuscitation cabinet s/n D099 - the cabinet has no door. Replacement cabinet body is required. A quotation has been issued for the supply of a cabinet body complete with equipment rails and a bed. Ref. QUO03620, dated 21/3/14.



Room 12

The radiant warmer in room 12 has cracks in the plastic cover around the screw holes and a small part of the case is loose. This does not constitute an immediate hazard as the cover is held in place by additional uncompromised screws, however, Viamed recommends that this be replaced either by the Trust's own engineers or upon the next service visit by a Viamed engineer.

The replacement casing is as follows:

| Part number | Description | Price |
|-------------|------------------|------------|
| 0330001 | Ceratherm casing | £98.00+VAT |

Note: two further radiant warmers in rooms 1 and 11 also have slightly cracked cases; these do not pose a hazard at this stage but warrant monitoring. Viamed will inspect these upon the next service visit and advise accordingly.

Room 3


Resuscitation cabinet s/n D007 - the edging strip on the door is peeling back on the leading edge. This is a primarily a cosmetic issue and does not affect the structural integrity of the cabinet, however, it may affect the ability to clean the cabinet effectively and may pose a minor risk of injury to a user.

Subject to a risk assessment by the Trust, it may be possible to glue the strip back in place with suitable glue. If repair is not considered appropriate, a replacement cabinet body offers an alternative solution.



Room – Maternity B

Resuscitation cabinet s/n D007 - has a crack across the door at the hinge, visible on the outside but not on the inside of the door. Because of the age of the cabinet we can only offer a new cabinet body as a repair but subject to a risk assessment it may be possible to repair the door.

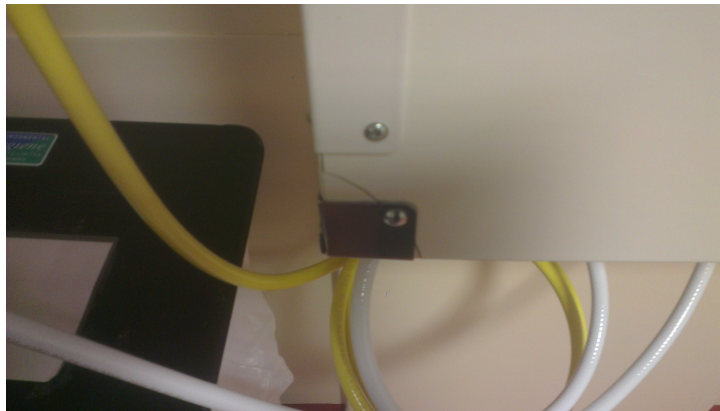

Outside view



Inside view