Instructions for use: 4000 Series Pulse Oximetry Wrap Sensors

DIRECTIONS FOR USE


Intended Use: Viamed 4000 Series oximetry sensors are indicated for use in continuous noninvasive monitoring of arterial oxygen saturation and pulse rate. They are original or fully compatible replacement sensors intended for use with major brands of pulse oximeters and patient monitors. Select the sensor model number that matches the brand of oximeter or patient monitor that you are using.

Precautions: For further instructions, warnings and precautions refer to the operating instructions provided with your oximeter. Do not autoclave.
CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

Warnings: Match the sensor model with your brand of oximeter. A mismatch may cause patient injury, inaccurate measurements or equipment damage. Check the sensor every 8 hours and reposition to an alternate location every 24 hours. Avoid application of the sensors to edematous or fragile tissue. Do not use the sensor if it is damaged. Use of a damaged sensor could cause patient injury or equipment failure. Excessive patient motion, excessive ambient light, electromagnetic interference, dysfunctional hemoglobin, low perfusion, intravascular dyes, finger nail polish and long or artificial finger nails may affect the sensor performance and the accuracy of the measurement. Note: The materials used in the manufacture of the sensor contain no natural latex protein. The materials have undergone extensive bio-compatibility testing. Further information is available upon request.

Application of the Sensor

1. Connect the sensor cable to the Viamed adaptor cable that fits your oximeter or directly to the patient monitor. Then switch on the oximeter and verify proper operation.
2. Select the sensor site on the patient. The preferred sensor sites are the index finger for adults and paediatrics, the hand for infants and on the foot below the toes for neonates.
3. Apply the sensor on the patient:
 - Position the emitter and detector directly opposite each other.
 - Apply the sensor firmly but not too tightly.
4. Secure the sensor using an adhesive or foam wrap.
5. Visually monitor the sensor site to ensure the integrity of the skin.
6. Secure the lead with medical tape.

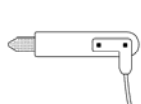Adult/Paediatric

Infant

Neonatal

Cleaning the Sensor

To clean the sensor between uses or between patients, wipe the sensor surfaces with alcohol.

Environmental Requirements:

- Operating temperature range:
10° - 40° C (50°-104° F)
- Storage and transportation temperature range
0° - 50° C (32°-122° F)
- Relative humidity:
0 - 75% non-condensing

SpO₂ Accuracy:

70 - 100%: A_{rms} <3.0%
0 - 69%: Unspecified

Accuracy test methods available from Viamed upon request.

LED Specifications

LED	Wavelength	Radiant Power (typical)	Brand
Red	660 nm	3.5 mW	All below
Infrared	885 nm	3.5 mW	Nellcor, CSI
	890 nm	3.5 mW	HP/Philips
	905 nm	3.5 mW	Datex, BCI, Viamed
	910 nm	3.5 mW	Nonin
	940 nm	3.5 mW	Ohmeda

Disposal:

Follow local regulations governing disposable recycling plans regarding disposal or recycling of device components. A list of component materials is available by contacting Viamed.

All registered trademarks recognized.

Viamed Model	Oximeter Brand	Cable Length
W4001VM	Nellcor	0.90 m
W4002VM	Nellcor	2.00 m
W4006VM	HP/Philips	2.95 m
W4007VM	HP/Philips	2.00 m
W4011VM	Datex	0.90 m
W4012VM	Datex	2.95 m
W4013VM	GE/Datex	2.95 m
W4016VM	Ohmeda	2.95 m
W4017VM	Ohmeda	1.00 m
W4021VM	BCI	1.20 m
W4026VM	Nonin	0.90 m
W4036VM	CSI	0.90 m
W4037VM	CSI	2.95 m
W6500VM	Viamed	1.20 m

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Viamed Ltd