

Parts required for Annual Service of TT480 and TT490 Tom Thumb

For the annual service of a TT480 or TT490 Tom Thumb infant resuscitator, the following parts are required:

Qty	Part No	Description
1	0330213	O-ring, thick
6	0330214	O-ring, thin
The following may be required if the gauge is not sufficiently tight:		
1	0330212	Gauge seal (diamond copper washer)

Please see separate price lists for current pricing.

For any other variants of Tom Thumb, please contact Viamed.

Notes:

The gauge seal is only required if the gauge is loose or has been removed.

This seal is a diamond-profile copper washer that it designed to compress as the gauge is tightened, allowing the gauge to be rotated up to 1 full turn from initial contact with the washer so as to allow correct alignment.

Do not re-use a gauge seal that has been previously compressed.

For variants of Tom Thumb that have an oxygen or air hose, Viamed recommends that these be replaced every 4 years.

Hoses are available in different lengths; please contact Viamed with the required gas type and hose length for a part number and price.