



**VIAMED Ltd.**

15 Station Road, Cross Hills, Keighley, West Yorkshire, BD20 7DT, UK.

Website: [www.viamed.co.uk](http://www.viamed.co.uk). Email: [info@viamed.co.uk](mailto:info@viamed.co.uk).

Tel: +44 (0)1535 634542. Fax: +44 (0)1535 635582.

**CERA THERM 600-3 radiant heater    Nufer Medical AG**

Check ID	Question / Area to Check	OK	Comments
<b>Mechanical condition</b>			
1	Check heating element for damage (cracks, chipping, loosening)		
2	Check housing and chassis (cracks, deformations)		
3	Check front film, warning signs and specification plate		
4	Tripod dolly (if present): Check rollers and brakes		
5	Tripod dolly (if present): Can be fixed in place, height can be altered		
6	Tripod dolly (if present): Check shock absorbers		
7	General checks for tripod dolly, wall and ceiling mounts (if present): Paintwork, cracks in plastic parts, deformations, loose parts, loosening		
8	Wall and ceiling mounts (if present): Check fastening element on unit mount every 24 months: thickness at least 1.5mm, length at least 11.5mm (see document 18.002.x, request if needed)		
<b>Function test</b>			
9	Check mains switch (correct functioning and display)		
10	Check buttons and displays (correct functioning and display)		
11	Measure power consumption at 240 V AC I 50 Hz: (Power consumption is variable - check the lowest value) Output level 1 Output level 4		
12	Check lighting (both levels)		
<b>Alarms</b>			
13	Acoustic and visual reminder alarm after		
14	Alarm reset, reminder alarm		
15	Acoustic and visual mains failure alarm after at least		
16	Alarm reset, acoustic mains failure alarm		
<b>Electrical safety</b>			
17	Check mains cable and unit socket for damage		
18	Electric safety test to standard EN 62353:2008		
<b>Misc./cleaning</b>			
19	Cleaning, inside and outside (remove any dust in the unit)		
20	Attach service label with new due date		
The unit has passed the safety checks and can be used for the intended purpose.			

Date:

Name:

Signature: