OEM LOGO

SpiroTrue® APC

Flow sensor

REF XXXXXXXXXXXXX

EN Instructions for use

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bluepoint MEDICAL GmbH&CO KG
An der Trave 15
23923 Selmsdorf, Germany
www.bluepoint-medical.com

0482

EN Instructions for use

SpiroTrue® APC – Flow sensor**Intended Use**

Hot wire anemometric flow sensor, for measuring gas flow in Dräger anaesthesia and ventilation equipment. The sensor is compatible to:

- **Spirolog - REF 8403735-06**
- **SpiroLife – REF MK01900**

Operation

- Due to fire risk - do not introduce any medicaments or other substances based on combustible solvents, such as alcohol, into the patient systems.
- Do not use explosive anaesthetic agents (e.g. ether or cyclopropane).
- After using with medicament nebulisers, immediately clean and treat the flow sensor as specified. Alternatively, install a filter upstream of the flow sensor.

Care

- Alignment, operation and sensor change must be carried out in accordance with the relevant device-specific instruction manual.
- Do not clean in automated cleaning or disinfection equipment. Do not clean with compressed air, water jets, brushes etc.
- Reusing the flow sensor may affect patient safety due to the possible contamination of the sensor.
- **Life cycle of the sensor is limited. It can be used as long as calibration in the anaesthesia or ventilation equipment is possible.**

Disinfection

The following instrument disinfectants can be used:

- Korsolex Basic/Extra/Plus (Bode Chemie, Hamburg, Germany)
- Gigasept FF/AF (Schülke&Mayr, Norderstedt, Germany)
- Incidin Liquid (Ecolab Deutschland GmbH, Germany)

Strictly follow the manufacturer's instructions on the disinfectant.

- Disinfect the flow sensor by immersion.
- Inspect the flow sensor for soiling and damage.
- Rinse the flow sensor under water (preferably sterile water) and then allow to dry in air for at least 30 minutes.

Sterilization

- The flow sensor **can be** autoclaved (134°C).

Disposal

The flow sensor must be disposed of as infectious waste.

Recommended storage and transport conditions

- Temperature: -20°C to 50°C
- Relative humidity: 5% to 95%, non-condensing
- Atmospheric pressure: 700hPa to 1060hPa

Recommended operating conditions

- Temperature: 10°C to 40°C
- Relative humidity: 5% to 95%, non-condensing
- Atmospheric pressure: 700hPa to 1060hPa

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