

NeoMask

Phototherapy Eye Mask

Intended Use

The NeoMask is intended to provide neonatal eye protection during phototherapy for the treatment of jaundice.



Viamed Limited · 15 Station Road · Cross Hills
Keighley · West Yorkshire · BD20 7DT · United Kingdom
Tel: +44 (0)1535 634 542 Fax: +44 (0)1535 635 582
Email: info@viamed.co.uk Website: www.viamed.co.uk

Instructions for Use

1. The NeoMask is not a bandage; cover any existing skin abrasions appropriately before use.
2. Remove the NeoMask from the packaging and disconnect the rear fastener.
3. Position the NeoMask such that the blue eye pad covers the patient's eyes. The other tab (upper) is positioned across the patient's forehead.
4. Wrap the NeoMask around the head, so that it covers the ears - then secure together with the rear fastener.
Do not over tighten. Ensure that the mask is not hooked over the ears and that the ears are not folded over.
5. Positioning of the NeoMask can be adjusted by using the front: upper and lower tabs.

 Monitor the position of the NeoMask on a regular basis and ensure that it does not obstruct the patient's nose. If the NeoMask becomes over-stretched, replace with a new one.

Type III Large


1114015: Occipital-Frontal Circumference **32 - 38cm**. Pack of 20

Type III Medium

1114016: Occipital-Frontal Circumference **26 - 32cm**. Pack of 20

Type III Small

1114017: Occipital-Frontal Circumference **20 - 26cm**. Pack of 20

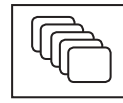

20



Onstecklong Medical Supply Corporation Limited.
Bldg 5, No. 2 Future Road, Park Zhongshan, Liuhe District
Nanjing, 210048 Jiangsu, P.R. China



Viamed Limited.
15 Station Road, Cross Hills,
Keighley, West Yorkshire
BD20 7DT, United Kingdom



LiKing GmbH
Fraunhofer Street 7
04178 Leipzig
Saxony, Germany